

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** 11-DEOXYCORTISOL
 - **Aultman Test ID:** DEOX
 - **LabCorp Test Code:** 500550
 - **Testing Facility:** LabCorp
 - **LabCorp Link:** [500550: 11-Deoxycortisol | LabCorp](#)
-

- **Test Name:** 11-DESOXYCORTISOL
 - **LC Test Code:** 500550
 - **Aultman Test Code:** DEOX
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** 17-HYDROXYCORTICOSTEROIDS
 - **LC Test Code:** 806730
 - **Aultman Test Code:** 17OHC
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** 17-HYDROXYPREGNENOLONE MS
 - **LC Test Code:** 140715
 - **Aultman Test Code:** PREGH
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** 17-OH PROGESTERONE LCMS
 - **LC Test Code:** 070085
 - **Aultman Test Code:** 17OHP
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** 18- HYDROXYCORTICOSTERONE S
 - **LC Test Code:** 500778
 - **Aultman Test Code:** 18OHC
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** 21-HYDROXYLASE ANTIBODY
 - **Test ID:** MISC
 - **Testing Facility:** Reference Laboratory
 - **Specimen Type:** SST
 - **Specimen Handling:** 1ML SERUM – REFRIGERATED
-

- **Test Name:** 21- HYDROXYLASE ANTIBODIES
 - **LC Test Code:** 504805
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** 24 HOUR URINE COLLECTION INSTRUCTIONS (For Inpatients Only)
- **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Volume:** 24 hr. urine collection
- **Specimen Handling:** Refrigerate. Do not pour it off into a secondary container. Send Out Department will process.
- **Additional Information:** All 24-hr urine specimens are to be collected in appropriate containers available from the laboratory. All 24-hr urine containers should be placed in a pink plastic bucket with ice packed around the specimen during collection. Accurate results can only be obtained if the urine is collected according to the following instructions:
 - Discard the first voided urine the morning of the test. Note the exact date and time.
 - Place all further urine specimens in the collection container.
 - Have the patient void at the exact time on the morning of the second day. Include this last specimen in the collection.
 - Note the exact date and time.
 - Send the properly labeled container with a requisition inquiry to the lab.
- **Test Name: 24 HOUR URINE COLLECTION INSTRUCTIONS (For Outpatients Only)**
- **Testing Facility:** Aultman Laboratory
- **Specimen Collection:** All 24-hr urine specimens are to be collected in appropriate containers available from the laboratory
- **Transfer Instructions:** Patient will need a brown collection container, large plastic biobag for transport, urine collection cup, and 24-Hour Urine Sample Instructions Form.

Drink the usual amount of liquid during the collection period, but NOT alcoholic beverages. Accurate results can only be obtained if the urine is collected according to the following instructions: a. Empty bladder when waking up in the morning. DO NOT COLLECT OR SAVE THIS URINE. b. Record the start date and time.

From this time on, collect and save all urine passed throughout the day and night. Pour into brown plastic container. Keep the urine refrigerated during and after collection. If this is not possible, place the brown container in a large container filled with ice. Keep the brown plastic container surrounded by ice but not totally immersed causing the lid or identification tag to become wet.

Empty bladder the next morning, collecting at the same time as above. Add this urine to the brown plastic container.

Record the stop date and time. (The start and stop time should match with a complete 24-hour collection)

record your height and weight.

Transport the brown urine container, in the large plastic biobag, to the laboratory or blood draw station as soon as possible. Mark your name and date of birth on the container.
- **Volume:** 24 hr urine collection
- **Specimen Handling:** Refrigerate or pack ice around container in a cooler.
- **Additional Information:** Blood tests may be needed in correlation with the 24-hour urine specimen, laboratory drop-offs are discouraged on Saturdays after 12 noon, or on Sundays.

-
- **Test Name: 5-HIAA PLASMA**
 - **LC Test Code:** 504510
 - **Aultman Test Code:** P5HIAA
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** A1C
 - **Test ID:** A1C

Aultman Hospital Laboratory Test Directory

- **Synonyms:** Hemoglobin A1C, glycosylated hemoglobin
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Results available on the same day. (Run's 24hrs a day, 7 days a week)
 - **Specimen Type:** Whole blood
 - **Volume:** 1.5 mL
 - **Container:** 5 mL EDTA
 - **Specimen Handling:** Store refrigerated.
-

- **Test Name:** AB SCREEN / INDIRECT AHG
 - **Specimen type:** plasma
 - **Additional Information:** Refer to comment in ABO
 - **Volume:** 6ml
 - **Container:** EDTA (pink top)
 - **TAT:** Done 7 days/week; 24 hrs./day. Results available on the same day
 - **Test includes** Blood bank will reflex additional testing when indicated
 - **Test Name:** ABO/RH
 - **Specimen type:** Plasma
 - **Volume:** 6ml
 - **Container:** EDTA (pink top)
 - **TAT:** Refer to comment in ABS
-

- **Test Name:** ACE, CSF
 - **LC Test Code:** 123231
 - **Aultman Test Code:** CACE
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** ACETAMINOPHEN
 - **Test ID:** ACETA
 - **Synonyms:** Paracetamol, Tylenol
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hrs./day. Results available on the same day.
 - **Specimen Type:** Serum.
 - **Volume:** 1 mL
 - **Container:** SST. Plain Red top. Also, acceptable plasma lithium heparin – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate.
-

- **Test Name:** ACETAZOLAMIDE (DIAMOX) S/P
 - **LC Test Code:** 811984
 - **Aultman Test Code:** ACETZ
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** ACHR- MODULATING AB

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 505225
 - **Aultman Test Code:** ACRMA
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ACHR BINDING ABS, SERUM
 - **LC Test Code:** 085902
 - **Aultman Test Code:** ACHRAB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ACHR BLOCKING ABS, SERUM
 - **LC Test Code:** 085926
 - **Aultman Test Code:** ACEBAB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ACTH, PLASMA
 - **LC Test Code:** 004440
 - **Aultman Test Code:** ACTH2
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ACYLCARNITINE PROFILE PLASMA
 - **LC Test Code:** 070228
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ADALIMUMAB DRUG + ANTIBODY
 - **LC Test Code:** 503890
 - **Aultman Test Code:** ADANEU
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ADAMTS 13 ACTIVITY REFLEX PANEL
 - **LC Test Code:** 117921
 - **Aultman Test Code:** ADM13
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ADH
 - **LC Test Code:** 010447
 - **Aultman Test Code:** AVAS
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ADIPONECTIN
 - **LC Test Code:** 004650
 - **Aultman Test Code:** ADIP
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** AHUS GENETIC ANALYSIS
- **LC Test Code:** 630320
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ASPERGILLUS ANTIBODIES BAL SERUM
 - **LC Test Code:** 183805
 - **Aultman Test Code:** ASGALS
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ASPERGILLUS AB QN DID
 - **LC Test Code:** 167389
 - **Aultman Test Code:** ASPER
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ADENOVIRUS PCR
 - **CCF Test Code:** ADEPCR
 - **Aultman Test Code:** ADEPCR
 - **LabCorp Link:** [Test Directory - Online Test Catalog](#) | [Cleveland Clinic Laboratories](#)

-
- **Test Name:** AFIRMA
 - **Reference Lab:** VERACYTE
 - **Aultman Test Name-Pathology:** AFIRMA
 - **Reference Lab Link:** [Afirma® Thyroid Molecular Diagnostics](#)

-
- **Test Name:** AFFIRM PATHOGENS DIRECT DNA PROBE
 - **Test ID:** VAGDNA
 - **Synonyms:** Bacterial Vaginosis Panel
 - **Test Includes:** Candida sp, Gardnerella vaginalis, Trichomonas vaginalis DNA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 3 days
 - **Specimen Collection:** BD Affirm VPIII Ambient Transport System
 - **Specimen Type:** Vaginal fluid
 - **Storage:** 72 hours refrigerated
 - **Additional Information:** Test does not detect Gc or clinical syndromes other than vaginitis/vaginosis. order culture gc only or n. gonorrhoeae pcr or culture wound as appropriate.

-
- **Test Name:** AGGRESSIVE B CELL LYMPHOMA FISH PROFILE
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** AGGRESSIVE B-CELL LYMPHOMA FISH
 - **Reference Lab Link:** [Test Menu](#) | [LabCorp Oncology](#)

-
- **Test Name:** ALA DELTA RANDOM URINE
 - **LC Test Code:** 007351

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** ALAU
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ALBUMIN
 - **Test ID:** ALB
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk., 24 hrs/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST) or serum
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis.
 - **Storage:** Refrigerate.
-

- **Test Name:** ALDOLASE
 - **LC Test Code:** 002030
 - **Aultman Test Code:** ALDO
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ALDOSTERONE LCMS, SERUM
 - **LC Test Code:** 004374
 - **Aultman Test Code:** ALDOS
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ALDOSTERONE RENIN RATIO
 - **LC Test Code:** 004354
 - **Aultman Test Code:** ALDREN
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ALDOSTERONE WITH NA AND K 24 HR UR
 - **CCF Test Code:** UALDOS
 - **Aultman Test Code:** UALDOS
 - **LabCorp Link:** [Test Directory - Online Test Catalog](#) | [Cleveland Clinic Laboratories](#)
-

- **Test Name:** ALK PHOS ISOENZYME
 - **LC Test Code:** 001612
 - **Aultman Test Code:** ALKISO
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

- **Test Name:** ALKALINE PHOSPHATASE, SERUM
- **Test ID:** AP
- **Synonyms:** Alk Phos.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
- **Specimen Type:** Plasma lithium heparin (PST)
- **Volume:** 1 mL

Aultman Hospital Laboratory Test Directory

- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
- **Specimen Handling:** Avoid hemolysis.
- **Storage:** Refrigerate.

-
- **Test Name:** ALK PHOSPHATASE BONE SPECIFIC
 - **LC Test Code:** 513002
 - **Aultman Test Code:** APBONE
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALLERGENS (9) NUTS
 - **LC Test Code:** 604309
 - **Aultman Test Code:** NUTPNL
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALLERGENS W/TOTAL IGE AREA 5
 - **LC Test Code:** 602632
 - **Aultman Test Code:** RESPR5
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALPHA-1-ANTITRYPSIN Phenotype
 - **LC Test Code:** 095653
 - **Aultman Test Code:** A1APHE
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALPHA FETOPROTEIN
 - **Test ID:** AFPS
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hrs/day. Results available on the same day
 - **Specimen Type:** Serum
 - **Volume:** 1 ML
 - **Container:** 7 mL SST preferred. Plain Red tube is also acceptable.
 - **Storage:** Refrigerate.

-
- **Test Name:** ALPHA-GAL IGE PANEL
 - **LC Test Code:** 650003
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALPHA SUBUNIT (FREE)
 - **LC Test Code:** 140269
 - **Aultman Test Code:** ALPSUB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALPHA-1 ANTITRYPSIN DEFICIENCY
 - **LC Test Code:** 511881

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALPHA-1-ANTITRYPSIN FECAL QN
 - **LC Test Code:** 123010
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALPHA-1-ANTITRYPSIN, SERUM
 - **LC Test Code:** 001982
 - **Aultman Test Code:** AAT
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** ALT (SGPT)
 - **Test ID:** ALT
 - **Synonyms:** SGPT
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Plasma Lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis.
 - **Storage:** Refrigerate.

-
- **Test Name:** ALZHEIMER'S EVALUATION CSF
 - **LC Test Code:** 484415
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMEBIASIS ANTIBODIES
 - **LC Test Code:** 006874
 - **Aultman Test Code:** EHISTO
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMIKACIN TROUGH SERUM
 - **LC Test Code:** 007205
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMINO ACID PROFILE QN PLASMA
 - **LC Test Code:** 700068
 - **Aultman Test Code:** PAABI
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMINO ACID PROFILE QN URINE

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 700140
- **Aultman Test Code:** UAABI
- **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMIODARONE (CORDARONE), S/P
 - **LC Test Code:** 702891
 - **Aultman Test Code:** AMIOD
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMITRIPTYLINE(ELAVIL) SERUM
 - **LC Test Code:** 0074
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMMONIA
 - **Test ID:** AMM
 - **Synonyms:** NH3
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Plasma
 - **Volume:** 2 - 4 mL
 - **Container:** 5 mL green top tube.
 - **Additional Information:** Must be collected in Outpatient Lab
 - **Specimen Handling:** Draw without clenched fist or tourniquet. Specimen should be filled completely, mix tubes by gentle inversion, place on ice, cold centrifuge and analyze within 30 minutes. Specimens for ammonia should not be drawn outside the main hospital campus. Deliver to main lab immediately on Ice. Avoid hemolysis.

-
- **Test Name:** AMPHETAMINES CONF, MS, URINE
 - **LC Test Code:** 071282
 - **Aultman Test Code:** UAMPC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMPHIPHYSIN ANTIBODY IGG
 - **LC Test Code:** 821258
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMYLASE
 - **Test ID:** AMY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL

Aultman Hospital Laboratory Test Directory

- **Container:** PST. Also, acceptable SST- Separate from cells within 2 hours of collection
- **Specimen Handling:** Avoid hemolysis.
- **Storage:** Refrigerate.

-
- **Test Name:** AMYLASE, BODY FLUID
 - **Test ID:** AMYBF
 - **Test Includes:** Body fluid type, amylase.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Body fluid
 - **Volume:** 2 - 4 mL
 - **Storage:** Refrigerate

-
- **Test Name:** AMYLASE ISOENZYMES
 - **LC Test Code:** 123110
 - **Aultman Test Code:** AMYISO
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** AMYLASE, 24 HOUR URINE
 - **Test ID:** AMU24
 - **Test Includes:** Collection time, volume, creatinine, amylase.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results are available on the same day if specimen is received before 1000.
 - **Precollection Instructions:** See 24-hr Urine Collection Instructions.
 - **Specimen Type:** 24-hr urine.
 - **Volume:** Submit entire urine collection to lab.
 - **Container:** Obtain collection container from lab.
 - **Specimen Handling:** Add no preservative. Collect on ice.
 - **Storage:** Refrigerate

-
- **Test Name:** AMYLASE, RANDOM URINE
 - **Test ID:** AMUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Random urine
 - **Volume:** 2 - 4 mL
 - **Container:** Urine tube
 - **Storage:** Refrigerate

-
- **Test Name:** ANA BY IFA SCREEN
 - **CCF Test Code:** ANAIFS

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** ANAIFA
- **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** ANAPLASMA PHAGOCYTOPHILUM AB
 - **LC Test Code:** 164672
 - **Aultman Test Code:** ANIGM
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** ANDROSTENEDIONE LCMS
 - **LC Test Code:** 004705
 - **Aultman Test Code:** ANDRO
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** ANGIOTENSIN-CONVERTING ENZYME
 - **LC Test Code:** 010116
 - **Aultman Test Code:** ACE
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** ANNATTO SEED IGE
 - **LC Test Code:** 827237
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** ANTI-AQUAPORIN 4(AQP4) CSF
 - **LC Test Code:** 505758
 - **Aultman Test Code:** NMOCSF
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** ANTIADRENAL ANTIBODIES QUANT
 - **LC Test Code:** 082024
 - **Aultman Test Code:** ADRENL
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-AQUAPORIN 4 (AQP4) CSF
 - **LC Test Code:** 505758
 - **Aultman Test Code:** NMOCSF
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-AQUAPORIN (AQP4) SERUM
 - **LC Test Code:** 505295
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTICARDIOLIPIN AB, IGA, QN

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 161836
 - **Aultman Test Code:** CARDA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTICARDIOLIPIN AB, IGG, QN
 - **LC Test Code:** 161810
 - **Aultman Test Code:** CARDIG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTICARDIOLIPIN AB, IGG, QN
 - **LC Test Code:** 161828
 - **Aultman Test Code:** CARDIM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-CENTROMERE AB BY IFA (RDL)
 - **LC Test Code:** 520065
 - **Aultman Test Code:** CENTRO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI- CHROMATIN AB IGG (RDL)
 - **LC Test Code:** 520063
 - **Aultman Test Code:** CHROMAT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-CN-1A (NT5C1A) IBM (RDL)
 - **LC Test Code:** 520061
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI- FODRIN AB, IGG, IGA (RDL)
 - **LC Test Code:** 520134
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-GBM ANTIBODIES
 - **LC Test Code:** 163050
 - **Aultman Test Code:** GLOM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTIHISTONE ANTIBODIES
 - **LC Test Code:** 012518
 - **Aultman Test Code:** HIST
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** ANTI-HMGCR AB
 - **LC Test Code:** 520385
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-HU, ANTI-RI, ANTI-YO IFA
 - **LC Test Code:** 505240
 - **Aultman Test Code:** HURIYO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-JO-1
 - **LC Test Code:** 161455
 - **Aultman Test Code:** JO1
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-MDA-5 AB (CADM-140) (RDL)
 - **LC Test Code:** 520002
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-MI-2 AB(RDL)
 - **LC Test Code:** 520000
 - **Aultman Test Code:** MI2AB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI MOG CSF
 - **LC Test Code:** 505776
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI MOG SERUM
 - **LC Test Code:** 505310
 - **Aultman Test Code:** MOGFAC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI MPO ANTIBODIES
 - **LC Test Code:** 163059
 - **Aultman Test Code:** MPO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-NMDAR SERUM
- **LC Test Code:** 505375
- **Aultman Test Code:** NMDAG
- **LabCorp Link:** [Search](#) | [Labcorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** ANTI- NXP-2 (P140) AB (RDL)
 - **LC Test Code:** 520004
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-PM/SCL-100 AB(RDL)
 - **LC Test Code:** 520007
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-SAE1 AB, IGG(RDL)
 - **LC Test Code:** 520011F
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTIPANCREATIC ISLET CELLS
 - **LC Test Code:** 160721
 - **Aultman Test Code:** ISLET
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTIPHOSPHATIDYLSERINE IGG/M/A
 - **LC Test Code:** 117994
 - **Aultman Test Code:** PHOGMA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-PLA2R
 - **LC Test Code:** 141330
 - **Aultman Test Code:** PLA2R
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-PR3 ANTIBODIES
 - **LC Test Code:** 163067
 - **Aultman Test Code:** PR3
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTI-RNA POLYMERASE III (RDL)
 - **LC Test Code:** 520021
 - **Aultman Test Code:** RNA 3
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ANTISCLERODERMA-70 ANTIBODIES
- **LC Test Code:** 018705

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** SCL70
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-SYNTHEASE PROFILE(RDL)
 - **LC Test Code:** 520193
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTITHROMBIN ANTIGEN
 - **LC Test Code:** 015057
 - **Aultman Test Code:** AT3AG
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-TH/TO AB(RDL)
 - **LC Test Code:** 520016
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-TIF-1 GAMMA AB (RDL)
 - **LC Test Code:** 520017
 - **Aultman Test Code:** MIS
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-DNASE B STREP ANTIBODIES
 - **LC Test Code:** 096289
 - **Aultman Test Code:** DNASE
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-FACTOR Xa (for Unfractionated Heparin)
 - **Test ID:** UFXA
 - **Synonyms:** Heparin Xa for unfractionated Heparin. This is not the same as Factor X.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Batched and done on Tuesday's.
 - **Specimen Type:** Citrated plasma
 - **Volume:** 1 mL
 - **Container:** 2 FULL blue top tubes
 - **Specimen Handling:** Patient should be drawn in Aultman Outpatient Department; Specimen must be processed in Aultman Coagulation Department within 1 hour of collection.

-
- **Test Name:** ANTI-FACTOR Xa (Low molecular weight heparin)
 - **Test ID:** LMXA
 - **Synonyms:** Heparin Xa for Lovenox. This is not the same as Factor X.
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** Batched and done on Tuesday's.
- **Specimen Type:** Citrated plasma
- **Volume:** 1 mL
- **Container:** 2 FULL blue top tubes
- **Specimen Handling:** Patient should be drawn in Aultman Outpatient Department; Specimen must be processed in Aultman Coagulation Department within 1 hour of collection.

-
- **Test Name:** ANTI-MULLERIAN HORMONE (AMH)
 - **LC Test Code:** 500183
 - **Aultman Test Code:** MULLER
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-SRP AB(RDL)
 - **LC Test Code:** 520014
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ANTI-THROMBIN III
 - **Test ID:** AT3
 - **Synonyms:** Anti-thrombin III activity.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Batched; ran on Tuesday's
 - **Specimen Type:** Plasma (citrated).
 - **Volume:** 2 mL.
 - **Container:** 2 FULL blue top tube (must be full)
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, it may be drawn at Aultman Outpatient Department.

-
- **Test Name:** APC RESISTANCE
 - **Test ID:** APCV
 - **Additional Information:** Factor V Leiden. This is the screening test for Factor V Leiden. Positive screens will be sent to a reference lab for PCR testing per physician approval.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Test batched and performed once per week.
 - **Specimen Type:** Plasma from 3.2% Sodium Citrate and whole blood EDTA.
 - **Volume:** 3 ml
 - **Container:** 2 full blue top tubes, 1 lavender tube.
 - **Specimen Handling:** Whole blood. Keep all tubes at room temperature. Do not refrigerate. Blood must be processed in Aultman Lab within 4 hours of collection; alternatively, it may be drawn in Aultman Outpatient Department.
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** A. PHAGOCYTOPHILUM PCR
 - **LC Test Code:** 138172
 - **Aultman Test Code:** ANAPCR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** APIXABAN ASSAY
 - **CCF Test Code:** APIXBN
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Test Directory - Online Test Catalog](#) | [Cleveland Clinic Laboratories](#)
-

- **Test Name:** APOE ALZHEIMER'S RISK
 - **LC Test Code:** 504040
 - **Aultman Test Code:** APOE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** APOLIPOPROTIEN A-1
 - **LC Test Code:** 016873
 - **Aultman Test Code:** APOA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** APOLIPOPROTEIN B
 - **Test ID:** APOB
 - **Testing Facility:** Reference Laboratory
 - **Specimen Handling:** 2 ML SERUM FROM SST TUBE - REFRIGERATED
-

- **Test Name:** APTT
 - **Test ID:** APT
 - **Test Includes:** Dosing information supplied by nursing.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 45 minutes; routine: 3 hours.
 - **Specimen Type:** Plasma (citratd).
 - **Volume:** 2 mL.
 - **Container:** 1 FULL blue top tube (MUST be full).
 - **Specimen Handling:** Non-heparinized patients: Testing must be done within 4 hours after collection in Aultman Lab; alternatively, may be drawn in Aultman Outpatient Department.
-

- **Test Name:** APTT, 50-50 MIXING STUDY
- **Test ID:** APTTM
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Urgent/STAT: 45 minutes; routine: 3 hours.
- **Specimen Type:** Plasma (citratd).
- **Volume:** 2 mL.

Aultman Hospital Laboratory Test Directory

- **Container:** 1 full blue top tube (MUST be full).
- **Specimen Handling:** Testing must be done within 4 hours after collection in Aultman Lab; alternatively, it may be drawn in Aultman Outpatient Department.

-
- **Test Name:** ARBOVIRUS AB – CSF
 - **Test ID:** MISCNB
 - **Testing Facility:** Reference Laboratory – Mayo
 - **Specimen Type:** 1 ML CSF - REFRIGERATED
 - **Additional Information:** INCLUDES IGG AND IGM FOR 4 ARBOVIRU

-
- **Test Name:** ARBO VIRUS IGM/IGM PANEL SERUM
 - **LC Test Code:** 820466
 - **Aultman Test Code:** ARBOP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ASO
 - **Test ID:** ASO
 - **Synonyms:** Anti streptolysin O.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/wk, 24hrs/day. Results available on the same day.
 - **Specimen Type:** Serum or Plasma lithium heparin – Separate from cells within 2 hours of collection.
 - **Volume:** 1 mL.
 - **Container:** SST or Plasma lithium heparin.
 - **Specimen Handling:** Avoid lipemia. Refrigerate, after 72 hours freeze.

-
- **Test Name:** ASPERGILLUS ANTIBODIES BAL SERUM
 - **LC Test Code:** 183805
 - **Aultman Test Code:** ASGALS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** AST (SGOT)
 - **Test ID:** AST
 - **Synonyms:** SGOT, aspartate transferase.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available within 1 day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL.
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis
 - **Storage:** Refrigerate.

-
- **Test Name:** ATN PROFILE

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 484400
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** AUTOIMMUNE ENCEPHALOPATHY EVALUATION SERUM
 - **CCF Test Code:** ENCSER
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Test Directory - Online Test Catalog](#) | [Cleveland Clinic Laboratories](#)

-
- **Test Name:** BABESIA DUNCANCI IGG WA1 AB
 - **LC Test Code:** 832621
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BABESIA MICROTI ANTIBODY PANEL
 - **LC Test Code:** 138315
 - **Aultman Test Code:** BMICGM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BABESIA SP DNA PCR
 - **LC Test Code:** 138318
 - **Aultman Test Code:** BABPCR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BARIUM S/P
 - **LC Test Code:** 811554
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BARTONELLA ANTIBODY PANEL
 - **LC Test Code:** 163162
 - **Aultman Test Code:** CAT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BARTONELLA DNA PCR
 - **LC Test Code:** 138350
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BASIC METABOLIC PANEL
 - **Test ID:** BMP
 - **Test Includes:** Glu, Na, K, Cl, CO2, Lyte balance, BUN, create, b/c ratio, calcium, GFR

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
- **Specimen Type:** Plasma lithium heparin (PST)
- **Volume:** 2 - 4 mL.
- **Container:** PST. Also, acceptable SST- Separate from cells within 2 hours of collection
- **Alterative Specimen:** Pasma from a lithium heparin tube, separate from cells within 2 hours of collection.
- **Specimen Handling:** Avoid hemolysis. Separate serum from cells within 2 hours of collection.
- **Storage:** Refrigerate.

-
- **Test Name:** B- CELL GENE REARRANGEMENTS PROFILE, IGH AND IGK
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** B-CELL GENE REARRANGEMENT(IGH/IGK)
 - **Reference Lab Link:** <https://www.labcorp.com/test-menu/>

-
- **Test Name:** BCR-ABL1 KINASE DOMAIN
 - **LC Test Code:** 480510
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BCR/ABL P190 QUANTITATIVE PCR BLOOD
 - **CCF Test Code:** P190PB
 - **Aultman Test Code:** P190PB
 - **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** BCR/ABL P210 QUANT BLOOD
 - **CCF Test Code:** P210PB
 - **Aultman Test Code:** P210PB
 - **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** BEACON CARRIER SCR 579 GENES
 - **LC Test Code:** 832549
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BEACON CARRIER EXPD GENE 436
 - **LC Test Code:** 831833
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BENZODIAZEPINES CONFIRM URINE
 - **LC Test Code:** 763910

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** UBENZPCM
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BENZTROPINE (COGENTIN)
 - **LC Test Code:** 803866
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BETA AMYLOID 42/40 RATIO
 - **LC Test Code:** 505725
 - **Aultman Test Code:** AMYBET
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BETA 2 MICROGLOBULIN
 - **Test ID:** B2
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/wk, 24hrs/day. Results available on the same day.
 - **Specimen Type:** plasma lithium heparin (PST)
 - **Volume:** 2 ML.
 - **Container:** PST. Also acceptable from SST- Separate from cells within 2 hours of collection

-
- **Test Name:** BETA CAROTENE
 - **LC Test Code:** 001529
 - **Aultman Test Code:** CAROT
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BETA STREP CULTURE ONLY
 - **Test ID:** BSO
 - **Synonyms:** Group A Beta Strep by culture.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 2 days.
 - **Specimen Collection:** Using a tongue blade, depress the tongue so the back of the throat is clearly visible.
 - **Transfer Instructions:** Without getting buccal or tongue contamination, insert the culturette to the back of the throat and swab both tonsillar areas, the posterior pharynx and any area of inflammation, ulceration or exudation. A good gag reflex indicates a satisfactory specimen.
3. Return swabs to culturette sheath.
 - **Specimen Type:** Throat swab.
 - **Volume:** 1 culturette (2 swabs)
 - **Container:** Culturette.
 - **Specimen Handling:** Deliver to Microbiology within 2 hours of collection for inpatients.

Aultman Hospital Laboratory Test Directory

- **Storage:** Refrigerate

-
- **Test Name:** BETA-2 GLYCOPROTEIN I AB, G/M
 - **LC Test Code:** 163002
 - **Aultman Test Code:** B2GPGM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BETA-2 GLYCOPROTEIN I AB, IGA
 - **LC Test Code:** 163900
 - **Aultman Test Code:** BETAA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BETA-2 MICROGLOBULIN URINE
 - **LC Test Code:** 010173
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BETA THALASSEMIA: HBB
 - **LC Test Code:** 252823
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** B-HYDROXYBUTYRATE
 - **Test ID:** BHB
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily
 - **Volume:** 1 mL
 - **Container:** 4mL Li Heparin green top tube preferred; SST acceptable
 - **Specimen Type:** Plasma lithium heparin (PST)

-
- **Test Name:** BILE ACIDS, FRACTIONATED LCMS
 - **LC Test Code:** 503640
 - **Aultman Test Code:** BILE
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** BILIRUBIN, DIRECT, ADULT OR INFANT
 - **Test ID:** BILAD
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** plasma lithium heparin (PST)
 - **Volume:** 1 ML
 - **Container:** PST. Also, acceptable SST- separate from cells within 2 hours of collection
 - **Specimen Handling:** Avoid hemolysis.

Aultman Hospital Laboratory Test Directory

- **Storage:** Refrigerate.

-
- **Test Name:** BILIRUBIN, INDIRECT PANEL
 - **Test ID:** BILAI
 - **Test Includes:** Total, direct and indirect bilirubin.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL.
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis. Protect specimen from light.
 - **Storage:** Refrigerate

-
- **Test Name:** BILIRUBIN, TOTAL, ADULT OR INFANT
 - **Test ID:** BILT
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL.
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis. Protect specimen from light.
 - **Storage:** Refrigerate.

-
- **Test Name:** BILIRUBIN, URINE DIPSTICK
 - **Test ID:** BILUA
 - **Synonyms:** Urine bilirubin.
 - **Test Includes:** Dipstick for urine bilirubin.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Random urine.
 - **Volume:** 2 mL.
 - **Container:** Plastic urine tube. Refrigerate or keep on ice.

-
- **Test Name:** BIOAVAILABLE TESTOSTERONE
 - **LC Test Code:** 375149
 - **Aultman Test Code:** BTESTO
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BIOPSY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday - Saturday, 2 working days. Small biopsies should be received by 4:00pm M-F, 12:00pm Saturday. Large Biopsies should be received by 12:00pm M-F, 10:30am Saturday.

Aultman Hospital Laboratory Test Directory

- **Precollection Instructions:** Use Tissue Examination Request (form 23A). If the surgeon is different than the requested physician, write the surgeon's name on the request. Include clinical diagnosis.
- **Specimen Collection /** Take specimen to Histology Department. After Histology is closed, take specimens to Microbiology. **Transfer Instructions:**
- **Container:** 10% Formalin Biopsy bottles are available in the Histology Department during department hours. After hours, biopsy bottles can be obtained from the frozen section room located off the receiving area in Surgery or in Microbiology. 10% Formalin Bottles are also available through ALS Customer Service for Physician offices.
- **Specimen Handling:** Culture:
 - If a culture is also ordered, split specimen and place part of the specimen in a sterile container for Microbiology and the rest of the specimen in 10% Formalin. If there is not enough specimen to split, do not put in formalin. Place entire specimen in a sterile container and deliver immediately to Microbiology, along with a Microbiology requisition and a Tissue Examination Request Form. If you come from a physician office and not in 10% Formalin, you must call the customer service department for a start pick-up. The specimen needs to be delivered to the lab within 2 hours.
- **Skin Biopsy for Immunofluorescence:**
 - Nerve, Muscle, Renal or Myocardial Biopsy.
 - These specimens require special handling and must be done when the Pathologist and Histotechnologist are available. Call Ext 33948 to schedule one of these tests.
- **Additional Information:** Store specimen at room temperature.

-
- **Test Name:** BKV QUANT PCR
 - **LC Test Code:** 138962
 - **Aultman Test Code:** BKQUAN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BKV QUANT PCR(URINE)
 - **LC Test Code:** 138880
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** BLADDER CANCER FISH MD REVIEW
 - **LC Test Code:** 130080
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BLASTOMYCES ABS QN DID
 - **LC Test Code:** 164293
 - **Aultman Test Code:** BLASTO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BLOOD GAS ANALYSIS
 - **Test ID:** BG
 - **Synonyms:** BG, gases, blood gas, ABGs.

Aultman Hospital Laboratory Test Directory

- **Test Includes:** pH, pCO₂, pO₂, HCO₃, total CO₂, base excess, O₂ saturation.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
- **Specimen Type:** Arterial blood.
- **Volume:** 2 - 3 mL.
- **Container:** Heparinized syringe.
- **Specimen Handling:** Collect on ice. Bring it to lab immediately.

-
- **Test Name:** BLOOD, URINE DIPSTICK
 - **Test ID:** BLDUA
 - **Synonyms:** Urinary blood/hemoglobin.
 - **Test Includes:** Dipstick for urine blood/hemoglobin.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** Random urine.
 - **Volume:** 2 mL.
 - **Container:** Plastic urine tube.
 - **Specimen Handling:** Refrigerate or keep on ice.

-
- **Test Name:** BODY FLUID FOR CYTOLOGY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm. 2 working days. Certain non-Gyn cases will require additional stains and will take longer.
 - **Precollection Instructions:** Use Form 308A, (Cytology Specimens) and Form 1745, (Body Fluid Worksheet). Mark the source of the specimen on the form and include any pertinent clinical information.
 - **Specimen Collection:** Deliver to the Cytology department. If Microbiology testing is also ordered, specimen is to be
 - **Transfer Instructions:** delivered to Microbiology first. After hours, deliver specimen to the Microbiology department.
 - **Specimen Type:** Body Fluid; Pleural, Pericardial or Peritoneal
 - **Volume:** 25 mL (Minimum: 5 mL).
 - **Container:** Collect in a clean container. If the specimen is also for culture, container must be sterile. Large sterile 2000 mL bottles are available from Central Service.
 - **Specimen Handling:** Refrigerate.

-
- **Test Name:** B. PERTUSSIS B. PARAPERTUSSIS PCR
 - **LC Test Code:** 138677
 - **Aultman Test Code:** BORAMP
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** B PERTUSSIS IGA AB

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 164384
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** B. PERTUSSIS IGM/M/A AB
 - **LC Test Code:** 164541
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** B PERTUSSIS IGG AB
 - **LC Test Code:** 161745
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** B PERTUSSIS IGG/IGM AB
 - **LC Test Code:** 163030
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BRC ASSURE COMPREHENSIVE PANEL
 - **LC Test Code:** 485030
 - **Aultman Test Code:** BRCA12
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BP180 AND BP230 IGG
 - **LC Test Code:** 163995
 - **Aultman Test Code:** BULLPE
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** B-TYPE NATRIURETIC PEPTIDE
 - **LC Test Code:** 140889
 - **Aultman Test Code:** BNPT
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** BRONCHOSCOPY SPECIMENS
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm, Saturday received by 11:00am. 2 working days. Certain non-Gyn cases will require additional stains and will take longer.
 - **Precollection Instructions:** Use Form 308A, (Cytology Specimens – one per specimen) Form 1217, (Bronchoscopy Specimen Worksheet) and Form 23A for biopsies. Include the name of the Bronchoscopist and the clinical diagnosis information on all requisitions.
 - **Specimen Type:** Bronchial Washing, Bronchial Brushing, Bronchial Biopsy, Transbronchial Needle Aspiration, Bronchoalveolar Lavage
 - **Specimen Handling:** Specimen Collection Procedure: Supplies:

Aultman Hospital Laboratory Test Directory

Cytology brush

Biopsy forceps

Small biopsy bottle with formalin

Cardboard folder for air-dried slide

Cytology container with preservative (plastic screw top with fixative) pre-filled containers can be obtained from the Cytology Department.

Needle to retrieve the biopsy specimen from the forceps

Specimen labels for each container

- **Procedure:**

Using a pencil, label one slide with two patient identifiers

After collecting the brushing sample, smear one slide. DO NOT SPRAY. This slide is for Microbiology.

Place the brush in the Cytology container with preservative and cut off the wire.

Place the biopsy sample in the small biopsy bottle containing formalin. If the biopsy is for culture, place the specimen in a sterile container with a small amount of saline.

Collect the washing or lavage in the appropriate collection container.

The Bronchoscopist should mark the Bronchoscopy Specimen Worksheet upon finishing the procedure.

Label all the specimens with two patient identifiers.

Place the air-dried slide in the cardboard folder.

Wrap the completed Tissue Requisition around the biopsy bottle and secure with a rubber band. Place all the samples and requisitions in the same container.

- **Additional Information:** Deliver to the Cytology department. After hours, deliver specimen to the Microbiology department.
- **Storage:** Refrigerate.

-
- **Test Name:** BRUCELLA ANTIBODY
 - **Test ID:** BRUC
 - **Testing Facility:** Reference Laboratory
 - **Specimen Handling:** SERUM 1 ML – REF

-
- **Test Name:** BRUCLLA ANTIBODY IGG/IGM
 - **LC Test Code:** 164632
 - **Aultman Test Code:** BRUAGG
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** BUN
 - **Test ID:** BUN
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
 - **Specimen Type:** plasma lithium heparin (PST)
 - **Volume:** 1 mL.
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate.
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** BUN, BODY FLUID
- **Test ID:** BUNBF
- **Synonyms:** Body fluid urea nitrogen.
- **Test Includes:** Body fluid type, BUN.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available the same day.
- **Specimen Type:** Body fluid.
- **Volume:** 2 - 4 mL.
- **Storage:** Refrigerate.

-
- **Test Name:** BUPRENORPHINE AND MTB SP
 - **LC Test Code:** 070103
 - **Aultman Test Code:** SBUP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** BUPRENORPHINE CONFIRM URINE
 - **LC Test Code:** 764400
 - **Aultman Test Code:** UQNTBU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** C001- IGE PENICILLOYL G
 - **LC Test Code:** 602646
 - **Aultman Test Code:** PENG
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** C002- IGE PENICILLOYL V
 - **LC Test Code:** 602647
 - **Aultman Test Code:** PENV
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** C074-IGE GELATIN
 - **LC Test Code:** 602843
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** C. DIFFICILE PCR
 - **Test ID:** CDPCR
 - **Test Includes:** Detection of Clostridium difficile toxin B gene (tcdB) by PCR.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice daily. Results available in 1 day.
 - **Precollection Instructions:** Precollection Instructions: Collect prior to therapy if possible. Include temperature, diagnosis and therapy.

Aultman Hospital Laboratory Test Directory

- **Specimen Collection :**1. Avoid contamination with water, urine or paper. Do not remove specimens from toilet bowl.
- **Transfer Instructions:**2. Collect specimens in a clean container.
- **Specimen Type:** Liquid or soft stools only. Formed stools and swabs will be rejected. A special request must be made by the physician if the stool is not of a liquid or soft consistency. Contact Microbiology at ext. 36113.
- **Volume:** 1 gram.
- **Container:** Clean container with a tight-fitting lid.
- **Specimen Handling:** Deliver to Microbiology within 2 hours of collection for inpatients. Storage: Refrigerate

-
- **Test Name:** C1 ESTERASE INHIBITOR FUNC
 - **LC Test Code:** 120220
 - **Aultman Test Code:** C1EFUN
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** C1 ESTERASE INHIBITOR SERUM
 - **LC Test Code:** 004648
 - **Aultman Test Code:** C1ESTIN
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** CA 125 TUMOR MARKER
 - **Test ID:** CA125
 - **Synonyms:** Cancer antigen 125
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Test performed 7 days per week, 24 hours per day. Results available the same day.
 - **Precollection Instructions:** It has been suggested that the assay not be performed until at least 3 weeks after the completion of primary chemotherapy and at least 2 months following abdominal surgery
 - **Specimen Type:** Serum only
 - **Volume:** 1 - 3 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis and gross lipemia
 - **Storage:** Refrigerate

-
- **Test Name:** CA 15-3
 - **Test ID:** CA15
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 7 days/wk, 24hr/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** SST
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** CA 19-9
- **Test ID:** CA19
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week. 24 hrs/day. Results available on the same day.
- **Specimen Type:** Serum
- **Volume:** 1 mL
- **Container:** 7 mL SST
- **Storage:** Refrigerate

-
- **Test Name:** CA 27-29
 - **Test ID:** CA27
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week. 24 hrs/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate

-
- **Test Name:** CADMIUM BLOOD
 - **LC Test Code:** 085340
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CALCITONIN SERUM
 - **LC Test Code:** 004895
 - **Aultman Test Code:** CALC
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** CALCITRIOL (1,25 di-OH VIT D)
 - **LC Test Code:** 081091
 - **Aultman Test Code:** VIDDH
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** CALCIUM, 24 HOUR URINE
 - **Test ID:** CAU24
 - **Test Includes:** Collection time, volume, creatinine, calcium
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Results available the same day if the specimen is received by 1000
 - **Precollection Instructions:** See 24-hr Urine Collection Instructions
 - **Specimen Type:** 24-hr urine
 - **Volume:** Submit entire collection to lab
 - **Container:** Obtain collection container from lab

Aultman Hospital Laboratory Test Directory

- **Specimen Handling:** Collect ice
- **Storage:** Refrigerate.

-
- **Test Name:** CALCIUM, IONIZED
 - **Test ID:** CAION
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day Results available the same day
 - **Specimen Type:** Whole blood (heparinized)
 - **Volume:** Tube must be full
 - **Container:** 2 mL or 4 mL green top tube
 - **Specimen Handling:** Draw specimen without tourniquet and without introducing air into specimen. Place specimen on ice. Specimen will be corrected for pH changes when analyzed but must be received within 8 hrs. Tube must be full.
 - **Storage:** Refrigerate.

-
- **Test Name:** CALCIUM, RANDOM URINE
 - **Test ID:** CAUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day Results available the same day
 - **Specimen Type:** Random urine
 - **Volume:** 2 - 4 mL
 - **Container:** Urine tube
 - **Storage:** Refrigerate.

-
- **Test Name:** CALCIUM, SERUM
 - **Test ID:** CA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** CALR MUTATION ANALYSIS
 - **LC Test Code:** 489450
 - **Aultman Test Code:** CALR
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CANDIDA ANTIBODIES IGG, IGA, IGM
 - **LC Test Code:** 163135
 - **Aultman Test Code:** CANAB
 - **LabCorp Link:** [Search | Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** ETHOSUXIMIDE (ZARONTIN) SERUM
 - **LC Test Code:** 007443
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** CANNABINOID CONFIRMATION MS UR
 - **LC Test Code:** 737735
 - **Aultman Test Code:** UTHCC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** CARASIL/CADASIL PANEL
 - **LC Test Code:** 630000
 - **Aultman Test Code:** UTHCC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** CARBAMAZEPINE
 - **Test ID:** CARB
 - **Synonyms:** Tegretol
 - **Test Includes:** Time of last dose, carbamazepine
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day Results available the same day
 - **Precollection Instructions:** Usual sampling time: trough level drawn prior (30 minutes) to next dose. A trough level approximates the lowest steady state concentration in serum between doses. Since the evening dose is often the longest dosing interval, a morning trough may be desirable but not necessary
 - **Specimen Type:** Serum or plasma lithium heparin
 - **Volume:** 1 mL
 - **Container:** SST/Plain red acceptable/plasma lithium heparin – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate.
-

- **Test Name:** CARBON DIOXIDE
 - **Test ID:** CO2
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** Plasma lithium heparin
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
-

- **Test Name:** CARBON MONOXIDE
- **Test ID:** CO
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** 2 hours
- **Specimen Type:** Whole blood (Heparinized)

Aultman Hospital Laboratory Test Directory

- **Volume:** 2 mL
- **Container:** 5 mL green top tube
- **Specimen Handling:** Return to lab immediately Do not centrifuge

-
- **Test Name:** CARNITINE TOTAL AND FREE
 - **LC Test Code:** 706500
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CATECHOLAMINES UR FREE 24 HR
 - **LC Test Code:** 004176
 - **Aultman Test Code:** CATU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** CATECHOLAMINES PLASMA
 - **LC Test Code:** 084152
 - **Aultman Test Code:** CATP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** CBC
 - **Test ID:** CBC
 - **Synonyms:** Complete blood count
 - **Test Includes:** WBC, RBC, hemoglobin, hematocrit, MCV, MCH, MCHC, RDW, platelet count, automated diff. Reflexes a manual differential if warranted
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent 1 hour; routine: 6 hours
 - **Specimen Type:** Whole blood (EDTA)
 - **Container:** 2.5 mL Lav top tube. Do not freeze
 - **Storage:** Room temperature, or if >12 hours refrigerate

-
- **Test Name:** CEA
 - **Test ID:** CEA
 - **Synonyms:** Carcinoembryonic antigen
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day
 - **Specimen Type:** Serum only
 - **Volume:** 2 - 4 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis
 - **Storage:** Refrigerate.

-
- **Test Name:** CELIAC DISEASE COMPREHENSIVE

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 165126
 - **Aultman Test Code:** CELPNL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** CELIAC DISEASE HLA DQ ASSOC
 - **LC Test Code:** 167082
 - **Aultman Test Code:** CELHLA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** CELL COUNT, BODY FLUID
 - **Test ID:** BFCT
 - **Test Includes:** Fluid type and cell count. Differential is done as warranted for dialysate fluid, pleural fluid, peritoneal fluid and pericardial fluid and shunt or ventricular fluid.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Shunt or Ventricular fluid within 1 hour. All other fluids: 4 hours.
 - **Specimen Type:** Body fluid
 - **Volume:** 3 mL
 - **Container:** Shunt or Ventricular fluid: sterile container. All other fluids: 5 mL lav top tube.
 - **Storage:** Room temperature, or if > 4 hours refrigerate.
-

- **Test Name:** CELL COUNT, CSF
 - **Test ID:** CSFCT
 - **Test Includes:** Fluid type, clarity and color, cell count. Differentials are done for CSF fluid
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** CSF: within 1 hour
 - **Specimen Type:** Body fluid
 - **Volume:** 3 mL
 - **Container:** Sterile container.
 - **Storage:** Room temperature.
-

- **Test Name:** CELL COUNT, SYNOVIAL FLUID
 - **Test ID:** SYNCT
 - **Test Includes:** Fluid type, clarity and color, cell count. Differential is done for all synovial fluids
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 4 hours
 - **Specimen Type:** Body fluid
 - **Volume:** 3 mL
 - **Container:** 5 mL Lav top tube
 - **Storage:** Room temperature, or if > 4 hours refrigerate.
-

- **Test Name:** CENOBAMATE (XCOPRI) PLASMA
- **LC Test Code:** 702735

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CERULOPLASMIN
 - **Test ID:** CERUL
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Performed 7days/wk, 24hours/day. Results available on the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** CHLAMYDIA PCR
 - **Test ID:** CTPCR
 - **Test Includes:** PCR test to detect C. trachomatis.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M - F. Results available within 1-2 days.
 - **Specimen Collection:**
 - **Female:**
 - Remove excess mucus from the cervical os and surrounding mucosa using one of the swabs provided.
 - Discard this swab.
 - Insert the second swab from the collection kit 1 - 12 cm into the endocervical canal.
 - Rotate the swab clockwise in the endocervical canal for 30 seconds to ensure adequate sampling.
 - Withdraw the swab carefully. Avoid any contact with vaginal mucosa.
 - Insert the swab in the transport tube. Snap off the shaft at score line or cut shaft to fit tube.
 - Cap tube. Label with the patient's name.
 - **Male:**
 - Collect 5 mL of urine in a sterile container.
 - Cap container. Label with the patient's name.
 - **Specimen Type:** Endocervix, vagina, urine.
 - **Volume:** 1 Cobas PCR swab, 5mL urine.
 - **Container:** Cobas PCR transport tube. NOTE: Specimens collected using the Cobas PCR collection kit cannot be used for culture.
 - **Specimen Handling:** Deliver to Microbiology
 - **Storage:** Room temperature. In urine submitted, refrigerate urine.

-
- **Test Name:** CHLAMYDIA PNEUMONIA (IGG/M)
 - **LC Test Code:** 138338
 - **Aultman Test Code:** CHLAM
 - **LabCorp Link:** [Search | Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** CHLAMYDIA TRACHOMATIS CULTURE
 - **LC Test Code:** 008565
 - **Aultman Test Code:** CTRACH
 - **LabCorp Link:** [Search | LabCorp](#)
-

- **Test Name:** CHLORIDE, 24 HOUR URINE
 - **Test ID:** CLU24
 - **Test Includes:** Collection time, volume, creatinine, chloride
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Results available the same day if specimen received by 1000
 - **Precollection Instructions:** See 24-hr Urine Collection Instructions
 - **Specimen Type:** 24-hr urine
 - **Volume:** Send entire collection to lab
 - **Container:** Obtain collection container from lab
 - **Specimen Handling:** Collect on ice. No preservatives
-

- **Name:** CHLORIDE, BODY FLUID
 - **Test ID:** CLBF
 - **Test Includes:** Body fluid type, chloride
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** Body fluid
 - **Volume:** 2 - 4 mL
-

- **Test Name:** CHLORIDE, CSF
 - **Test ID:** CLCSF
 - **Test Includes:** Body fluid type, chloride
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** CSF
 - **Volume:** 2 - 4 mL
-

- **Test Name:** CHLORIDE, RANDOM URINE
 - **Test ID:** CLUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** Random urine
 - **Volume:** 2 - 4 mL
 - **Container:** Plastic urine tube
-

- **Test Name:** CHLORIDE, SERUM
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** Plasma lithium heparin (PST)
- **Volume:** 1 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
- **Storage:** Refrigerate.

-
- **Test Name:** CHL PNEUMONIAE (IGG/IGM/IGA)
 - **LC Test Code:** 139317
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CHOLINESTERASE PLASMA
 - **LC Test Code:** 160200
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CHOLESTEROL, SERUM
 - **Test ID:** CHOL
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection
 - **Storage:** Refrigerate.

-
- **Test Name:** CHROMIUM PLASMA
 - **LC Test Code:** 071522
 - **Aultman Test Code:** CHRSER
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CHROMIUM AND COBALT WB (MOM)
 - **LC Test Code:** 738770
 - **Aultman Test Code:** CHROBALT
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CHROMOGRANIN A
 - **LC Test Code:** 140848
 - **Aultman Test Code:** CHROMA
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** CHROMOSOME BLOOD
 - **CCF Test Code:** CHRBLD
 - **Aultman Test Code:** CGBKC
 - **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** CIRCULATING LUPUS ANTICOAGULANTS
- **Test ID:** CIRAN
- **Synonyms:** Circulating anticoagulants, lupus anticoagulants, lupus antibody, DRVVT Screen & Confirm, DRVVT Screen with reflex testing.
- **Test Includes:** Platelet neutralization, circulating antibodies
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Batched and done on Tuesday
- **Specimen Type:** Citrated plasma
- **Volume:** 4 mL
- **Container:** (3) 5ml blue top tubes (**MUST be full**)
- **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, may be drawn in Aultman Outpatient Lab

-
- **Test Name:** CITRIC ACID (CITRATE) RANDOM U
 - **LC Test Code:** 123866
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CITRIC ACID (CITRATE) URINE
 - **LC Test Code:** 016865
 - **Aultman Test Code:** CITU
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CK, TOTAL+ ISO ENZYMES, SERUM
 - **LC Test Code:** 002154
 - **Aultman Test Code:** CKISO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** CKMB PROFILE
 - **Test ID:** CKMB
 - **Test Includes:** Total CK, CKMB, relative index
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available within 2 hours after they received in lab
 - **Specimen Type:** Heparinized plasma for CKMB & CK
 - **Volume:** 1mL serum; 1 mL plasma
 - **Container:** PST
 - **Specimen Handling:** Avoid hemolysis
 - **Storage:** Refrigerate

-
- **Test Name:** CLOBAZAM (ONFI)
 - **LC Test Code:** 790500
 - **Aultman Test Code:** CLOBAZ

Aultman Hospital Laboratory Test Directory

- **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CLOMIPRAMINE SERUM
 - **LC Test Code:** 706465
 - **Aultman Test Code:** CLOM
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CLOZAPINE (CLOZARIL) SERUM
 - **LC Test Code:** 706440
 - **Aultman Test Code:** CLOZ
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CMA QUANT DNA PCR (PLASMA)
 - **LC Test Code:** 139149
 - **Aultman Test Code:** CMVQNT
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CMV ANTIBODY
 - **Test ID:** CMV
 - **Synonyms:** Cytomegalovirus titer
 - **Test Includes:** CMV IgG and IgM Acute/convalescent specimens are tested if requested and if there was a CMV order in the past 2 Mos
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly Monday through Friday. Results available within 3 days
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** CMV IGG/AVIDITY
 - **LC Test Code:** 006027
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CMV PCR
 - **LC Test Code:** 138693
 - **Aultman Test Code:** CMVCSF
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CMV QUANT DNA PCR(URINE)
 - **LC Test Code:** 139144
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** COAGULATION PANEL
- **Test Includes:** APTT, protime, fibrinogen, platelet count
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** 1 hour
- **Specimen Type:** Citrated plasma and EDTA whole blood
- **Container:** 1 EDTA tube and 1 blue top tube (must be full)
- **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, may be drawn in Aultman Outpatient Lab

-
- **Test Name:** COCAINE, URINE SCREEN
 - **Test ID:** COC
 - **Test Includes:** Screening for cocaine Does not include confirmation
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 1 hour
 - **Specimen Type:** Random urine
 - **Volume:** 5 mL
 - **Container:** Urine tube

-
- **Test Name:** COCCIDIoidES ABS IGG/IGM EIA
 - **LC Test Code:** 164798
 - **Aultman Test Code:** COCIMG
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** COENZYME Q10 TOTAL
 - **LC Test Code:** 120251
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** COLD AGGULTININ TITER QUANT
 - **LC Test Code:** 006353
 - **Aultman Test Code:** CAGGL
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** COMPLEMENT C1Q QUANTITATIVE
 - **LC Test Code:** 016824
 - **Aultman Test Code:** C1Q
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** COMPLEMENT C3/C4
 - **Test ID:** C3C4A
 - **Synonyms:** Complement studies
 - **Test Includes:** Complement C3 and C4
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** Performed 7days/wk, 24hours/day. Results available on the same day.
- **Specimen Type:** Serum only
- **Volume:** 1 mL
- **Container:** 7 mL SST

-
- **Test Name:** COMPLEMENT TOTAL (CH50)
 - **LC Test Code:** 001941
 - **Aultman Test Code:** COMPD
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** COMPREHENSIVE AUTOIMMUNE PANEL (See Lupus Analyzer)
 - **Test Includes:** Antinuclear Antibody, ENA Panel, SCL-70 Ab, Microsomal Ab, Complement C3/C4, Rheumatoid Factor, Mitochondrial Ab, Parietal Cell Ab, Ribosomal Ab, Smooth Muscle Ab, Cyclic Citrullinated Peptide
 - **Testing Facility:** Aultman Laboratory

-
- **Test Name:** COMPLIANCE DRUG ANALYSIS, UR
 - **LC Test Code:** 790600
 - **Aultman Test Code:** PMDS
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** COMPREHENSIVE METABOLIC PANEL
 - **Test ID:** CMP
 - **Test Includes:** Glu, Na K, Cl, BUN, creat, b/c ratio, calcium, protein, albumin, globulin, a/g ratio, total Bili, alk phos, AST
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** plasma lithium heparin (PST)
 - **Volume:** 2-4 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis. Separate from cells within 2 hours
 - **Storage:** Refrigerate.

-
- **Test Name:** CONVENTIONAL PAP SMEAR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm. 4 working days. Abnormal PAP smear results take longer pending Pathologist review.
 - **Precollection Instructions:** Requisition: In-patient Pap smears should be ordered in Cerner. Use the source "Pap smear". The Cerner requisition will accompany the slides to Cytology. Out-patients are ordered on paper requisitions, Cytology Specimen Req. Form 308A.
 - **Specimen Collection:** Diagnostic accuracy is dependent on factors such as the site from which the specimen is obtained

Aultman Hospital Laboratory Test Directory

- **Transfer Instructions:** the conditions under which the specimen is obtained (i.e. menstruation, lubricant contamination, glove powder contamination), and the technique used to obtain the sample.
 - Materials need to obtain a conventional pap smear:
 - Cytology requisition Form 308A for out-patients
 - Cerner requisition for in-patients
 - Vaginal speculum
 - Bottle of spray fixative (can be obtained from Cytology Dept., McKinley 374)
 - Pink frosted-end slides
 - Lead pencil to write two patient identifiers and source on the label-end.
 - Cytology requisition Form 308A, cardboard folders and specimen collection devices are available through ALS for physician offices. Materials for in-house patients can be obtained from Central Service.
 - Proper labeling of specimen slides:
 - Slides must be labeled with two patient identifiers and the source of the specimen (i.e. cx, endocx, vag, etc) on the frosted end of the slide. Please use lead pencil, not ink.
- **Please include the following information on the requisition:**
 - Two patient identifiers
 - The doctor submitting the Pap smear
 - The patient's LMP
 - Pertinent clinical history (history of abnormal bleeding, previous abnormalities, previous cancer treatments, birth control or hormone replacement, hysterectomy)
 - The date of the procedure
 - Patient's gender
- **Preparation:** Before using the spray fixative, "prime" the bottle by pumping the nozzle a few times to bring the fixative into the tubing. It is extremely important that you do not let the cellular material on the slides air dry. If the nozzle is clogged, clean it with alcohol or water.
- **Collection procedure:**
 - NOTE: The Pap smear collection must be performed by a physician or a nurse practitioner.
 - Place the patient on her back.
 - Gently insert the vaginal speculum. Warm water can be used to lubricate and warm up the speculum. Do not use lubricating jelly because this will. For a sample to be considered adequate, it is necessary to obtain a specimen from the endocervix and exocervix for patients with a cervix intact.
 - Exocervical samples: use either a wooden or plastic collection device to gently scrape the area.
 - Endocervical samples: The Transformation zone must be sampled by using an endocervical brush.
 - Vaginal samples can be obtained using a spatula.
 - Evenly spread the cellular material on all slides being submitted and spray fix immediately.
 - When spraying the slides, hold the bottle of fixative about 6-8 inches from the slide and give it 3 quick sprays. Holding the bottle too close will result in dispersing the cellular material. Holding the bottle too far away will result in air-drying.
- **Specimen Handling:** Conventional Pap smears are kept at room temperature. It is not necessary to refrigerate the slides, but they can be. Place the properly labeled dried fixed glass slides in the

Aultman Hospital Laboratory Test Directory

cardboard folders. Place the cardboard folder in a biohazard bag and seal it. Place the appropriate paperwork in the pouch on the outside of the bag.

- Out-patient Pap smears will be picked up and delivered by courier to Cytology.
- In-patient Pap smears can be sent to Cytology via the tube system (832) or hand-delivered directly to Cytology.

-
- **Test Name:** COPPER RANDOM URINE
 - **LC Test Code:** 383700
 - **Aultman Test Code:** UCOPR
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** COPPER, SERUM OR PLASMA
 - **LC Test Code:** 001586
 - **Aultman Test Code:** CUS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** COPPER URINE
 - **LC Test Code:** 003343
 - **Aultman Test Code:** CUU
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CORTISOL URINARY FREE
 - **LC Test Code:** 004432
 - **Aultman Test Code:** CORTU
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** CORTISOL FREE DIALYSIS LCMS
 - **LC Test Code:** 504020
 - **Aultman Test Code:** FRCORT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** CORTISOL, SERUM
 - **Test ID:** CORT
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 24 hours/day 7 days per week. Results available on the same day
 - **Specimen Type:** Serum
 - **Volume:** 1 - 2 mL
 - **Container:** 7ml Plain Red top or SST.
 - **Specimen Handling:** If drawn in red top: Centrifuge and transfer serum to plastic vial.
 - **Storage:** Refrigerate.

-
- **Test Name:** CORTISONE SERUM LCMS
 - **LC Test Code:** 503725

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** COVID SARS PCR Screen (AH ONLY)
 - **Test ID:** COVPCR
 - **Test Includes:** Detection of SARS-COV-2 (COVID-19)
 - **Turnaround Time:** 2 hours. Offered 24 hours a day.
 - **Specimen Type:** NPH SWAB
 - **Container:** Viral Transport Media
 - **Storage:** Refrigerate

-
- **Test Name:** COVID/FLU/RSV PCR Screen
 - **Test ID:** CVFLURV
 - **Test Includes:** Detection of SARS-COV-2 (COVID-19) Influenza a, Influenza B, and RSV
 - **Turnaround Time:** 2 hours. Offered 24 hours a day
 - **Specimen Type:** NPH Swab
 - **Container:** Viral Transport Media
 - **Storage:** Refrigerate

- **Test Name:** COXSACKIE B VIRUS ANTIBODIES
- **LC Test Code:** 096001
- **Aultman Test Code:** COXB
- **LabCorp Link:** [Search | LabCorp](#)

- **Test Name:** COXSACKIE A IGG/IGM ANTIBODY
- **LC Test Code:** 163295
- **Aultman Test Code:** COXA
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** C-PEPTIDE
 - **Test ID:** CPEP
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Performed 7days/wk, 24hours/day. Results available on the same day.
 - **Precollection Instructions:** Fasting specimen
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** 10 mL red top or 7 ml SST tube
 - **Specimen Handling:** If drawn in red top: Centrifuge and transfer serum to plastic vial. Freeze serum.

-
- **Test Name:** CPK, SERUM
 - **Test ID:** CK
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
- **Specimen Type:** plasma lithium heparin (PST)
- **Volume:** 1 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
- **Specimen Handling:** Avoid hemolysis
- **Storage:** Refrigerate.

-
- **Test name: C-REACTIVE PROTEIN**
 - **Test ID:** CRP
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** SST

-
- **Test Name: C-REACTIVE PROTEIN, HIGHLY SENSITIVE**
 - **Test ID:** CRPHS
 - **Synonyms:** Cardiac CRP
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days per week, 24 hours per day. Results available on the same day.
 - **Specimen Type:** Serum or plasma lithium heparin – Separate from cells within 2 hours of collection.
 - **Volume:** 1 mL
 - **Container:** SST or Plasma Lithium Heparin
 - **Specimen Handling:** Centrifuge and separate serum from cells within 2 hours
 - **Storage:** Refrigerate

- **Test Name: CREATININE CLEARANCE**
- **Test ID:** CRCL
- **Test Includes:** Collection time, volume, urine creatinine, serum creatinine, creatinine clearance
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily Results available the same day if specimen received by 1000
- **Precollection Instructions:** See 24-hr Urine Collection Instructions Patient's height (inches), weight (lbs.) and collection period (hrs) is required
- **Specimen Type:** 24-hr urine, serum
- **Volume:** Urine: submit entire collection to lab; serum: 1 mL
- **Container:** Urine: obtain collection container from lab; serum: 7 mL SST
- **Specimen Handling:** Collect urine on ice Use no preservative
- **Additional Information:** Serum creatinine must be drawn within 24 hours of urine collection

-
- **Test Name: CREATININE, 24 HOUR URINE**
 - **Test ID:** CRU24
 - **Test Includes:** Collection time, volume, creatinine

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily. Results available the same day if specimen is received by 1000
- **Precollection Instructions:** See 24-hr Urine Collection Instructions
- **Specimen Type:** 24-hr urine
- **Volume:** Submit entire collection to lab
- **Container:** Obtain collection container from lab
- **Specimen Handling:** Collect on ice Use no preservative

-
- **Test Name:** CREATININE, BODY FLUID
 - **Test ID:** CRBF
 - **Test Includes:** Body fluid type, creatinine
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day Results available same day
 - **Specimen Type:** Body fluid
 - **Volume:** 2 - 4 mL

-
- **Test Name:** CREATININE, RANDOM URINE
 - **Test ID:** CRUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day Results available same day
 - **Specimen Type:** Random urine
 - **Volume:** 2 - 4 mL
 - **Container:** Urine tube

-
- **Test Name:** CREATININE, SERUM
 - **Test ID:** CRE
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/wk, 24 hrs/day. Results available on the same day
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST- Separate from cells within 2 hours of collection
 - **Storage:** Refrigerate.

-
- **Test Name:** CRYOGLOBULIN
 - **Test ID:** CRYO
 - **Test Includes:** This is the Qualitative test
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Within 48 hours
 - **Precollection Instructions:** Prewarm tube to 37C
 - **Specimen Type:** Serum
 - **Volume:** 5 mL

Aultman Hospital Laboratory Test Directory

- **Container:** Red top tube (DO NOT use SST)
- **Specimen Handling:** Immediately after drawing, place tube in a cup of 37C water and return it to the lab ASAP. If drawing outside the hospital, keep tube in 37o water for one hour before centrifuging. Centrifuge and transfer serum to plastic vial Keep at room temp

-
- **Test Name:** CRYPTOCOCCUS ANTIGEN, CSF
 - **Test ID:** CRYPC
 - **Synonyms:** Cryptococcal antigen, Crypto-LA
 - **Test Includes:** Latex agglutination test for Cryptococcus antigen
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available on the same day
 - **Specimen Type:** CSF
 - **Volume:** 0.2 mL
 - **Container:** CSF collection tube #3
 - **Specimen Handling:** Collect aseptically according to established procedures

-
- **Test Name:** CRYPTOCOCCUS ANTIGEN, SERUM
 - **Test ID:** CRYPS
 - **Synonyms:** Cryptococcal antigen, Crypto-LA
 - **Test Includes:** Latex agglutination test for Cryptococcus antigen
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available within 24 hours
 - **Precollection Instructions:** Fasting specimens to avoid lipemia
 - **Specimen Type:** Serum or CSF
 - **Volume:** 0.5 mL
 - **Container:** Serum: 7 mL SST, CSF: Sterile tube
 - **CSF:** Sterile tube
 - **Specimen Handling:** Avoid hemolysis. Refrigerate

-
- **Test Name:** CRYPTOSPORIDIUM AG SCREEN, STOOL
 - **Test ID:** Crypt
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 48-72 HOURS
 - **Specimen Handling:** Refrigerate
 - **Additional Information:** Sterile Container or SAF

-
- **Test Name:** CRYSTALS, BODY FLUID
 - **Test ID:** CRYBF
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 24-48 hours
 - **Specimen Type:** Synovial Fluid
 - **Volume:** 2 - 4 mL
 - **Container:** EDTA tube

Aultman Hospital Laboratory Test Directory

- **Storage:** Room temperature.

-
- **Test Name:** CT/GC NAA, PHARYNGEAL
 - **LC Test Code:** 188698
 - **Aultman Test Code:** GCCTP
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** CT/GC NAA, RECTAL
 - **LC Test Code:** 188672
 - **Aultman Test Code:** GCCTR
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** C-TELEPEPTIDE SERUM
 - **LC Test Code:** 500089
 - **Aultman Test Code:** CTELO
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** CULTURE ACID FAST
 - **Test ID:** CAF
 - **Test Includes:** Microscopic examination and culture for Mycobacteria.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Setup M, W, F. Stain is also done STAT by request. Culture results are available in 8 weeks. Results of the stain are available on the same day.
 - **Specimen Type:** Sputum, stool, urine, CSF, wounds, body fluids, tissues, blood, bronch wash.
 - **Volume:** Fluids: 10 mL; wounds: 2 swabs; tissue: 1 cm.
 - **Container:** Sterile leak-proof container or double culturette.
 - **Specimen Handling:** Collection:
 - Sputum: See CULTURE RESPIRATORY, LOWER.
 - Stool: See CULTURE STOOL.
 - CSF: See CULTURE SPINAL FLUID.
 - Wounds: See CULTURE WOUND.
 - Body fluids: See CULTURE BODY FLUID.
 - Tissue: See CULTURE TISSUE.
 - Blood: See CULTURE BLOOD.
 - Urine: See CULTURE URINE.
 - Bronch wash: See CULTURE RESPIRATORY, LOWER.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate: stool, sputum, urine. Room temperature: Wound, CSF, tissue, blood, body fluids

-
- **Test Name:** CULTURE BLOOD
 - **Test ID:** CBL

Aultman Hospital Laboratory Test Directory

- **Test Includes:** Aerobic and anaerobic culture for pathogens in the blood. Sensitivity testing is performed if indicated.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily. Results available in 5 days.
- **Precollection Instructions:** For a diagnosis of rules out fungus, order a routine blood culture and enter the comment Hold for fungus.
- For Mycobacteria requests, draw 2 5mL green top vacutainers in addition to blood culture bottles.
- Current recommendations are that 2 blood cultures be collected over a 24-hour period. If additional cultures are requested, the physician needs to order blood cultures x X, where X is the number of draws.
- Exceptions to the recommendation of 2 blood cultures per 24-hour period are diagnoses of SBE or AIDS.
- Blood cultures drawn by venipuncture should be collected at times 0 and 30 minutes. Blood cultures drawn by nursing personnel should be drawn at least 30 minutes apart. Do not draw multiple blood cultures from one site at the same time.
- **Specimen Type:** Blood.
- **Volume:** 8 - 10 mL: use aerobe bottle and anaerobe bottle; if <1 mL is collected, use aerobe bottle only; For NICU, draw 1 mL and inoculate aerobic bottle only.
- **Container:** Blood culture bottles (aerobic and anaerobic).
- **Specimen Handling:**
 - **PLEASE REFER TO BLOOD CULTURE COLLECTION PROCEDURE.**
 - Transport: Deliver to Microbiology within 2 hours of collection.
- **Storage:** Room temperature.

-
- **Test Name:** CULTURE BODY FLUID
 - **Test ID:** CBF
 - **Test Includes:** Gram stain, aerobic and anaerobic cultures for bacterial pathogens. Sensitivity testing performed if indicated.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results are available over 4-5 days.
 - **Specimen Type:** Body fluid, bone marrow aspirate, joint fluid, paracentesis fluid, pericardial fluid, pleural fluid, prostatic fluid, synovial fluid, thoracentesis fluid, suprapubic urine aspirates.
 - **Volume:** 10 - 20 mL. Additional volume is necessary if other tests are ordered.
 - **Container:** Sterile container or a syringe WITHOUT A NEEDLE. If the specimen is a bone marrow aspirate, place in SPS.
 - **Specimen Handling:** Collection:
 - The specimen is usually collected by a physician using standard aseptic techniques.
 - A body fluid worksheet is available to assist with ordering and computer entry.
 - **Transport:** Deliver to Laboratory accession area within 2 hours of collection for inpatients.
 - **Storage:** Room temperature

-
- **Test Name:** CULTURE BONE
 - **Test ID:** CBONE
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** Done daily 0700 – 2400. Results available within 7 days.
- **Container:** Sterile Container, Room temperature

-
- **Test Name:** CULTURE BRONCH, QUANTITATIVE
 - **Test ID:** QBAL
 - **Test Includes:** Gram stain and quantitative culture for potential respiratory pathogens including S. pneumoniae, H. influenzae, MRSA and gram-negative bacilli. Identification and sensitivity performed if threshold is met
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 – 2400. Results are available within 2 days.
 - **Specimen Type:** Bronchoalveolar lavage, mini-BAL or protected specimen brush
 - **Volume:** 5 mL
 - **Container:** Sterile container
 - **Specimen Handling:** Collection: Specimen collection is performed by a trained physician
 - **Transport:** Deliver to Microbiology within 2 hours of collection.

-
- **Test Name:** CULTURE CATHETER TIP
 - **Test ID:** CCATH
 - **Testing Facility:** Aultman Hospital
 - **Turnaround Time:** Results available in 7 days

-
- **Test Name:** CULTURE DIALYSIS FLUID
 - **Test ID:** CDIAL
 - **Synonyms:** Peritoneal dialysis culture.
 - **Test Includes:** Gram stain, WBCs by leukocyte stains, aerobic and anaerobic culture.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 5 days.
 - **Specimen Type:** Peritoneal dialysis fluid.
 - **Volume:** 100 mL.
 - **Container:** Sterile container
 - **Specimen Handling:** Collection: Pour approximately 100 mL of peritoneal dialysis fluid into a sterile container. Transport: Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Room temperature.
-
- **Test Name:** CULTURE EAR
 - **Test ID:** CEAR
 - **Test Includes:** Bacterial pathogens routinely cultured include S. aureus, S. pneumoniae, Moraxella catarrhalis and Gram-negative rods. Sensitivity testing is performed if indicated.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 2 days.
 - **Specimen Type:** Ear.

Aultman Hospital Laboratory Test Directory

- **Volume:** 1 culturette (2 swabs).
- **Container:** Culturette.
- **Specimen Handling:** Collection: Using a sterile swab from a culturette, obtain material from the area of inflammation. Transport: Deliver to Microbiology within 2 hours of collection for inpatients. Storage: Refrigerate.

-
- **Test Name:** CULTURE EYE
 - **Test ID:** CEYE
 - **Test Includes:** Bacterial pathogens routinely cultured include *S. aureus*, *S. pneumoniae*, *Haemophile's*, *Neisseria* and Gram-negative rods. Sensitivity testing performed if indicated.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 2 days.
 - **Specimen Type:** Eye, conjunctiva.
 - **Volume:** 1 culturette (2 swabs).
 - **Container:** Culturette.
 - **Specimen Handling:** Collection: Using a sterile culturette swab, obtain the sample from the conjunctiva of the infected eye.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate EXCEPT when the request includes culture for *N. gonorrhoeae*. Then store at room temperature.

-
- **Test Name:** CULTURE FUNGUS
 - **Test ID:** CFUNG
 - **Test Includes:** Stain and culture for yeast and molds.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Performed daily 0700 - 2400. Stain results are available in 1 day. Culture results are available in 4 weeks.
 - **Specimen Type:** Varies. See specimen collection instructions.
 - **Volume:** Varies. See specimen collection instructions.
 - **Container:** Sterile container, Petri dish or culturette.
 - **Specimen Handling:**
 - Collection:
 - Respiratory: collect according to sputum and bronchoscopy instructions.
 - Ear, eye, nose, throat, vagina, cervix: Use culturette.
 - Skin: cleanse lesion with a 70% alcohol saturated pad. Air dry. Using a blade or blunt end of forceps, obtain scraping from the active border of the lesion. Place in a sterile container.
 - Nails: clean the site with a 70% alcohol saturated pad. Collect shavings from under the nail plate into a sterile container.
 - Hair: remove dull hair with forceps. Place hair in a sterile container.
 - Bone marrow: collect in a yellow top (SPS) vacutainer tube.

Aultman Hospital Laboratory Test Directory

- CSF: collect 1 mL by standard aseptic techniques.
- Abscess: aspirate at least 0.5 mL into a syringe.
- Tissue: place specimen in a sterile container with a sterile gauze pad moistened with sterile saline.
- Urine: collect 20 - 50 mL of the first morning void.
- Blood: collect 10 mL of blood according to procedure for standard blood cultures.
- Transport:
 - Deliver to Microbiology within 2 hours of collection for inpatients.
- Storage:
 - **Refrigerate:** Urine and respiratory specimens.
 - **Room temperature:** All other specimens.
- **Test Name: CULTURE FUNGUS, HAIR, SKIN AND NAILS**
- **Test ID:** CFUNG(HSN)
- **Testing Facility:** Aultman Hospital
- **Turnaround Time:** Weekly
- **Specimen Type:** Skin Scrapings, Nails, Nail Shavings, Hair
- **Container:** Sterile Cup
- **Specimen Handling:** Room Temperature

- **Test Name: CULTURE GC ONLY**
- **Test ID:** CGC
- **Testing Includes:** Culture for N. gonorrhoeae only.
- **Testing Facility:** Aultman Laboratory
- **Turnaround time:** Done daily 0700 - 2400. Results are available in 3 days.
- **Specimen Type:** Cervix, vagina, genital, urethral, penis, eye, throat, rectal swab.
- **Volume:** 1 culturette (2 swabs) or 1 NPH swab.
- **Container:** Culturette, NPH swab or inoculated GC media in transport bag with tablet.
- **Specimen Handling:** Collection:
 - Cervix, vagina, genital: See CULTURE GENITAL FEMALE.
 - Urethra, penis: See CULTURE URETHRA.
 - Throat: See CULTURE RESPIRATORY, UPPER.
 - Eye: See CULTURE EYE.
 - Rectal swab: Collect with Culturette.
 - Remove swab from sheath.
 - Insert through anal orifice and move from side to side.
 - Return swab to sheath.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Room temperature.

-
- **Test Name: CULTURE HERPES**
 - **Test ID:** CHER
 - **Test Includes:** Culture for HSV 1 and 2 and/or Varicella in non-genital sources.

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 6 days, except for Varicella (14 days).
 - **Precollection Instructions:** Collect specimen prior to treatment with ointment. Note in comments if Varicella virus is suspected.
 - **Specimen Type:** Fluid, lesion material, gargle.
 - **Volume:** 1 culturette or 1 mL of fluid, 5-8 mL gargle
 - **Container:** Culturette in M4 viral transport media or sterile container.
 - **Specimen Handling:** Collection: Collect fluid or cellular material from the base of the lesion with a culturette. Place the culturette in M4 viral transport media (VTM). M4 is available from Microbiology (x36113).
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate.
-
- **Test Name:** CULTURE MEDICAL DEVICE
 - **Test ID:** CMED
 - **Testing Facility:** Aultman Hospital
 - **Turnaround Time:** Results available in 7 days
 - **Specimen Type:** Surgically removed Screws, Plates, Ports, and Tubing
 - **Handling:** Room Temperature
-
- **Test Name:** CULTURE RESPIRATORY, LOWER
 - **Test ID:** CRESP
 - **Testing Includes:** Gram stain and culture for common respiratory pathogens including Staphylococcus, Streptococcus, S. pneumoniae, H. influenzae and Gram-negative bacilli. Sensitivity testing is performed if indicated.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 2 days.
 - **Precollection Instructions:** If antibiotic and culture are ordered simultaneously, hold the antibiotic until the culture has been obtained, but no longer than 2 hours
 - **Specimen Type:** Sputum, tracheal aspirate, bronch wash, bronch brush, Lukens tube.
 - **Volume:** 5 mL.
 - **Container:** Sterile container.
 - **Specimen Handling:** Collection:
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate.
-
- **Test Name:** CULTURE RESPIRATORY, UPPER
 - **Test ID:** CTHROAT
 - **Testing Includes:** Culture for common upper respiratory pathogens including S. aureus, S. pneumoniae, Group A Beta Strep and Haemophilus (if <12 years old). Sensitivity testing is performed if indicated.
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** Done daily 0700 - 2400. Results available in 2 days.
- **Specimen Type:** Throat
- **Volume:** 1 culturette (2 swabs) or 1 NPH swab.
- **Container:** Culturette or NPH culturette.
- **Specimen Handling:** Collection:
Nasopharyngeal:
 - Use a culturette swab to clear the nostril.
 - Using an NPH wire swab bent close to the cotton tip, enter the nostril until resistance is met. Turn the swab downward and continue.
 - Obtain the specimen from above the soft palate area.
- Return the swab to the sheath. b. Throat:
 - With a tongue blade, depress the tongue so the back of the throat is clearly visible.
 - Without getting buccal or tongue contamination, insert the culturette to the back of the throat. Swab both tonsillar areas, the posterior pharynx and any areas of inflammation, ulceration or exudation. A good gag reflex is indicative of a satisfactory specimen.
- c. Nose:
 - Insert the tip of a culturette into the nostril until resistance is met. Gently rotate the swab and withdraw.
 - Specimens to determine Staph carriers should be taken from the anterior nares only, not inside the nostrils.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Refrigerate.

-
- **Test Name:** CULTURE SPINAL FLUID
 - **Test ID:** CCSF
 - **Synonyms:** Lumbar puncture, shunt fluid, ventricular fluid.
 - **Test Includes:** Lumbar puncture source includes gram stain and aerobic culture. Shunt and ventricular fluid sources include gram stains, aerobics and anaerobic culture. Sensitivity testing is performed if indicated.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Routine Gram stain results are available within 8 hours; urgent/STAT within 30 minutes. Culture results are available in 4 days.
 - **Precollection Instructions:** The CSF worksheet (form 360) must accompany all CSF specimens. The physician must indicate the tests requested in order of preference.
 - **Specimen Type:** Cerebrospinal fluid, shunt fluid, ventricular fluid.
 - **Volume:** 1 mL. Additional CSF will be required for each test requested (CFUNG, CAF, CRYPTC, MEPCR).
 - **Container:** Sterile body fluid tubes.
 - **Specimen Handling:** Collection: The specimen is usually collected by a physician.
 - **Transport:** Deliver the specimen to Microbiology immediately and hand the specimen directly to a technologist. Between midnight and 0600, deliver specimens to a Hematology technologist.
 - **Storage:** Room temperature.
-
- **Test Name:** CULTURE STOOL
 - **Test ID:** CST

Aultman Hospital Laboratory Test Directory

- **Testing Includes:** Bacterial pathogens routinely cultured include Salmonella, Shigella and Campylobacter.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily 0700 - 2400. Results are available in 3 days.
- **Precollection Instructions:** Note in the order commenting if the request is to include Aeromonas or Pleisiomonas.
- **Specimen Type:** Stool, rectal swabs.
- **Volume:** 1 gram or at least visible material on 1 culturette (2 swabs).
- **Container:** Clean container with a tight-fitting lid or 1 culturette (2 swabs).
- **Specimen Handling:**
 - Collection:
 - Stool:
- The entire contents of the first specimen of the day are preferred.
- Avoid contamination with water, urine, paper, barium or mineral oil. Do not remove specimens from the toilet bowl.
- Specimens received in a diaper are not acceptable and will be rejected by Microbiology.
 - Rectal swabs:
- Remove swab from the sheath.
- Insert through the anal orifice and move from side to side to sample crypts.
- Return swab to the sheath
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Refrigerate.

-
- **Test Name:** CULTURE STOOL FOR VRE
 - **Test ID:** CSVRE
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Results available in 48-72 hours
 - **Precollection Instructions:** Note in the order commenting if the request is to include Aeromonas or Pleisiomonas.
 - **Specimen Type:** Stool or rectal swabs.
 - **Container:** Clean container with a tight-fitting lid or 1 dual culturette swab
 - **Specimen Handling:** REFRIGERATE
-
- **Test Name:** CULTURE STOOL WITH YERSINIA
 - **Test ID:** CSTY
 - **Test Includes:** Bacterial pathogens routinely cultured include Salmonella, Shigella, and Campylobacter. Yersinia is routinely done on patients ages 12 and under.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results are available in 3 days.
 - **Precollection Instructions:** Note in the order comments if the request is to include Aeromonas or Pleisiomonas.

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** Stool, rectal swabs.
- **Volume:** 1 gram or at least visible material on 2 culturettes (4 swabs).
- **Container:** Clean container with a tight-fitting lid or 2 culturettes (4 swabs).
- **Specimen Handling:** Collection: A. Stool:
 - The entire contents of the first specimen of the day is preferred.
 - Avoid contamination with water, urine, paper, barium or mineral oil. Do not remove specimens from the toilet bowl.
 - Specimens received in a diaper are not acceptable and will be rejected by Microbiology.
- B. Rectal swabs:
 - Remove swab from the sheath.
 - Insert through the anal orifice and move from side to side to sample crypts.
 - Return swab to the sheath.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Refrigerate.

- **Test Name:** CULTURE SURVEILLANCE
- **Test ID:** CSURV
- **Testing Includes:** Culture to rule out Group B Beta Strep, S. aureus, S. pneumoniae and Listeria.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily 0700 - 2400. Results available in 2 days.
- **Specimen Type:** Cord, axilla.
- **Volume:** 1 culturette (2 swabs).
- **Container:** Culturette (2 swabs).
- **Specimen Handling:**
 - Collection:
- If excessive exudate is present, cleanse the area with normal saline.
- Enter wound with culturette swabs to obtain drainage.
- Return swabs to sheath.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Room temperature.

-
- **Test Name:** CULTURE TISSUE
 - **Test ID:** CTISS
 - **Testing Includes:** Gram stain, aerobic and anaerobic culture for bacterial pathogens. Sensitivity testing will be performed if indicated.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results are available in 7 days.
 - **Precollection Instructions:** Specimens placed in formalin CANNOT be cultured for bacteria and are rejected by Microbiology.
 - **Specimen Type:** Tissues including lung, bronch biopsy, lymph node, autopsy, etc.
 - **Volume:** Minimal, approximately 1 cm².

Aultman Hospital Laboratory Test Directory

- **Container:** Sterile container. Do NOT place tissue in culturettes or anaerobic transport media.
- **Specimen Handling:** Collection: Specimens are collected surgically or at autopsy. Place specimen in sterile container with sterile gauze pad moistened with sterile saline.
- **Transport:** Use only sterile containers to transport tissue specimens. Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Room temperature.

-
- **Test Name:** CULTURE UREAPLASMA ONLY
 - **Test ID:** CUREA
 - **Test Includes:** Culture for Ureaplasma urealyticum. Mycoplasma hominis culture is included by request only.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 5 days.
 - **Precollection Instructions:** Specify in order comment R/O Mycoplasma if indicated.
 - **Specimen Type:** Cervix, vagina, penis, urethra.
 - **Volume:** 1 culturette or NPH swab.
 - **Container:** Culturette or NPH swab.
 - **Specimen Handling:**
 - Collection:
 - Cervix/vagina: Moisten speculum with water, not lubricant. Use a double culturette to obtain specimen.
 - Urethra/Penis: Collect specimen prior to first voided morning urine using an NPH swab inserted into the anterior urethra. Avoid contamination with external sources and skin surfaces.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Room temperature.

-
- **Test Name:** CULTURE URETHRA
 - **Test ID:** CURET
 - **Synonyms:** Gram stain and cultures for N. gonorrhoeae and Ureaplasma.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Gram stain results are available on the same day. Culture results are available in 5 days.
 - **Specimen Type:** Urethral or penis discharge.
 - **Volume:** 2 NPH swabs.
 - **Container:** NPH swabs. Note: Gonopak Martin plates are available in Microbiology for direct inoculation in the detection of N. gonorrhoeae.
 - **Specimen Handling:** Collection: (SEE NEXT PAGE)
 - Collect urethral discharge prior to first voided morning urine using an NPH swab inserted into the anterior urethra. Two swabs are necessary if the Gram stain is to be done.
 - Avoid contamination with skin surfaces.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Room temperature.

Aultman Hospital Laboratory Test Directory

- **Test Name:** CULTURE URINE
- **Test ID:** CUR
- **Test Includes:** Bacterial culture with colony count. Sensitivity testing will be performed in indicated.
- **Testing Facility:** Aultman Main Laboratory
- **Turnaround Time:** Done daily. Culture results and sensitivity (if indicated) are available within 2-3 days.
- **Precollection Instructions:** Collect urine aseptically in the sterile urine container with the special tube port. Peel back paper covering over the port, invert the preservative test tube, then push into the port. Allow the tube to fill completely, remove and shake the preservative test tube vigorously. Replace the paper covering. Send the preservative test tube for culture and blue cup for UA if needed.
- **Specimen Requirements:** A full preservative test tube or minimum of 1ml of urine in a sterile cup.
- **Container:** Urine preservative test tube or sterile container.
- **Specimen Handling:** Preservative test tube is acceptable at room temperature for 48 hours. Urines collected in sterile container (without preservative) will need refrigerated and must be deliver to microbiology lab within 24 hours.

- **Test Name:** CULTURE VIBRIO
- **Test ID:** CVIB
- **Testing Includes:** Culture for Vibrio species including V. cholera and V. parahaemolyticus.
- **Testing Facility:** Aultman Laboratory
- **Turnaround time:** Done daily 0700 - 2400. Results available in 2 days.
- **Specimen Type:** Stool, rectal swabs.
- **Volume:** 1 gram or at least a visible specimen on 2 swabs of double culturette.
- **Container:** Clean container with a tight-fitting lid or culturette (2 swabs).
- **Specimen Handling:** Collection:
 - Stool:
 - The entire contents of the first specimen of the day is preferred.
 - Avoid contamination with water, urine, paper, barium or mineral oil. Do not remove specimens from the toilet bowl.
 - Rectal swabs:
 - Remove swab from the sheath.
 - Insert through the anal orifice and move from side to side to sample crypts.
 - Return swab to the sheath.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Refrigerate.

- **Test Name:** CULTURE WOUND
- **Test ID:** CWD

Aultman Hospital Laboratory Test Directory

- **Testing Includes:** Gram stain and culture for aerobic pathogens. Sensitivity testing will be performed if indicated.
- **Testing Facility:** Aultman Laboratory
- **Turnaround time:** Done daily 0700 - 2400. Results available in 2-3 days.
- **Specimen Type:** Abscess, boil, burn, decubitus, incision, pustule, skin, ulcer, surface wound, lesion.
- **Volume:** 1 culturette (2 swabs) or 1 mL fluid.
- **Container:** Culturette, sterile container, syringe with needle removed.
- **Specimen Handling:** Collection:
 - Evaluate whether free fluid might be obtained deep in peripheral areas via syringe aspiration.
 - Otherwise, collect specimen with a culturette (2 swabs).
 - If excessive exudate is present, cleanse area with normal saline.
 - If necessary, massage area to obtain deep exudate.
 - Enter wounds with culturette swabs to obtain drainage.
 - Return swabs to sheath.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients. Do not transport a syringe with the needle still attached.
- **Storage:** Room temperature.

-
- **Test Name:** CULTURE WOUND DEEP PANEL
 - **Test ID:** CWDP
 - **Test Includes:** Anaerobic culture for bacterial pathogens. This test MUST be ordered in conjunction with an aerobic culture and gram stain (CWD). Sensitivity testing will be performed if indicated.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results are available in 7 days.
 - **Precollection Instructions:** Anaerobic transport media MUST be used for collection. To minimize oxygen exposure, keep transport tubes in an upright position when opened.
 - **Specimen Type:** Abscess, appendix, Bartholin cyst, bile, boil, cyst, endometrium, gall bladder, incision, IUD, peritoneum, placenta, ulcer, uterus, deep wounds. NOTE: Anaerobes are not cultured from superficial wounds such as burns, scrapings or the perianal region where stool contamination is evident.
 - **Volume:** 1 mL fluid or 1 culturette (2 swabs) AND 1 anaerobic transport tube (2 swabs).
 - **Container:** Sterile container, syringe or 1 culturette (2 swabs) AND 1 anaerobic transport tube (2 swabs). Anaerobic transport media is available in Microbiology (x36374). The anaerobic culture will NOT be set up if a culturette is received without also sending anaerobic transport media.
 - **Specimen Handling:**
 - Collection: Room Temperature
 - Evaluate whether free fluids might be obtained deep in peripheral areas via syringe aspiration. Otherwise, use a double culturette AND anaerobic transport media. B. If excessive exudate is present, cleanse area with normal saline.
 - If necessary, massage area to obtain deep exudate.
 - Enter wound with swabs to obtain drainage.
 - Return aerobe swabs to sheath. Place anaerobic swab in anaerobic transport media tube (glass tube). Cap tube.

Aultman Hospital Laboratory Test Directory

- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients. Do not transport a syringe with the needle still attached.
- **Test Name:** CYCLIC CITRULLINE PEPTIDE
- **Test ID:** CCP
- **Synonyms:** CCP
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done twice weekly
- **Specimen Type:** Serum
- **Volume:** 1 mL
- **Container:** 7 mL SST
- **Specimen Handling:** Refrigerate specimen
- **Test Name:** CYCLOSPORA SMEAR STOOL
- **LC Test Code:** 183145
- **Aultman Test Code:** MISCNB
- **LabCorp Link:** [Search](#) | [Labcorp](#)
- **Test Name:** CYCLOSPORINE BLOOD
- **LC Test Code:** 706557
- **Aultman Test Code:** CYCLO
- **LabCorp Link:** [Search](#) | [LabCorp](#)
- **Test Name:** CYSTATIN C
- **LC Test Code:** 121251
- **Aultman Test Code:** CYSTC
- **LabCorp Link:** [Search](#) | [Labcorp](#)
- **Test Name:** CYSTATIN C WITH EGFR
- **LC Test Code:** 121265
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [LabCorp](#)
- **Test Name:** CYSTIC FIBROSIS 97 VARIANTS
- **LC Test Code:** 481025
- **Aultman Test Code:** CFSCR
- **LabCorp Link:** [Search](#) | [Labcorp](#)
- **Test Name:** CYSTINE QUANT 24 HR URINE
- **LC Test Code:** 700195
- **Aultman Test Code:** MISCNB
- **LabCorp Link:** [Search](#) | [Labcorp](#)
- **Test Name:** CYTOMEGALOVIRUS (CMV) CULTURE

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 008201
- **Aultman Test Code:** MISCNB
- **LabCorp Link:** [Search](#) | [LabCorp](#)

- **Test Name:** D001-IGE D PTERONYSSINUS
- **LC Test Code:** 602467
- **Aultman Test Code:** DPTERN
- **LabCorp Link:** [Search](#) | [LabCorp](#)

- **Test Name:** D002-IGE D FARINAE
- **LC Test Code:** 602475
- **Aultman Test Code:** DFARN
- **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** D-DIMER
 - **Test ID:** DIMER
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 45 minutes, Routine: 3 hours
 - **Specimen Type:** Plasma (citrated).
 - **Volume:** 3 mL
 - **Container:** 2.7mL blue top tube (**Must be full**)
 - **Specimen Handling:** Must be tested within 4 hours of collection in Aultman Lab; alternatively, may be drawn in Aultman Outpatient Lab.

- **Test Name:** DENGUE VIRUS IGM ANTIBODY
- **LC Test Code:** 819198
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** DEOXYCORTICOSTERONE(DOC) S
- **LC Test Code:** 500138
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** DES 1 AND DES 3 IGG
- **LC Test Code:** 164122
- **Aultman Test Code:** DESMO
- **LabCorp Link:** [Search](#) | [LabCorp](#)

- **Test Name:** DEXAMETHASONE
- **LC Test Code:** 500118
- **Aultman Test Code:** DEXA
- **LabCorp Link:** [Search](#) | [LabCorp](#)

Aultman Hospital Laboratory Test Directory

- **Test Name:** DEXAMETHASONE SUPPRESSION TEST
- **Testing Facility:** Aultman Laboratory
- **Additional Information:** Dr gives dexamethasone between 11 and 12 pm. Draw serum cortisol between 7-9 am next morning.

-
- **Test Name:** DEXAMETHASONE SUPPRESSION, PSYCH
 - **Synonyms:** Dexamethasone suppression test used in psychiatric testing.
 - **Test Includes:** Cortisol levels measured prior to and following the administration of dexamethasone.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Cortisols are done Monday – Friday. Results are available on the same day if specimen is received prior to 1:00 p.m.
 - **Precollection Instructions:** The following are the standard conditions unless modified by the physician:
 - Day 1: At 2300, give the patient 1mg Dexamethasone (oral).
 - Day 2: Draw a cortisol level at 1600 and 2300. Minor variations (~1 hour) from these times will not affect the test.
 - Each cortisol level must be ordered separately in the computer.
 - **Specimen Type:** Serum from plain red top.
 - **Volume:** 1mL for each cortisol level.
 - **Container:** 7ml SST

-
- **Test Name:** DHEA- SERUM
 - **LC Test Code:** 004100
 - **Aultman Test Code:** DHEAU
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** DHEA-SO4
 - **Test ID:** DHEAS
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1mL
 - **Container:** SST

-
- **Test Name:** DIGOXIN
 - **Test ID:** DIG
 - **Test Includes:** Time of last dose, digoxin.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

Aultman Hospital Laboratory Test Directory

- **Storage:** Refrigerate

-
- **Test Name:** DIHYDROTESTOSTERONE
 - **LC Test Code:** 500142
 - **Aultman Test Code:** DHT
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** DIPHTHERIA ANTITOXOID AB
 - **LC Test Code:** 163709
 - **Aultman Test Code:** DIPIGG
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** Direct Coombs/Direct AHG
 - **Synonyms:** Direct antiglobulin test
 - **Testing facility:** Aultman Laboratory
 - **Specimen type:** Plasma
 - **Volume:** 6ml
 - **Container:** EDTA (pink top)
 - **TAT:** Refer to comment in ABS
 - **Test includes** DAT complement and DAT IgG; Blood Bank will reflex additional testing when indicated

-
- **Test Name:** DPYD GENOTYPING
 - **LC Test Code:** 512275
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** DRUG ANALYSIS 12, ORAL FLUID
 - **LC Test Code:** 702705
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** DRUG SCREEN, ER
 - **Test ID:** ERDS
 - **Test Includes:** Serum screened for: salicylate, acetaminophen, ethanol, tricyclic antidepressants. Urine screened for: salicylate, cannabinoid, cocaine, benzodiazepines, opiates, barbiturates, PCP, and amphetamine. The screening DOES NOT include confirmation.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Within 2 hours.
 - **Precollection Instructions:** List of currently known medications.
 - **Specimen Type:** Urine and serum from plain red top tube or SST or plasma lithium heparin – Separate from cells within 2 hours of collection.
 - **Volume:** Urine: 20mL, Serum: 3mL
 - **Container:** Urine: Screw top urine container, Serum: plain red top tube/SST/plasma lithium heparin

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** DRUG SCREEN 9 W/CONF MECONIUM
 - **LC Test Code:** 791430
 - **Aultman Test Code:** MECDRG
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** DRUG SCREEN 12 W/CONF SE
 - **LC Test Code:** 701978
 - **Aultman Test Code:** DRUG12B
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** DRUG SCREEN, SERUM
 - **Test ID:** DRUGS
 - **Test Includes:** Screening for acetaminophen, salicylate, ethanol, tricyclic antidepressants.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day
 - **Specimen Type:** SST or plain red top – Separate from cells within 2 hours of collection.
 - **Volume:** 2 mL
 - **Container:** SST/Red Top
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial when using plain red top.

-
- **Test Name:** DRUG SCREEN, URINE
 - **Test ID:** DRUGU
 - **Test Includes:** Screening for salicylate, cannabinoids, cocaine, benzodiazepines, opiates, barbiturates, PCP, amphetamine, propoxyphene, methadone. Also includes pH and specific gravity. Does not include confirmation.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available at 1600 if received by 0800.
 - **Precollection Instructions:** List patient's current medications.
 - **Specimen Type:** Random urine
 - **Volume:** minimum 10 mL
 - **Container:** Screw top urine container.
 - **Specimen Handling:** Store 2-8 C for 24 hours. If it is longer than 24 hours, freeze specimen.

-
- **Test Name:** E001-IGE CAT DANDER
 - **LC Test Code:** 602454
 - **Aultman Test Code:** CATDND
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** E003-IGE HORSE DANDER
 - **LC Test Code:** 602684
 - **Aultman Test Code:** HORSE
 - **LabCorp Link:** [Search | LabCorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** E005-IGE DOG DANDER
 - **LC Test Code:** 602456
 - **Aultman Test Code:** K9DND
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

- **Test Name:** E072-IGE MOUSE URINE
- **LC Test Code:** 602689
- **Aultman Test Code:** MOUR
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** E082-IGE RABBIT EPITHELIA
 - **LC Test Code:** 602700
 - **Aultman Test Code:** RABEPI
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** E086-IGE DUCK FEATHERS
 - **LC Test Code:** 602704
 - **Aultman Test Code:** DUCKFE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** E089-IGE TURKEY FEATHERS
 - **LC Test Code:** 602708
 - **Aultman Test Code:** TURKFE
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** EBV ANTIBODY PROFILE
 - **LC Test Code:** 240610
 - **Aultman Test Code:** EBVPNL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** EBV ANTIBODY TITER
 - **Test ID:** EBV
 - **Synonyms:** Epstein Barr Virus antibody titer, Mono titer.
 - **Test Includes:** EBV IgM and IgG. Acute/convalescent specimens tested if there was an EBV order in the past 2 months.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly Mon through Fri. Results are available within 3 days.
 - **Specimen Type:** Serum
 - **Volume:** 1mL
 - **Container:** 7mL SST
 - **Specimen Handling:** Avoid hemolysis.

-
- **Test Name:** EBV DNA QUANTITATIVE PCR

Aultman Hospital Laboratory Test Directory

- **CCF Test Code:** EBVQNT
- **Aultman Test Code:** EBVQNT
- **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** EBV EARLY ANTIGEN AB IGG
 - **LC Test Code:** 096248
 - **Aultman Test Code:** EBVEA
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ECHINOCOCCUS ANTIBODY
 - **LC Test Code:** 138768
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ECHOVIRUS SERUM AB PANEL
 - **LC Test Code:** 823361
 - **Aultman Test Code:** ECHO
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** EHRLICHIA CHAFFEENSIS AB
 - **LC Test Code:** 164680
 - **Aultman Test Code:** ECHAFF
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** EHRLICHIA DETECTION PCR
 - **LC Test Code:** 138412
 - **Aultman Test Code:** EHRPCR
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ELECTROLYTES, SERUM
 - **Test ID:** LYLES
 - **Test Includes:** Sodium, potassium, chloride, CO2, electrolyte balance.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 2 – 4mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis. Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate.

-
- **Test Name:** ENA ANTIBODY PANEL
 - **CCF Test Code:** ENAID
 - **Aultman Test Code:** ENA1

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** ENDOMYSIAL ANTIBODY
 - **Test ID:** ENDO
 - **Synonyms:** Transglutaminase IgA, Tissue Transglutaminase
 - **Test Includes:** Done twice weekly
 - **Testing Facility:** Aultman Laboratory
 - **Specimen Type:** Serum
 - **Volume:** 1mL
 - **Container:** 7mL SST
 - **Specimen Handling:** Refrigerate serum.

-
- **Test Name:** ENDOSCOPIC BRUSHINGS FOR CYTOLOGY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm. 2 working days. Certain non-Gyn cases will require additional stains and will take longer.
 - **Precollection Instructions:** Use Form 308A, (Cytology Specimens) Mark the source of the specimen on the form and include any clinical diagnosis.
 - **Specimen Handling:** Collection:
 - Preferred method: Place the brush in a Cytology container with fixative. Cut off the excess wire. Alternative method: Spread the brush on a slide and immediately spray fix the smears with Cytology fixative. Label specimen and slides with two patient identifiers. Place the slides in a cardboard or plastic container for transport.
 - **Transport:** Deliver to the Cytology department. After hours, deliver specimen to the Microbiology department.
 - **Storage:** Refrigerate or store at room temperature.

- **Test Name:** ENHANCED LIVER FIBROSIS (ELF)
- **LC Test Code:** 550659
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ENTEROVIRUS RT PCR
 - **LC Test Code:** 138636
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** EOSINOPHIL COUNT
 - **Test ID:** EOCT
 - **Synonyms:** Absolute EO count.
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** 6 hours
- **Specimen Type:** Whole blood
- **Volume:** 3mL
- **Container:** 5mL Lav top tube.
- **Storage:** Room temperature, or if > 12 hours refrigerate.

-
- **Test Name:** EOSINOPHIL SMEAR
 - **Test ID:** EOS
 - **Test Includes:** Specimen source, eosinophil count
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 24 – 48 hours
 - **Specimen Type:** Secretions from the nose, paranasal sinuses, trachea or bronchi, or urine.
 - **Container:** As appropriate: nasal or sputum swab, sputum container, bronchial lavage container, urine tube.
 - **Storage:** Room temperature.

-
- **Test Name:** EPSTEIN-BARR VIRUS PCR
 - **LC Test Code:** 138289
 - **Aultman Test Code:** CSFEBV
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ERYTHROCYTE SED RATE
 - **Test ID:** ESR
 - **Synonyms:** ESR, Sed Rate, WSR, Westergren Sed Rate
 - **Test Includes:** Modified Westergren Sed rate.
 - **Testing Facility:** Aultman Laboratory
 - **Specimen Type:** Whole blood (EDTA)
 - **Volume:** 2 mL
 - **Container:** 2.5 mL Lav top tube.
 - **Storage:** Sample is good stored 4 hours at room temperature, 24 hrs refrigerated

-
- **Test Name:** ERYTHROPOIETIN (EPO), SERUM
 - **LC Test Code:** 140277
 - **Aultman Test Code:** ERYTH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ESTRADIOL
 - **Test ID:** E2
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Serum only
 - **Volume:** 1 – 2mL

Aultman Hospital Laboratory Test Directory

- **Container:** SST
- **Storage:** Refrigerate.

-
- **Test Name:** ESTRADIOL SENSITIVE
 - **LC Test Code:** 140244
 - **Aultman Test Code:** ESTMS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ESTRADIOL SERUM
 - **LC Test Code:** 004614
 - **Aultman Test Code:** EST
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ESTRADIOL, FREE SERUM
 - **LC Test Code:** 500649
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ESTROGENS FRACTIONATED, BLOOD
 - **LC Test Code:** 004606
 - **Aultman Test Code:** ESTGEN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ESTROGEN TOTAL
 - **LC Test Code:** 004549
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ESTRONE, SERUM
 - **LC Test Code:** 004564
 - **Aultman Test Code:** ESTR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ETHANOL
 - **Test ID:** ALC
 - **Test Includes:** Ethanol only.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available within 1 hour of receipt.
 - **Specimen Type:** SST or plain red top – Separate from cells within 2 hours of collection.
 - **Volume:** 1mL
 - **Container:** SST/Red top
 - **Specimen Handling:** Keep tube sealed until testing. Separate serum from cells ASAP.
 - **Storage:** Refrigerate.

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** ETHANOL, URINE
 - **Test ID:** ETOHU
 - **Testing Facility:** Aultman Hospital
 - **Specimen Type:** Random URINE 10 ml
 - **Container:** Screw top urine container.
-

- **Test Name:** ETHOSUXIMIDE (ZARONTIN) SERUM
 - **LC Test Code:** 007443
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** ETHYL GLUCURONIDE URINE
 - **LC Test Code:** 737610
 - **Aultman Test Code:** UEGLOC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** EUGLOBULIN LYSIS TIME
 - **LC Test Code:** 505530
 - **Aultman Test Code:** EULYS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** EVEROLIMUS
 - **LC Test Code:** 700003
 - **Aultman Test Code:** EVERO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** EXTENDED RAS/RAF PATHWAY MUTATIONS PANEL
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** KRAS/NRAS/BRAF (COLON)
 - **Reference Lab Link:** [Test Menu](#) | [Labcorp Oncology](#)
-

- **Test Name:** F001-IGE EGG WHITE
 - **LC Test Code:** 602452
 - **Aultman Test Code:** EGGWHT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** F002-IGE MILK
 - **LC Test Code:** 602453
 - **Aultman Test Code:** MILKC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** F003-IGE CODFISH
- **LC Test Code:** 602465

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** CODFSH
- **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** F004-IGE WHEAT
 - **LC Test Code:** 602459
 - **Aultman Test Code:** WHEAT`
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F005-IGE RYE
 - **LC Test Code:** 602522
 - **Aultman Test Code:** RYEFOOD
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F006- IGE BARLEY
 - **LC Test Code:** 602504
 - **Aultman Test Code:** BARLEY
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F007-IGE OAT
 - **LC Test Code:** 602553
 - **Aultman Test Code:** OAT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F008-IGE CORN
 - **LC Test Code:** 602460
 - **Aultman Test Code:** CORNS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F009 IGE RICE
 - **LC Test Code:** 602511
 - **Aultman Test Code:** RICE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F010-IGE SESAME SEED
 - **LC Test Code:** 602485
 - **Aultman Test Code:** SESAME
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F011-IGE BUCKWHEAT
 - **LC Test Code:** 602715
 - **Aultman Test Code:** BUKWHT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F012- IGE GREEN PEA
 - **LC Test Code:** 070029

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** ZINCRB
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F013-IGE PEANUT
 - **LC Test Code:** 602451
 - **Aultman Test Code:** PEANUT
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** F014-IGE SOYABEAN
 - **LC Test Code:** 602457
 - **Aultman Test Code:** SOYBN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F015-IGE WHITE BEAN
 - **LC Test Code:** 602525
 - **Aultman Test Code:** WTBEAN
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** F017-IGE HAZELNUT(FILBERT)
 - **LC Test Code:** 602483
 - **Aultman Test Code:** HAZEL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F018-IGE BRAZIL NUT
 - **LC Test Code:** 602477
 - **Aultman Test Code:** BRAZIL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F020-IGE ALMOND
 - **LC Test Code:** 602479
 - **Aultman Test Code:** ALMOND
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F023-IGE CRAB
 - **LC Test Code:** 602493
 - **Aultman Test Code:** CRAB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F024- IGE SHRIMP
 - **LC Test Code:** 602473
 - **Aultman Test Code:** SHRIMP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F025- IGE TOMATO
 - **LC Test Code:** 602468
 - **Aultman Test Code:** TOMATO

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F026- IGE PORK
 - LC Test Code: 602498
 - Aultman Test Code: PORK
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F027-IGE BEEF
 - LC Test Code: 602509
 - Aultman Test Code: BEEFMT
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F031-IGE CARROT
 - LC Test Code: 602521
 - Aultman Test Code: CARROT
 - LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: F033- IGE ORANGE
 - LC Test Code: 602472
 - Aultman Test Code: ORNGE
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F035-IGE POTATO WHITE
 - LC Test Code: 602499
 - Aultman Test Code: POTATO
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F036-IGE COCONUT
 - LC Test Code: 602555
 - Aultman Test Code: COCNUT
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F040-IGE TUNA
 - LC Test Code: 602510
 - Aultman Test Code: TUNA
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F041-IGE SALMON
 - LC Test Code: 602507
 - Aultman Test Code: SMON
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F042-IGE HADDOCK
 - LC Test Code: 602720
 - Aultman Test Code: MISC

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F044- IGE STRAWBERRY
 - LC Test Code: 602513
 - Aultman Test Code: STRBRY
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F045- IGE YEAST
 - LC Test Code: 602567
 - Aultman Test Code: YEAST
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F047-IGE GARLIC
 - LC Test Code: 602519
 - Aultman Test Code: GARLIC
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F048-IGE ONION
 - LC Test Code: 602723
 - Aultman Test Code: ONIONS
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F049-IGE APPLE
 - LC Test Code: 602724
 - Aultman Test Code: APPLES
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F054-IGE GREAPE FRUIT
 - LC Test Code: 602769
 - Aultman Test Code: GRAFRU
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F075- IGE EGG (YOLK)
 - LC Test Code: 602487
 - Aultman Test Code: EGYOLK
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F079-IGE GLUTEN
 - LC Test Code: 602497
 - Aultman Test Code: GLUTEN
 - LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: F080-IGE LOBSTER
 - LC Test Code: 602495
 - Aultman Test Code: LOBSTR

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F081-IGE CHEESE CHEDDAR TYPE
 - LC Test Code: 602732
 - Aultman Test Code: CHEDCH
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F083-IGE CHICKEN
 - LC Test Code: 602500
 - Aultman Test Code: CHCKN
 - LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: F085-IGE CELERY
 - LC Test Code: 602735
 - Aultman Test Code: CELERY
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F087-IGE MELON
 - LC Test Code: 602737
 - Aultman Test Code: MELON
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F088-IGE LAMB
 - LC Test Code: 602738
 - Aultman Test Code: LAMB
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F089-IGE MUSTARD
 - LC Test Code: 602739
 - Aultman Test Code: MUSTD
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F090-IGE MALT
 - LC Test Code: 602740
 - Aultman Test Code: MLTIGE
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F91- IGE MANGO
 - LC Test Code: 602741
 - Aultman Test Code: MISC
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F092-IGE BANANA
 - LC Test Code: 602742
 - Aultman Test Code: BNANA

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F093-IGE CHOCOLATE/CACAO
 - LC Test Code: 602492
 - Aultman Test Code: COCOA
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F094-IGE PEAR
 - LC Test Code: 602743
 - Aultman Test Code: PEAR
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F095-IGE PEACH
 - LC Test Code: 602744
 - Aultman Test Code: PEACH
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F096-IGE AVOCADO
 - LC Test Code: 602745
 - Aultman Test Code: AVOCAD
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F147-IGE FLOUNDER
 - LC Test Code: 602759
 - Aultman Test Code: FLOUND
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F182-IGE LIMA BEAN
 - LC Test Code: 602765
 - Aultman Test Code: LIMA
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F201-IGE PECAN NUT
 - LC Test Code: 602470
 - Aultman Test Code: PECAN
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F202- IGE CASHEW NUT
 - LC Test Code: 602461
 - Aultman Test Code: CASHEW
 - LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: F203-IGE PISTACHIO NUT
 - LC Test Code: 602486
 - Aultman Test Code: PISTAC

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: F204-IGE TROUT
 - LC Test Code: 602523
 - Aultman Test Code: TROUT
 - LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: F205-IGE HERRING
 - LC Test Code: 602767
 - Aultman Test Code: MISC
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F207-IGE CLAM
 - LC Test Code: 602529
 - Aultman Test Code: CUR
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F208-IGE LEMON
 - LC Test Code: 602768
 - Aultman Test Code: LEMON
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F210-IGE PINEAPPLE
 - LC Test Code: 602770
 - Aultman Test Code: PINEAP
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F21-IGE BLACKBERRY
 - LC Test Code: 602772
 - Aultman Test Code: MISC
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F212-IGE MUSHROOM
 - LC Test Code: 602773
 - Aultman Test Code: MUSHRM
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F214-IGE SPINACH
 - LC Test Code: 602775
 - Aultman Test Code: SPIN
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: F215-IGE LETTUCE
 - LC Test Code: 602776
 - Aultman Test Code: LETUCE
 - LabCorp Link: [Search](#) | [Labcorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** F216-IGE CABBAGE
 - **LC Test Code:** 602777
 - **Aultman Test Code:** CABBAG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F218-IGE PAPRIKA/SWEET PEPPER
 - **LC Test Code:** 602779
 - **Aultman Test Code:** GPEP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F220-IGE CINNAMON
 - **LC Test Code:** 602781
 - **Aultman Test Code:** CINAMN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F221-IGE COFFEE
 - **LC Test Code:** 602782
 - **Aultman Test Code:** COFFEE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F222-IGE TEA
 - **LC Test Code:** 602782
 - **Aultman Test Code:** TEA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F225-IGE PUMPKIN
 - **LC Test Code:** 602786
 - **Aultman Test Code:** SQUASH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F234-IGE VANILLA
 - **LC Test Code:** 602793
 - **Aultman Test Code:** VANILLA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F236-IGE WHEY
 - **LC Test Code:** 602501
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F237-IGE APRICOT
 - **LC Test Code:** 602795
 - **Aultman Test Code:** APR
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** F242- IGE BING CHERRY
 - **LC Test Code:** 602796
 - **Aultman Test Code:** CHERRY
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F244-IGE CUCUMBER
 - **LC Test Code:** 602798
 - **Aultman Test Code:** CUCUM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F253-IGE PINE NUT PIGNOLES
 - **LC Test Code:** 602550
 - **Aultman Test Code:** PINENT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F255-IGE PLUM
 - **LC Test Code:** 602800
 - **Aultman Test Code:** PLUMM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F256-IGE WALNUT
 - **LC Test Code:** 602530
 - **Aultman Test Code:** WALNUT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F259-IGE GRAPE
 - **LC Test Code:** 602802
 - **Aultman Test Code:** GRAPE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F260-IGE BROCCOLI
 - **LC Test Code:** 602803
 - **Aultman Test Code:** BROCCI
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F261-IGE ASPARAGUS
 - **LC Test Code:** 602804
 - **Aultman Test Code:** ASPAR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F270-IGE GINGER
 - **LC Test Code:** 602810
 - **Aultman Test Code:** GING
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** F280-IGE BLACK PEPPERCORN
 - **LC Test Code:** 602818
 - **Aultman Test Code:** BLAPEP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F282-IGE NUTMEG
 - **LC Test Code:** 602820
 - **Aultman Test Code:** NMEG
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** F284-IGE TURKEY
 - **LC Test Code:** 602823
 - **Aultman Test Code:** TURKEY
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F287-IGE KIDNEY BEAN
 - **LC Test Code:** 602824
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F288-IGE BLUEBERRY
 - **LC Test Code:** 602825
 - **Aultman Test Code:** BLUBRY
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F290-IGE CAULIFLOWER
 - **LC Test Code:** 602827
 - **Aultman Test Code:** CAULI
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F297- IGE ACACIA GUM
 - **LC Test Code:** 602851
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F300-IGE GOAT'S MILK
 - **LC Test Code:** 602526
 - **Aultman Test Code:** GMILK
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F304-IGE CARMINE RED DYE
 - **LC Test Code:** 604506
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** F309-IGE CHICKPEA
- **LC Test Code:** 602835
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F312- IGE SWORDFISH
 - **LC Test Code:** 602787
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F315-IGE GREEN BEAN
 - **LC Test Code:** 602838
 - **Aultman Test Code:** GRBEAN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F319-IGE RED BEET
 - **LC Test Code:** 602767
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F320-IGE CRAYFISH FRESHWATER
 - **LC Test Code:** 602840
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F329-IGE WATERMELON
 - **LC Test Code:** 602718
 - **Aultman Test Code:** WATMEL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F333-IGE LINSEED
 - **LC Test Code:** 605610
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F338-IGE SCALLOP
 - **LC Test Code:** 602478
 - **Aultman Test Code:** SCALOP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F341-IGE CRANBERRY
 - **LC Test Code:** 602764
 - **Aultman Test Code:** CRANBY
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F342-IGE OLIVE, BLACK

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 602756
- **Aultman Test Code:** OLIVF
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F343-IGE RASPBERRY
 - **LC Test Code:** 602717
 - **Aultman Test Code:** RASPBYP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F345-IGE MACADAMIA NUT
 - **LC Test Code:** 602568
 - **Aultman Test Code:** MACADA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F369-IGE CATFISH
 - **LC Test Code:** 602845
 - **Aultman Test Code:** CTFISH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F381-IGE RED SNAPPER
 - **LC Test Code:** 602857
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F384- IGE WHITEFISH
 - **LC Test Code:** 602858
 - **Aultman Test Code:** WHIFSH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** F8 AG (see Von Willebrand Antigen)
 - **Test ID:** F8AG
 - **Testing Facility:** Aultman Hospital
 - **Specimen Handling:** Blood must be processed in Aultman Laboratory within 4 hours of collection; alternatively, it may be drawn in Aultman Outpatient Department
 - **Volume:** 2 ML
 - **Container:** 2 Full blue top tubes (MUST be FULL)
 - **Turn Around Time:** Batched & run once a week
 -

-
- **Test Name:** FACTOR V LEIDEN MUTATION
 - **LC Test Code:** 511154
 - **Aultman Test Code:** F5LEI
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FACTOR II, DNA ANALYSIS
 - **LC Test Code:** 511162

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** PTGEN
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** FACTOR VII ACTIVITY
- **LC Test Code:** 800599
- **Aultman Test Code:** F7
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** FACTOR IX
 - **Test ID:** F9
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 3 hours, Routine: Batched
 - **Precollection Instructions:** Notify the lab (coag) if this test is to be ordered at any time other than M – F 0700 – 1530 so that adequate staffing can be provided.
 - **Specimen Type:** Plasma (citrated)
 - **Volume:** 5mL
 - **Container:** 2 **FULL** blue top tubes (MUST be full)
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, it may be drawn in Aultman Outpatient Department.

-
- **Test Name:** FACTOR VIII
 - **Test ID:** F8
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 3 hours, Routine: Batched
 - **Precollection Instructions:** Notify the lab (coag) if this test is to be ordered at any time other than M – F 0700 – 1530 so that adequate staffing can be provided.
 - **Specimen Type:** Plasma (citrated)
 - **Volume:** 5mL
 - **Container:** 2 **FULL** blue top tube (MUST be full).
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, it may be drawn in Aultman Outpatient Department.

-
- **Test Name:** FACTOR X
 - **Test ID:** F10 Activity; do not confuse with Anti-XA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 3 hours, Routine: batched
 - **Specimen Type:** Plasma (citrated)
 - **Volume:** 2 mL
 - **Container:** (2) **FULL** blue top tubes (MUST be full)
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, it may be drawn in Aultman Outpatient Department.
-
- **Test Name:** FACTOR XI ACTIVITY

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 086314
- **Aultman Test Code:** F11
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FACTOR XII ACTIVITY
 - **LC Test Code:** 086322
 - **Aultman Test Code:** F12
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FACTOR XIII ACTIVITY
 - **LC Test Code:** 500185
 - **Aultman Test Code:** F13
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** FAMILIAL CARDIOMYOPATHY 67 GENE
 - **LC Test Code:** 832533
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FAMILIAL CARDIOMYOPATHY PANEL
 - **LC Test Code:** 482189
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FAT FECAL QUALITATIVE
 - **LC Test Code:** 001677
 - **Aultman Test Code:** FFATQL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FATTY ACID PROFILE, ESSENTIAL
 - **LC Test Code:** 821279
 - **Aultman Test Code:** FACID
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FDP PLASMA
 - **LC Test Code:** 115402
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FECAL CHYMOTRYPSIN
 - **LC Test Code:** 213092
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** FECAL LEUKOCYTES
- **Test ID:** FECL2
- **Test Includes:** Microscopic examination of stool for WBCs.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily 0700 - 2400. Results available the same day.
- **Specimen Type:** Stool.
- **Volume:** Minimum of 1 gram.
- **Container:** Clean container with a tight-fitting lid.
- **Specimen Handling:**
 - **Collection:**
 - The entire contents of the first specimen of the day are preferred.
 - Avoid contamination with water, urine, paper, barium or mineral oil. Do not remove specimens from the toilet bowl.
 - Specimens received in a diaper are not acceptable and will be rejected by Microbiology.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Refrigerate.

-
- **Test Name:** FECAL IMMUNOCULT TEST
 - **Test ID:** FIT
 - **Testing Facility:** Aultman Hospital
 - **Turnaround Time:** 7 days a week, on day shift
 - **Specimen Handling:** Feces on a FIT collection Kit or Sterile Container
 - **Additional Information:** Room Temperature stable for 14 days

-
- **Test Name:** FELBAMATE (FELBATOL(R)) SERUM
 - **LC Test Code:** 716530
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** FENTANYL/NORFENTANYL S/P
 - **LC Test Code:** 071543
 - **Aultman Test Code:** FENT
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** FERRITIN
 - **Test ID:** FERR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin
 - **Volume:** 1mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis.
 - **Storage:** Refrigerate.
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** FETAL MATERNAL HEMORRHAGE
- **Synonym:** Fetal Screen
- **Specimen type:** Plasma
- **Volume:** 6ml
- **Container:** EDTA (pink top)
- **TAT:** Refer to comment in ABS

-
- **Test Name:** FGF-23
 - **LC Test Code:** 004380
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** FIBRINOGEN
 - **Test ID:** FIB
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 45 minutes, Routine
 - **Specimen Type:** Plasma (citrated)
 - **Volume:** 2mL
 - **Container:** 1 Full blue top tube (MUST be full)
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, it may be drawn in Aultman Outpatient Department.

-
- **Test Name:** FIBRINOGEN ANTIGEN
 - **LC Test Code:** 117052
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** FINE NEEDLE ASPIRATION
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm. 2 working days. Certain non-Gyn cases will require additional stains and will take longer.
 - **Precollection Instructions:** This procedure is performed in the Radiology Department with a Cytotechnologist and a Pathologist in attendance. These procedures are scheduled in advance through the Radiology Department. The procedures are performed Monday through Friday from 8am-3pm. Any questions concerning these procedures may be directed to the Cytology Department
 - **Specimen Collection:**
 - Cerner requisition for In-patients. Use Form 308A (Cytology requisition) for Out-patients.
 - The following information must be included on the requisition:
 - The patient's name and medical record number (if there is one. New patients will be assigned a medical record number by the computer)
 - Patient's birth date
 - Patient's address, if the patient is to be billed
 - Insurance information
 - Patient's Medicare or Medicaid number

Aultman Hospital Laboratory Test Directory

- The source of the specimen. Indicate the source of each specimen if there are multiple sites done during the same procedure.
- Any pertinent clinical information
- Indicate any other special testing to be done on this specimen. (i.e. Flow Cytometry, special stains)
- Cerner requisitions must include a Cerner Order ID number with the source of the specimen.
- **Supplies:** Pre-filled specimen containers with Cytology Non-Gyn preservative.
 - Specimen and requisition labels
 - Aspirate needle(s)
 - Glass slides
 - Bottle of spray fixative (supplied by Cytology on request)
 - Cardboard folder(s)
- **Collection:**
 - Cytology prefers the collection of any aspirates into a container with Cytology Non-Gyn preservative. Pre-filled containers will be supplied by Cytology by request from office.
*Please do not rinse the needle in the preservative until the very last pass of the needle. This preservative should not be introduced into the patient, and it is not sterile. *
 - If the physician does not want to aspirate the specimen into preservative, smears from the aspirate can be made by aspirating the material onto a glass slide and spray fixing the slide with spray fixative. This must be done immediately after the specimen is introduced onto the glass slide to prevent airdrying. Hold the spray fixative bottle about 4 inches over the slide and give 3-4 quick squirts of spray to cover the entire area of the specimen. Let this air-dry before closing any slides in the cardboard folders. This will prevent any specimen being lost by sticking to the folder.
 - A preferred method is to smear a specimen sample between 2 slides and allow (1) to air dry and spray fix the other slide.
 - It is very important to label all slides and containers with two patient identifiers and the source of the specimen(s) especially if the physician is obtaining specimens from multiple sites.
- **Specimen Handling:**
 - Transport to the Lab:
 - Any specimens from outside offices will be picked up by an ALS courier and delivered to the Cytology Department for processing. Specimens obtained inside the hospital will be transported by lab courier, any nursing personnel or radiology staff to Cytology for processing. After hours, deliver specimen to the Microbiology department.
 - **Storage:** Specimens collected in preservative can be stored at room temperature for 2 weeks after collection. Glass slide specimens can be kept at room temperature. Either specimen can be refrigerated but it is not necessary.
 - Any specimen not in preservative needs to be refrigerated immediately after collection until transport to Cytology by the courier.

-
- **Test Name: FLECAINIDE (TAMBOCOR(TM)) S**
 - **LC Test Code: 141598**

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** FLEC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FLOW CYTOMETRY
 - **Test ID:** FLOW
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Specimen only good for 48 hours
 - **Specimen Type:** Peripheral blood or bone marrow.
 - **Volume:** 10mL
 - **Container:** Bone marrow – 10mL green top tube (sodium heparin), Peripheral blood- 10ml Sodium Heparin and 4ml EDTA Lavender tube
 - **Specimen Handling:** Keep specimen at room temperature. Keep in original collection tube.
 - **Additional Info:** Testing performed Mon – Thurs 7am – 3pm Friday – 7am – 2 pm.

-
- **Test Name:** FLOW CYTOMETRY HEMATOLYMPHOID/PLASMA CELL NEOPLASIA ASSESSMENT (HNA) (IMMUNOPHENOTYPING BY FLOW)
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** FLOW CYTOMETRY
 - **Reference Lab Link:** [Test Menu](#) | [Labcorp Oncology](#)

-
- **Test Name:** FOOD ALLERGY PROFILE
 - **LC Test Code:** 602989
 - **Aultman Test Code:** FOODAD
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** FOLATE, RBC
 - **Test ID:** FOLRBC
 - **Testing Facility:** Reference Laboratory
 - **Specimen Handling:** EDTA WHOLE BLOOD – 2 LAVENDER TOP TUBES
 - **Additional Information:** Frozen blood should be protected from light during and after collection
 - Transfer to amber send out tube for the frozen sample; regular send out tube for refrigerated sample

-
- **Test Name:** FOLATE, RBC
 - **LC Test Code:** 266015
 - **Aultman Test Code:** FOLR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FOLATE, SERUM
 - **Test ID:** FOL
 - **Synonyms:** Folic acid
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Precollection Instructions:** Fasting specimen is preferable.
 - **Specimen Type:** Serum.
 - **Volume:** 1mL

Aultman Hospital Laboratory Test Directory

- **Container:** 7mL SST
- **Specimen Handling:** Avoid hemolysis.
- **Storage:** Refrigerate.
- **Test Name:** FRAGILE X SYNDROME CARRIER
- **LC Test Code:** 481684
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FREE AND TOTAL INSULIN
 - **LC Test Code:** 501561
 - **Aultman Test Code:** FINS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** Free K+L LT CHAINS, QN, S
 - **LC Test Code:** 121137
 - **Aultman Test Code:** KLFRS
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** Free K+L LT CHAINS, QN, UR
 - **LC Test Code:** 121228
 - **Aultman Test Code:** UFLCKL
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** FREE VALPROIC ACID (DEPAKOTE)
 - **LC Test Code:** O70789
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FRUCTOSAMINE
 - **LC Test Code:** 100800
 - **Aultman Test Code:** FRUC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FSH
 - **Test ID:** FSH
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1mL
 - **Container:** 7mL SST

Aultman Hospital Laboratory Test Directory

- **Specimen Handling:** Avoid hemolysis. Centrifuge and refrigerate.
- **Storage:** Refrigerate.

-
- **Test Name:** FREE T4 BY DIALYSIS/MASS SPEC
 - **LC Test Code:** 501902
 - **Aultman Test Code:** AUREO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** FTA-ABS
 - **Test ID:** FTA
 - **Synonyms:** Fluorescent Treponemal Antibody test for the confirmation of reactive RPRs.
 - **Test Includes:** It is recommended to order an RPR in conjunction with the FTA.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done as needed. Results available the following day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7mL SST
 - **Specimen Handling:** Refrigerate

-
- **Test Name:** FUNGITELL BETA-D-GLUCAN
 - **LC Test Code:** 832599
 - **Aultman Test Code:** BDGLUC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** FUNGAL ANTIBODIES, DID
 - **LC Test Code:** 164284
 - **Aultman Test Code:** FUNID
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** G002-IGE BERMUDA GRASS
 - **LC Test Code:** 602464
 - **Aultman Test Code:** BRMUDA
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** G004-IGE FESCUE MEADOW
 - **LC Test Code:** 602549
 - **Aultman Test Code:** MFESCU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** G006-IGE TIMOTHY GRASS
 - **LC Test Code:** 602506
 - **Aultman Test Code:** TIMOTHY

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: G008-IGE BLUEGRASS KENTUCKY
 - LC Test Code: 602496
 - Aultman Test Code: JUNEGR
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: G010-IGE JOHNSON GRASS
 - LC Test Code: 602503
 - Aultman Test Code: JONSON
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: G012-IGE RYE GRASS
 - LC Test Code: 602872
 - Aultman Test Code: RYEGRS
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: G6PD, QN, BLD AND HGB
 - LC Test Code: 121003
 - Aultman Test Code: G6PD
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: GABAPENTIN(NEURONITIN) SERUM
 - LC Test Code: 716811
 - Aultman Test Code: GAB
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: GABAPENTIN URINE
 - LC Test Code: 790394
 - Aultman Test Code: MISCNB
 - LabCorp Link: [Search](#) | [LabCorp](#)

-
- Test Name: GAD-65 AUTOANTIBODY
 - LC Test Code: 143008
 - Aultman Test Code: GAD65
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: GALECTIN-3
 - LC Test Code: 004110
 - Aultman Test Code: GAL3
 - LabCorp Link: [Search](#) | [Labcorp](#)

-
- Test Name: GAMMA GT
 - Test ID: GGT
 - Testing Facility: Aultman Laboratory
 - Turnaround Time: Done 7 days/week, 24 hours/day. Results available the same day.

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** Plasma lithium heparin (PST)
- **Volume:** 1 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
- **Specimen Handling:** Avoid hemolysis
- **Storage:** Refrigerate.

-
- **Test Name:** GANGLIONIC ACHRA B SERUM
 - **LC Test Code:** 085970
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GANGLIOSIDE PANEL
 - **CCF Test Code:** GANGAB
 - **Aultman Test Code:** GANGAB
 - **LabCorp Link:** [Test Directory - Online Test Catalog](#) | [Cleveland Clinic Laboratories](#)

-
- **Test Name:** GASTROINTESTINAL PCR PANEL
 - **Test ID:** GIPCR
 - **Testing Facility:** Aultman Hospital
 - **Turnaround Time:** 0700-2400 results available
 - **Specimen Handling:** fresh stool or stool in Cary Blair Preservative
 - **Additional Information:** Fresh Stool – Refrigerate

-
- **Test Name:** GASTRIN SERUM
 - **LC Test Code:** 004390
 - **Aultman Test Code:** GAST
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GAUCHER ENZYME ANALYSIS
 - **LC Test Code:** 451780
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GELATIN PORCINE IGE
 - **LC Test Code:** 827274
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GENESEQ PLUS TTR
 - **LC Test Code:** 482353
 - **Aultman Test Code:** ATTRZ
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GENOSURE PRIME(R)

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 551700
- **Aultman Test Code:** HIVNGS
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** GENITAL MYCOPLASMAS NAA URINE
 - **LC Test Code:** 180040
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** GENTAMICIN
 - **Test ID:** GENTP (PEAK); GENTT (TROUGH)
 - **Synonyms:** Garamycin, Gentacin
 - **Test Includes:** Time of last dose, gentamicin
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Precollection Instructions:**
 - Usual sampling times:
 - Three separate levels should be done for 1 hour, 2 hours and 4 hours after IV infusion is completed. Order on 3 separate requisitions, identifying the specimen in order comments.
 - One order must have the following information in order comments:
 - Patient weight.
 - Dosage given.
 - Time infusion started and finished.
 - All other antibiotics given.
 - Copies of individualized dosing levels are sent to pharmacy for dosing calculations. Pharmacy makes recommendations.
 - If patient's most recent serum creat is > 2.0 mg/dL, call pharmacy for timing for individualized dosing specimens.
 - **Specimen Type:** Serum/Plain Red Top/Plasma Lithium heparin – Separate from cells within 2 hours of collection.
 - **Volume:** 1 mL
 - **Container:** SST/Plain Red Top/Plasma Lithium Heparin
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial. Timing is critical. Return to lab immediately. Avoid hemolysis.
 - **Storage:** Refrigerate.

-
- **Test Name:** GLA SEQUENCING
 - **LC Test Code:** 252225
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | LabCorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** GLIAL FIBRILLARY ACID PROTEIN
- **LC Test Code:** 484440
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GIARDIA ANTIGEN
 - **Test ID:** GIARD
 - **Test Includes:** Enzyme immunoassay for Giardia antigen.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done as needed. Results are available in 2 - 5 days.
 - **Precollection Instructions:** See Ova & Parasite Exam for instructions.
 - **Specimen Type:** Stool, feces-coated rectal swab. Specimens collected in diapers are acceptable for this assay.
 - **Volume:** 1 gram or 1 mL liquid.
 - **Container:** Clean container with a tight-fitting lid.
 - **Specimen Handling:** Collection: See Ova & Parasite Exam for instructions.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate. Microbiology will freeze specimen upon receipt.

-
- **Test Name:** GLIADIN ANTIBODY
 - **Test ID:** GLIAD
 - **Test Includes:** Gliadin antibody IgG and IgA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate.

-
- **Test Name:** GLUCAGON PLASMA
 - **LC Test Code:** 004622
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GLUCOSE TOLERANCE, 2 HOUR PREGNANCY
 - **Test ID:** GTTP2
 - **Synonyms:** Diabetes screening for patients with gestational diabetes during pregnancy
 - **Test Includes:** Serum glucose: Fasting, 1 hour, 2 hours. No urines
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 2 hours after the last specimen is received in the lab
 - **Precollection Instructions:** See Precollection Instructions for GLUCOSE TOLERANCE. Dose: 75g glucose.
 - **Specimen Type:** Serum

Aultman Hospital Laboratory Test Directory

- **Volume:** 1 mL
- **Container:** 5 mL SST
- **Storage:** Refrigerate.

-
- **Test Name:** GLUCOSE TOLERANCE, 2, 3, 4 Hour
 - **Test ID:** GTT2, GTT3, GTT4
 - **Test Includes:** Serum glucose
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 2 hours after last specimen is received in lab
 - **Precollection Instructions:**
 - Schedule test with venipuncture 24 hours in advance,
 - Patients should be NPO after midnight except for water. NO SMOKING.
 - Obtain a bottle of glucola (normal dose for 3 or 4 hours: 100 grams glucose) (Normal dose for 2 hour is 75 grams glucose) (Note: Patients weighing less than 100 lb. DO NOT receive a full bottle of glucola. Call Chemistry (ext. 33994) for the exact amount to be given.)
 - The morning of the test, both a fasting glucose and a fasting urine must be collected and analyzed prior to the administration of the glucola. Chemistry will call the nursing unit with the time to administer the glucola. The glucola must be consumed within 20 minutes of when it is first administered.
 - **SPECIAL CONDITIONS**
 - Do Not schedule any other procedures, X-rays or therapy with the tolerance test. As per the American Diabetes Association, test accuracy depends upon minimal stress and activity by the patient.
 - Do Not catheterize the patient for the test unless specifically ordered by the attending physician.
 - During the test, avoid any medication that coats the stomach or intestines. If barium has been given previous to the tolerance test, allow 2 - 3 days for the barium to clear the intestinal tract before the tolerance is performed.
 - If the patient becomes ill during the tolerance test and vomits, notify the lab to discontinue testing. Notify the physician that the test was discontinued.
 - **Specimen Type:** Serum
 - **Volume:** For each collection: Serum: 1 mL. For each collection: Serum: 1 mL
 - **Container:** Serum: 7 mL SST
 - **Storage:** Refrigerate.

-
- **Test Name:** GLUCOSE, 1 HOUR CHALLENGE
 - **Test ID:** GLU1P
 - **Precollect Instructions:** 50 Grams of Glucola
 - **Test Includes:** 1-hour glucose
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 5 mL SST

Aultman Hospital Laboratory Test Directory

- **Storage:** Refrigerate.

-
- **Test Name:** GLUCOSE, 2 HOUR POSTPRANDIAL
 - **Test ID:** GLU2P
 - **Test Includes:** 2-hour glucose
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Precollection Instructions:** A glucose level is to be drawn 2 hours after a 75g glucose load is given (preferable) or following a breakfast or lunch containing at least 100 g of carbohydrate.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 5 mL SST
 - **Storage:** Refrigerate.

-
- **Test Name:** GLUCOSE, 24 HOUR URINE
 - **Test ID:** GLU24
 - **Test Includes:** Collection time, volume, creatinine, glucose
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week. Results are available on the same day if the specimen is received by 1000.
 - **Precollection Instructions:** See 24-hour Urine Collection Instructions.
 - **Specimen Type:** 24-hour urine
 - **Volume:** Submit entire collection to laboratory
 - **Container:** Obtain collection container from laboratory
 - **Specimen Handling:** Keep specimen on ice during collection
 - **Storage:** Refrigerate.

-
- **Test Name:** GLUCOSE, BODY FLUID
 - **Test ID:** GLUBF
 - **Test Includes:** Body fluid type, glucose
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Body fluid
 - **Volume:** 2 – 4 mL
 - **Storage:** Refrigerate.

-
- **Test Name:** GLUCOSE, RANDOM URINE
 - **Test ID:** GLUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Random urine

Aultman Hospital Laboratory Test Directory

- **Volume:** 2 – 4 mL
- **Container:** Plastic urine tube
- **Storage:** Refrigerate.

-
- **Test Name:** GLUCOSE, SERUM
 - **Test ID:** GLUCOSE
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
Volume: 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate.

-
- **Test Name:** GLYCOMARK (R) (1,5 AG)
 - **LC Test Code:** 500115
 - **Aultman Test Code:** GLYMRK
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** GM1 IGG/M AUTOANTIBODIES
 - **LC Test Code:** 140250
 - **Aultman Test Code:** GM1ABP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GONADOTROPIN RELEASING HORMONE
 - **LC Test Code:** 502421
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** GONORRHOEAE PCR - FEMALE
 - **Test ID:** NGPCR1
 - **Test Includes:** PCR test to detect N. gonorrhoeae. NOTE: Specimens collected using the Cobas PCR collection kit cannot be used for culture.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily M - F. Results available within 1-2 days.
 - **Precollection Instructions:** Use only Powder Free gloves.
 - **Specimen Type:** Endocervix, vagina, urine.
 - **Volume:** 1 Cobas PCR female swab, 5mL urine.
 - **Container:** Cobas PCR transport tube.
 - NOTE: Specimens collected using the Cobas PCR collection kit cannot be used for culture.
 - **Specimen Handling:**
 - **Female:**

Aultman Hospital Laboratory Test Directory

- Remove excess mucus from the cervical os and surrounding mucosa using one of the swabs provided. Discard this swab.
- Insert the second swab from the collection kit 1 - 12 cm into the endocervical canal.
- Rotate the swab clockwise in the endocervical canal for 30 seconds to ensure adequate sampling.
- Withdraw the swab carefully. Avoid any contact with vaginal mucosa.
- Insert the swab into the Remel transport tube. Snap off the shaft at the score line or cut shaft to fit tube.
- Cap tube. Label with the patient's name.
- **Male:**
 - 5mL urine in a sterile container.
- **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
- **Storage:** Room temperature. If submitting urine, refrigerate urine.

-
- **Test Name:** GROUP B STREP PCR
 - **Test ID:** GBSPCR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done Monday-Saturday. Results available in 2 days.
 - **Specimen Type:** Vagina, rectum.
 - **Volume:** 1 swab of each.
 - **Container:** Culturette (2 swabs)
 - **Specimen Handling:** Collection: With a culturette, obtain one swab each of the vaginal introitus and the anorectum. Cervical cultures are not acceptable for determining colonization. A speculum should not be used for culture collection.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate.

-
- **Test Name:** GROWTH HORMONE SERUM
 - **LC Test Code:** 004275
 - **Aultman Test Code:** GH
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** HER2 BY FISH
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** HER2 FISH
 - **Reference Lab Link:** [Test Menu | Labcorp Oncology](#)

-
- **Test Name:** H. PYLORI STOOL AG, EIA
 - **LC Test Code:** 180764
 - **Aultman Test Code:** HPYLAG
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** H. PYLORI IGG ANTIBODY

Aultman Hospital Laboratory Test Directory

- **Test ID:** HP
- **Synonyms:** Helicobacter pylori Antibody
- **Test Includes:** Detects Helicobacter pylori IgG antibody only
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done twice weekly M – F. Results are available within 3 days.
- **Specimen Type:** Serum
- **Volume:** 1 mL
- **Container:** 7 mL SST

-
- **Test Name:** HAEMOPHILUS INFLUENZAE B IGG
 - **LC Test Code:** 138271
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HAPTOGLOBIN
 - **Test ID:** HAPTO
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis and lipemia. Refrigerate, after 72 hours freeze.

- **Test Name:** HBV GENOTYPE
- **LC Test Code:** 551710
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** HCG, QUANTITATIVE
- **Test ID:** HCG
- **Synonyms:** This assay is used to monitor hCG as an assessment of pregnancy. It is not the assay of choice for use as a tumor marker. To order hCG as a tumor marker, order AHCGT.
- **Test Includes:** LMP, hCG
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 24 hours/day, 7 days/week. Results available on the same day.
- **Precollection Instructions:** Note the date of LMP when ordering.
- **Specimen Type:** Serum
- **Volume:** 2 mL
- **Container:** 7 mL SST
- **Storage:** Refrigerate.

-
- **Test Name:** HCG, TUMOR MARKER

Aultman Hospital Laboratory Test Directory

- **Test ID:** HCGT
 - **Synonyms:** This test is intended to be used as a tumor marker. It should not be used for the assessment of pregnancy. For pregnancy assessment, order "HCG."
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 24 hours/day, 7 days/week. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate.
-
- **Test Name:** HCV GENOSURE(R) NS3/4A
 - **LC Test Code:** 550540
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** HCV GENOTYPING NON-REFLEX
 - **LC Test Code:** 550475
 - **Aultman Test Code:** HCGEN
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-
- **Test Name:** HCV GT3 NS5A DRUG RESIST ASSAY
 - **LC Test Code:** 550603
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** HCV RT-PCR, QUANT(NON-GRAPH)
 - **LC Test Code:** 550080
 - **Aultman Test Code:** HCQPCR
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
-
- **Test Name:** HDL CHOLESTEROL ONLY
 - **Test ID:** HDO
 - **Test Includes:** HDL cholesterol ONLY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results are available the next day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate
-
- **Test Name:** HDV ANTIBODY IGG/IGM
 - **LC Test Code:** 144012
 - **Aultman Test Code:** MISC

Aultman Hospital Laboratory Test Directory

- **LabCorp Link:** [Search](#) | [Labcorp](#)
 - **Test Name:** HEAVY METALS PROFILE II, BLOOD
 - **LC Test Code:** 706200
 - **Aultman Test Code:** HMETB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
 - **Test Name:** HEAVY METALS PROFILE II, URINE
 - **LC Test Code:** 250281
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [LabCorp](#)
 - **Test Name:** HELPER/SUPPRESSOR
 - **Test ID:** HELP1
 - **Synonyms:** CD3/CD4
 - **Test Includes:** Total lymphocytes, number of T cells, natural killer cells, helper cells and suppressor cells, % total of each cell type and helper/suppressor ratio (CD3, CD4, CD8, CD19, CD56).
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 2 working days
 - **Specimen Type:** Whole blood
 - **Volume:** 5 mL
 - **Container:** 5 mL lav top tube
 - **Specimen Handling:** Deliver specimen to lab ASAP. Keep specimen at room temperature.
 - **Test Name:** Helper/Suppressor (Pediatrics <12 years old)
 - **Test ID:** IMMDEF
 - **Testing Facility:** Reference Laboratory
 - **Specimen Type:** Edta Whole Blood
 - **Container:** Edta Lavender Top Tube
 - **Storage:** Room Temperature
 - **Special Instructions:** Deliver specimens to testing laboratory within 24 hours of collection.
-
- **Test Name:** HEMATOCRIT
 - **Test ID:** HCT
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 30-60 minutes, Routine: 6 hours
 - **Specimen Type:** Whole blood (EDTA)
 - **Volume:** 2 mL
 - **Container:** 2.5 mL lav top tube
 - **Storage:** Room temperature.

Aultman Hospital Laboratory Test Directory

- **Test Name:** HEMATOCRIT, SPUN
- **Test ID:** SHCT
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** 6 hours
- **Specimen Type:** Whole blood, EDTA
- **Container:** 2.5 ml lav top tube
- **Storage:** Room temperature

- **Test Name:** HEMOGLOBIN
- **Test ID:** HGB
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Urgent/STAT: 30-60 minutes, Routine: 6 hours
- **Specimen Type:** Whole blood (EDTA)
- **Volume:** 2 mL
- **Container:** 2.5 mL lav top tube
- **Storage:** Room temperature.

-
- **Test Name:** HEMOGLOBIN ELECTROPHORESIS
 - **Test ID:** HGBE
 - **Synonyms:** Hemoglobin fractionation
 - **Test Includes:** Hemoglobin A1, A2, S and F. Test includes interpretation by a pathologist.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done weekly on Thursday
 - **Specimen Type:** Whole blood (EDTA)
 - **Volume:** 5 mL
 - **Container:** 5 mL lav top tube

-
- **Test Name:** HEMOGLOBIN, PLASMA
 - **Test ID:** HGBP
 - **Testing Facility:** Reference Laboratory
 - **Specimen Handling:** 2 ML PLASMA FROM NA HEPARIN - REF
 - **Additional Information:** Avoid hemolysis; separate from cells within 1 hour of collection.
 - **Test Name:** HEMOGRAM, WITH PLATELETS
 - **Test ID:** HGMP
 - **Test Includes:** WBC, RBC, hemoglobin, hematocrit, indices and platelets
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 30 minutes, Routine: 6 hours
 - **Specimen Type:** Whole blood (EDTA)
 - **Volume:** 2 mL
 - **Container:** 2.5 mL lav top tube
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** HEMOSIDERIN URINE

- **LC Test Code:** 829482
- **Aultman Test Code:** HEMOS
- **LabCorp Link:** [Search](#) | [LabCorp](#)

- **Test Name:** HEP BE AB

- **LC Test Code:** 006635
- **Aultman Test Code:** HB
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** HEP BE AG

- **LC Test Code:** 006619
- **Aultman Test Code:** HBEAG
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** HEPARIN INCLUDED PLATELET W/RFX

- **LC Test Code:** 150031
- **Aultman Test Code:** HIT
- **LabCorp Link:** [Search](#) | [LabCorp](#)

- **Test Name:** HEPATIC FUNCTION PANEL

- **Test ID:** HFP
- **Test Includes:** Albumin, total bilirubin, direct bilirubin, alk phos, AST, ALT, total protein
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
- **Specimen Type:** Plasma lithium heparin (PST)
- **Volume:** 2 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
- **Specimen Handling:** Avoid hemolysis. Separate serum from cells within 2 hours of collection.

- **Test Name:** HEPATITIS An ANTIBODY IGG

- **CCF Test Code:** AHAVG
- **Aultman Test Code:** AHAVG
- **LabCorp Link:** [Test Directory - Online Test Catalog](#) | [Cleveland Clinic Laboratories](#)

- **Test Name:** HEPATITIS A IGM ANTIBODIES

- **Test ID:** AHAVM
- **Test Includes:** Screening for IgM antibodies to Hepatitis A testing.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
- **Specimen Type:** Serum or plasma lithium heparin – Separate from cells within 2 hours of collection.
- **Volume:** 1 – 2 mL

Aultman Hospital Laboratory Test Directory

- **Container:** SST or Plasma Lithium Heparin.
- **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** HEPATITIS B CORE IGM
 - **Test ID:** HBCM
 - **Synonyms:** HBcIgM, Core-M, Core-IgM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** HEPATITIS B SURFACE ANTIBODY
 - **Test ID:** HBSAB
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** HEPATITIS B SURFACE ANTIGEN
 - **Test ID:** HBSAG
 - **Test Includes:** All repeatedly reactive (positive) Hepatitis B Surface Ag tests are confirmed by neutralization.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** HEPATITIS B VIRAL DNA ULTRA QUANT
 - **CCF Test Code:** HBVDNU
 - **Aultman Test Code:** HBDNA
 - **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** HEPATITIS C ANTIBODY
 - **Test ID:** HCV
 - **Test Includes:** Hepatitis C antibody (2nd generation)
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done Daily

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** Serum
- **Volume:** 2 mL
- **Container:** SST
- **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** HEPATITIS E IGGG/IGM AB
 - **LC Test Code:** 832665
 - **Aultman Test Code:** HEPE
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** HEPATITIS, ACUTE PROFILE
 - **Test ID:** HEPAC
 - **Test Includes:** Hepatitis B Surface Ag, Hepatitis B Core IgM Ab, Hepatitis C Ab, Hepatitis A IgM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 3 – 5 mL
 - **Container:** SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** HERED HEMOCHROMATOSIS, DNA
 - **LC Test Code:** 511345
 - **Aultman Test Code:** HEMOC
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** 5-HIAA QUANT 24 HR URINE
 - **LC Test Code:** 004069
 - **Aultman Test Code:** 5HIAA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** HHV-6 IGG
- **LC Test Code:** 161075
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HHV-6 IGG/M ANTIBODIES QUANT
 - **LC Test Code:** 138973
 - **Aultman Test Code:** HV6ABS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HYALURONIC ACID SERUM
 - **LC Test Code:** 829680
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** HISTAMINE DETERMINATION BLOOD
 - **LC Test Code:** 081315
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HISTAMINE DETERMINATION URINE
 - **LC Test Code:** 144618
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HISTAMINE PLASMA
 - **LC Test Code:** 144600
 - **Aultman Test Code:** PHISTA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HISTOPLASMA CAPSULATUM ABS
 - **LC Test Code:** 138635
 - **Aultman Test Code:** HISIDCF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HISTOPLASMA GAL MANNAN AG UR
 - **LC Test Code:** 183560
 - **Aultman Test Code:** HISTOAG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HIV 1 / 2 ANTIBODY
 - **Test ID:** HIV
 - **Test Includes:** Assay for the detection of HIV p24 antigen and antibodies to HIV Type 1, including Group O (HIV-1 +
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** HLA-A29 UVEITIS
 - **LC Test Code:** 167390
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HLA B 27 DISEASE ASSOCIATION
 - **LC Test Code:** 006924

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** HLAB27
- **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** HLAB 51 DISEASE ASSOCIATION
 - **LC Test Code:** 167389
 - **Aultman Test Code:** HLAB51
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HLA B5701 TEST
 - **LC Test Code:** 006926
 - **Aultman Test Code:** B5701
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HNK1 (CD57) PANEL
 - **LC Test Code:** 505026
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HOMOCYSTEINE
 - **Test ID:** HOMO
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 24 hours/day, 7 days/week. Results available on the same day.
 - **Transfer Instructions:** transfer on ice
 - **Specimen Type:** Plasma only (Lithium Heparin)
 - **Volume:** 1 mL
 - **Container:** Lithium Heparin
 - **Specimen Handling:** Keep on ice until centrifuged. After centrifugation, separate plasma from cells ASAP.
 - **Storage:** Refrigerate.

-
- **Test Name:** HPA-1A (PLA 1 PLT AG) GENOTYP
 - **LC Test Code:** 500314
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HPV PCR SCREEN
 - **Test ID:** HPV
 - **Test Includes:** DNA probe for the detection of Human Papilloma Virus (HPV), high risk types including Genotype 16 & 18.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Results are available within 2-3 days.
 - **Specimen Collection /** Collection: Preservative solutions: Follow collection device instructions.

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** Preserve (Thin Prep) preservative
- **Container:** PreservSITE (Thin Prep) container
- **Specimen Handling:** Transport: Deliver to Microbiology within 2 hours of collection for inpatients.

-
- **Test Name:** HSV 1 AND 2 AB, IGG
 - **LC Test Code:** 164099
 - **Aultman Test Code:** HSVG12
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HSV ½ PCR
 - **LC Test Code:** 138651
 - **Aultman Test Code:** PCRHSV
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HSV PCR CSF
 - **LC Test Code:** 139800
 - **Aultman Test Code:** HSPCRC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HSV AND VZV PCR PANEL
 - **LC Test Code:** 139367
 - **Aultman Test Code:** HSVVZV
 - **LabCorp Link:** [Search | LabCorp](#)

-
- **Test Name:** HTLV-I/II ANTIBODIES QUAL
 - **LC Test Code:** 163246
 - **Aultman Test Code:** HTLV1
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HUMAN ANTI MOUSE ANTIBODIES
 - **LC Test Code:** 140657
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HUMAN EPIDIDYMIS PROTEIN 4
 - **LC Test Code:** 081700
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** HUMAN PLACENTAL LACTOGEN
 - **LC Test Code:** 283891
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** HYDROCOCODONE+ METABOLITES U
- **LC Test Code:** 803301
- **Aultman Test Code:** HYDCU
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** HYPERSENSITIVITY PNEUMONITIS
 - **LC Test Code:** 660670
 - **Aultman Test Code:** HYPNE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I001-IGE HONEYBEE
 - **LC Test Code:** 602886
 - **Aultman Test Code:** HONBEE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** I002-IGE HORNET, WHITE FACE
- **LC Test Code:** 602887
- **Aultman Test Code:** WTHORN
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** I003-IGE YELLOW JACKET
- **LC Test Code:** 602889
- **Aultman Test Code:** YELJAK
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** I004-IGE PAPER WASP
- **LC Test Code:** 602890
- **Aultman Test Code:** PAWASP
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I005-IGE HORNET YELLOW
 - **LC Test Code:** 602891
 - **Aultman Test Code:** YLHORN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I006-IGE COCKROACH GERMAN
 - **LC Test Code:** 602488
 - **Aultman Test Code:** CROACH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I008-IGE MOTH
 - **LC Test Code:** 602892
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** I070-IGE FIRE ANT (INVICTA)
- **LC Test Code:** 602893
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** I071-IGE WHOLE BODY: MOSQUITO
- **LC Test Code:** 602894
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I073-IGE MIDGE
 - **LC Test Code:** 602895
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I204-IGE HONESTLY
 - **LC Test Code:** 602893
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I204-IGE HORSEFLY
 - **LC Test Code:** 602898
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** I205-IGE BUMBLEBEE
 - **LC Test Code:** 602897
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** IA-2 AUTOANTIBODIES
 - **LC Test Code:** 141531
 - **Aultman Test Code:** IA2AB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** IDH1/IDH2 MUTATION ANALYSIS
 - **LC Test Code:** 481484
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** IFE, SERUM
 - **Test ID:** IFES
 - **Synonyms:** Serum immunofixation electrophoresis
 - **Test Includes:** Interpretation of IFE by a pathologist

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily as needed. Results available the same day.
- **Precollection Instructions:** Fasting specimen preferred.
- **Specimen Type:** Serum
- **Volume:** 2 mL
- **Container:** 7 mL SST.
- **Specimen Handling:** Avoid hemolysis and lipemia.

-
- **Test Name:** IFE, URINE
 - **Test ID:** IFEU
 - **Synonyms:** Urine immunofixation electrophoresis.
 - **Test Includes:** Interpretation of IFE by a pathologist.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily as needed. Results available the same day.
 - **Precollection Instructions:** A 24-hour collection is preferred. See 24-hour Urine Collection Instructions. A random urine can be used instead of a 24-hour urine. The random urine should be the first morning void.
 - **Specimen Type:** 24-hour urine or random urine.
 - **Volume:** Submit entire urine collection to lab. Random urine should be at least 50 mL.
 - **Container:** Obtain urine collection container from lab
 - **Specimen Handling:** Refrigerate. Keep collected urine on ice during collection.

-
- **Test Name:** IGE CASHEW NUT W/COMPONENT REFLEX
 - **LC Test Code:** 604159
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGE HAZELNUT W/COMPONENT RFLX
 - **LC Test Code:** 604734
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGE MILK COMPONENT PROFILE
 - **LC Test Code:** 603849
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGE MILK W/COMPONENT REFLEX
 - **LC Test Code:** 603927
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGE WALNUT W/COMPONENT REFLEX
 - **LC Test Code:** 603927

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGF-1
 - **LC Test Code:** 010363
 - **Aultman Test Code:** SOMAT
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGF-2
 - **LC Test Code:** 506262
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGF-BP1
 - **LC Test Code:** 140822
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGP-BP3
 - **LC Test Code:** 140152
 - **Aultman Test Code:** IGFBP3
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGG INDEX + SYNTHESIS RATE
 - **LC Test Code:** 203836
 - **Aultman Test Code:** IGGSF
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGG SUBCLASS 4
 - **LC Test Code:** 160531
 - **Aultman Test Code:** IGG4
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGG, SUBCLASSES (1-4)
 - **LC Test Code:** 209601
 - **Aultman Test Code:** IG1234
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IGHV SOMATIC HYPERMUTATION
 - **LC Test Code:** 113753
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IL-2 RECEPTOR ALPHA

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 142455
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** IMIPRAMINE(TOFRANIL) SERUM
 - **LC Test Code:** 007468
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** IMMUNOGLOBULIN A
 - **Test ID:** IGA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** IMMUNOGLOBULIN D, QUANT, SERUM
 - **LC Test Code:** OO2162
 - **Aultman Test Code:** IGD
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** IMMUNOGLOBULIN E
 - **Test ID:** IGE
 - **Synonyms:** IgE
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available the same day.
 - **Specimen Type:** Serum only
 - **Volume:** 1 mL
 - **Container:** 7 mL SST.

-
- **Test Name:** IMMUNOGLOBULIN G
 - **Test ID:** IGG
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** IMMUNOGLOBULIN M
 - **Test ID:** IGM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available the same day.

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** plasma lithium heparin (PST)
- **Volume:** 1 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** IMMUNOGLOBULIN PANEL
 - **Test ID:** IMMUN
 - **Test Includes:** IgA, IgG and IgM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** SST

-
- **Test Name:** INFLAMMATORY BOWEL DISEASE-IBD
 - **LC Test Code:** 164830
 - **Aultman Test Code:** IBDDIA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** INFLIXIMAB DRUG + ANTIBODY
 - **LC Test Code:** 503870
 - **Aultman Test Code:** IFXNEU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** INHIBIN ULTRASENSITIVE
 - **LC Test Code:** 146803
 - **Aultman Test Code:** INHIBA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** INHIBIN B
 - **LC Test Code:** 146795
 - **Aultman Test Code:** INHIBB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** INSULIN
 - **Test ID:** INSLN
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 ml SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** INSULIN ANTIBODIES
 - **LC Test Code:** 141598

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** INSLAB
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** INTERLEUKIN-2 SERUM
 - **LC Test Code:** 140912
 - **Aultman Test Code:** INT2
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** INTERLEUKIN-6 SERUM
 - **LC Test Code:** 140916
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** INTERLEUKIN-8 PLASMA
 - **LC Test Code:** 140818
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** INTRINSIC FACTOR ABS, SERUM
 - **LC Test Code:** 010413
 - **Aultman Test Code:** INTAB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IODINE SERUM PLASMA
 - **LC Test Code:** 070034
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** IRON STUDIES
 - **Test ID:** FES
 - **Synonyms:** Fe studies
 - **Test Includes:** Total iron, iron binding capacity (IBC) and % saturation
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Test performed 7 days/week, 24 hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis. Separate serum from cells within 2 hours of collection.
 - **Storage:** Refrigerate.

-
- **Test Name:** IRON STUDY, LIVER BIOPSY

Aultman Hospital Laboratory Test Directory

- **Test ID:** PATH
- **Test Number:** FET
- **Testing Facility:** Reference Laboratory
- **Specimen Handling:** LIVER BX OR PARAFFIN BLOCK
- **Additional Information:** REFRIGERATE; SEND PATH REPORT WITH SPECIMEN

-
- **Test Name:** IRON, TOTAL, SERUM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** SST
 - **Specimen Handling:** Avoid hemolysis. Separate serum from cells within 2 hours of collection.
 - **Storage:** Refrigerate.

-
- **Test Name:** ITRACONAZOLE AND MTB S/P
 - **LC Test Code:** 700833
 - **Aultman Test Code:** ITRAC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** JAK2 V617 MUTATION DETECTION BLOOD
 - **CCF Test Code:** JAK2
 - **Aultman Test Code:** JAK2 V617F
 - **LabCorp Link:** [Test Directory - Online Test Catalog](#) | [Cleveland Clinic Laboratories](#)

-
- **Test Name:** JALAPENO PEPPER IGE
 - **LC Test Code:** 826359
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** JC VIRUS DNA, PCR (CSF)
 - **LC Test Code:** 139340
 - **Aultman Test Code:** JCPCRB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** K082-IGE LATEX
 - **LC Test Code:** 602669
 - **Aultman Test Code:** LATEXA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test ID:** KETONES, URINE DIPSTICK
 - **Test ID:** KETUA

Aultman Hospital Laboratory Test Directory

- **TEST INCLUDES:** Dipstick for urine ketones.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
- **Specimen Type:** Random urine
- **Volume:** 2 mL
- **Container:** Plastic urine tube
- **Specimen Handling:** Keep on ice or refrigerating

-
- **Test Name:** KLEIHAUER-BETKE STAIN
 - **Test ID:** KBS
 - **Synonyms:** KB Stain
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 24 hours
 - **Specimen Type:** Whole blood (EDTA)
 - **Volume:** 3 mL
 - **Container:** 5 mL lav top tube
 - **Storage:** Refrigerated.

-
- **Test Name:** LACOSAMIDE
 - **LC Test Code:** 007012
 - **Aultman Test Code:** LACOS
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LACTIC ACID
 - **Test ID:** LAC
 - **Test Includes:**
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Grey top tube (Sodium fluoride/Potassium oxalate plasma)
 - **Specimen Handling:** Collect on ice. Avoid hemolysis. Deliver to lab immediately. Specimens for lactic acid should not be drawn outside the main hospital campus. Specimen must be processed 30 minutes from collection
 - **Volume:** 1 mL
 - **Container:** Sodium fluoride/Potassium oxalated plasma - grey top

-
- **Test Name:** LACTIC ACID, BODY FLUID
 - **Test ID:** LACBF
 - **Test Includes:** Body fluid type, lactic acid
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Body fluid
 - **Volume:** 2 – 4 ml
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** LACTIC ACID, CSF
- **Test ID:** LACSF
- **Synonyms:** Lactate
- **Test Includes:** Specimen type, lactic acid
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
- **Specimen Type:** CSF
- **Volume:** 1 mL
- **Container:** Deliver to lab ASAP

-
- **Test Name:** LACTIC ACID, WHOLE BLOOD
 - **Test ID:** WBLAC
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Additional Information:** Do not spin down
 - **Specimen Type:** Whole blood
 - **Volume:** 2mL
 - **Container:** 1 Lithium Heparin green top or 1 arterial/venous blood gas syringe
 - **Specimen Handling:** Collect on ice. Avoid hemolysis. Deliver to lab immediately. Specimens for lactic acid should not be drawn outside the main hospital campus.

-
- **Test Name:** LECTOFERRIN FECAL QUANT
 - **LC Test Code:** 123016
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LACTOSE TOLERANCE TEST
 - **Test ID:** LTT
 - **Test Includes:** Fasting glucose and glucose levels drawn at 15, 30, 45, 60, 90, and 120 minutes after the administration of lactose.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Within 2 hours after the last specimen is received.
 - **Precollection Instructions:** Schedule test with venipuncture 24 hours in advance.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 5 mL SST

-
- **Test Name:** LAMOTRIGINE (LAMICTAL), SERUM
 - **LC Test Code:** 716944
 - **Aultman Test Code:** LAM
 - **LabCorp Link:** [Search | Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** LD ISOENZYMES
 - **LC Test Code:** 001842
 - **Aultman Test Code:** LDISO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** LD, BODY FLUID
 - **Test ID:** LDBF
 - **Synonyms:** Body fluid LDH
 - **Test Includes:** Body fluid type, LD
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available on the same day
 - **Specimen Type:** Body fluid
 - **Volume:** 2 – 4 mL
-

- **Test Name:** LD, SERUM
 - **Test ID:** LD
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis
 - **Storage:** Refrigerate.
-

- **Test Name:** LDL, CHOLESTEROL DIRECT
 - **Test ID:** LDLC
 - **Testing Facility:** Aultman Hospital
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
-

- **Test Name:** LEAD, BLOOD(ADULT)
 - **LC Test Code:** 007625
 - **Aultman Test Code:** LEADA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** LEAD, BLOOD (PEDIATRIC)
 - **LC Test Code:** 717009
 - **Aultman Test Code:** LEADP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** LEGIONELLA PNEUMOPHILA ABS
- **LC Test Code:** 164616

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** LEGAB
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LEGIONELLA SPECIES CULTURE
 - **LC Test Code:** 086868
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LEGIONELLA URINE ANTIGEN
 - **Test ID:** LUA
 - **Test Includes:** Rapid immunochromatographic essay for detection of legionella pneumophila serogroup 1 antigen in urine.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done Mon – Fri, 24 hours a day, Sat and Sun 0700 – 2400. Performed Sat and Sun 0000-0700, Stat requests only. Results available the same day.
 - **Specimen Type:** Urine (catheter or clean void).
 - **Volume:** 2 mL.
 - **Container:** Sterile container.
 - **Specimen Handling:** Collection:
 - Catheter: Swab catheter port with povidone iodine. Puncture the port with a needle and aspirate the urine into a syringe. Do Not collect urine from the drainage bag.
 - Clean void: Cleanse urinary meatus with towelettes. Have the patient void a small amount for discard. Collect a midstream urine specimen.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate.

-
- **Test Name:** LEPTIN SERUM
 - **LC Test Code:** 146712
 - **Aultman Test Code:** LEPTIN
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LEPTOSPIRA IGM
 - **LC Test Code:** 830035
 - **Aultman Test Code:** LEPTAB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LEVETIRACETAM(KEPPRA), S
 - **LC Test Code:** 716936
 - **Aultman Test Code:** LKMAB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LH
 - **Test ID:** LH

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily. Results available the same day.
- **Specimen Type:** Serum
- **Volume:** 1 mL
- **Container:** SST
- **Storage:** Refrigerate.

-
- **Test Name:** LIDOCAINE(XYLOCAINE) SERUM
 - **LC Test Code:** 007013
 - **Aultman Test Code:** LIDO
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LIPASE, SERUM
 - **Test ID:** LIP
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate.

-
- **Test Name:** LIPID PROFILE
 - **Test ID:** LIPID
 - **Test Includes:** Cholesterol, triglycerides, HDL, LDL and VLDL cholesterol and cholesterol/HDL cholesterol ratio
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the next day.
 - **Specimen Type:** Serum or Plasma lithium heparin – Separate from cells within 2 hours of collection.
 - **Volume:** 2 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate
 - **LDL Calculation:** $\text{Cholesterol} - (\text{HDL Cholesterol} + \text{Triglycerides} / 5)$

-
- **Test Name:** LIPOPROTEIN (a)
 - **LC Test Code:** 120188
 - **Aultman Test Code:** LIPOA
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** LIQUID-BASED PAP SMEAR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm. 4 working days. Abnormal PAP smear results take longer pending Pathologist review.
 - **Precollection Instructions:**

Aultman Hospital Laboratory Test Directory

- Requisition: Use Cerner requisition for in-patient specimens. Use Form 308-A (Cytology) for outpatient specimens
 - **Specimen Collection:** The following information must be included on the requisition:
 - **Transfer Instructions:** The patient's name, birth date, medical record number
 - Mark the source (cervical, vaginal, endocervical, vault)
 - Include any pertinent clinical information (esp. post-menopausal bleeding, previous abnormal history)
 - Mark the HPV Reflex box if additional testing is requested with an ASCUS result
 - Mark the "HPV Testing Regardless" box if HPV testing is requested no matter what the diagnosis is.
 - List any physicians that require a copy of the report
 - The patient's Social Security number is helpful but not required
 - **Materials needed:**
 - Vial used to collect and transport the specimen
 - Cervix brush collection device or the combo kit collection device
 - Specimen vials and collection device supplies are available through Cytology.
 - **Collection procedure**
 - NOTE: THIS PROCEDURE MUST BE PERFORMED BY A PHYSICIAN OR A NURSE PRACTITIONER UNDER THE DIRECTION OF A PHYSICIAN.
 - Place the patient in the dorsolithotomy position.
 - Insert a vaginal speculum. Use warm water to warm up and lubricate the speculum. Do not use vaginal lubricants prior to specimen collection. Lubricants can interfere with cell transfer to the slide and obscure cells.
 - Insert the collection device into the endocervical canal.
 - Gently apply pressure until the bristles form against the Endocervical brush/spatula collection device - slowly rotate 1/4 or 1/2 turn in one direction. DO NOT over rotate.
 - Broom-like device - rotate the broom in a clockwise direction five times.
 - Rinse the collection device into the vial (10 times, swirling vigorously to further release material)
 - Discard collection device.
 - Tighten the cap to prevent leaking.
 - Vials with specimen will be viable for up to 4 weeks after collection. They do not need to be refrigerated.
 - **Container:** It is very important to record two patient identifiers on the specimen vial. The date of collection and the physician's name are also helpful for accessioning purposes. Specimen vials do not need to be refrigerated before or after collection. Do not use the vials after the expiration date on the vial. Specimens are viable up to 4 weeks after collection.
 - **Specimen Handling:** Place the vial and the requisition into a biohazard bag for transport to the laboratory by the ALS courier. In-house Sure path specimens can also arrive via the tube system. During normal day shift hours, the specimens should be delivered directly to Cytology and placed in the designated drop-off collection box in the department. After hours, the specimens can be dropped off in the ALS accessioning area in the designated drop-off place.
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** LITHIUM
- **Test ID:** LITH
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
- **Specimen Type:** Plasma lithium heparin
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** LIVER CYTOSOL TYPE 1 ABS
 - **LC Test Code:** 831680
 - **Aultman Test Code:** LIVERC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** LIVER-KIDNEY MICROSOMAL AB
 - **LC Test Code:** 163980
 - **Aultman Test Code:** LKMAB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** LP-PLA2 ACTIVITY
 - **LC Test Code:** 123282
 - **Aultman Test Code:** PLAA2
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** LYME DISEASE SEROLOGY WITH REFLEX
 - **LC Test Code:** 164266
 - **Aultman Test Code:** LMERLY
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** LYME (B. BURGDORFERI) PCR
 - **LC Test Code:** 138685
 - **Aultman Test Code:** LYPCR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** LYME, LINE BLOT, CSF
 - **LC Test Code:** 160457
 - **Aultman Test Code:** LYMECSF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** LYME, LINE BLOT, SERUM
 - **LC Test Code:** 163600
 - **Aultman Test Code:** LYMEWB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** LYSOZYME SERUM
 - **LC Test Code:** 080713

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** LYSOP
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M001 IGE PENCILLIUM CHRYSOGEN
 - **LC Test Code:** 602502
 - **Aultman Test Code:** PNOTAT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M002-IGE CLADOSPORIUM HERBARUM
 - **LC Test Code:** 602462
 - **Aultman Test Code:** CHERB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M003-IGE ALTERNARIA ALTERNATA
 - **LC Test Code:** 602455
 - **Aultman Test Code:** ATENS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M004-IGE MUCOR RACEMOSUS
 - **LC Test Code:** 602562
 - **Aultman Test Code:** MUCOR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M005- IGE CANDIDA ALBICANS
 - **LC Test Code:** 602548
 - **Aultman Test Code:** CNDIDA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M006-IGE ASPERGILLUS FUMIGATUS
 - **LC Test Code:** 602471
 - **Aultman Test Code:** AFUMIG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M009-IGE FUSARIUM PROLIFERATUM
 - **LC Test Code:** 602560
 - **Aultman Test Code:** FUSAR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M011-IGE RHIZOPUS NIGRICANS
 - **LC Test Code:** 602555
 - **Aultman Test Code:** RHIZOP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M013-IGE IGE PHOMA BETAE
 - **LC Test Code:** 602552

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** PHOMA
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M014-IGE EPICOCCUM PURPUR
 - **LC Test Code:** 602559
 - **Aultman Test Code:** EPIC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M016-IGE CURVULARIA LUNATA
 - **LC Test Code:** 602901
 - **Aultman Test Code:** CURL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M046-IGE BIPOLARIS
 - **LC Test Code:** 602909
 - **Aultman Test Code:** FCURV
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M202-IGE ACREMONIUM KILIENSE
 - **LC Test Code:** 602558
 - **Aultman Test Code:** ACRKI
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M208-IGE CHAETOMIUM GLOBOSUM
 - **LC Test Code:** 602917
 - **Aultman Test Code:** CHAETG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MACROPROLACTIN
 - **LC Test Code:** 500324
 - **Aultman Test Code:** MACPRO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MACROSCOPIC EXAM ATHROPOD
 - **Test ID:** EXAM A
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 7am-3pm Monday – Friday Results available in 8-24 hours
 - **Specimen Type:** Ticks, Bed Bugs, Body Louse, Fleas, and Mites
 - **Container:** Submit in Sterile Container
 - **Storage:** Room Temperature

-
- **Test Name:** MACROSCOPIC EXAM PARASITE
 - **Test ID:** EXAMP
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** 7am-3pm Monday – Friday Results available in 8-24 hours
- **Specimen Type:** Larve and Worms
- **Container:** Submit in Sterile Container
- **Storage:** Room Temperature

-
- **Test Name:** MAG IGM AUTOANTIBODIES
 - **LC Test Code:** 140120
 - **Aultman Test Code:** MAGIGM
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** MAGNESIUM URINE
 - **LC Test Code:** 003400
 - **Aultman Test Code:** MGU24
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** MAGNESIUM RBC
 - **LC Test Code:** 080283
 - **Aultman Test Code:** RBCMAG
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** MAGNESIUM, SERUM
 - **Test ID:** MG
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week – 24 hours/day. Results available the same day.
 - **Specimen Type:** plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis
 - **Storage:** Refrigerate.

-
- **Test Name:** MALARIA and Blood Parasites
 - **Test ID:** MAL
 - **MUST PROVIDE:** Patient must provide recent travel history.
 - **Test Includes:** Microscopic examination for Malaria (Plasmodium sp.) and other blood parasites such as Babesia, Microfilaria and Trypanosomes.
 - **Testing Facility:** Aultman Hospital Laboratory
 - **Turnaround Time:** Performed daily with STAT priority. Must be received within the main lab within 1 hour.
 - **Specimen Type:** 4 ml EDTA tube (lavender). Must be drawn at Aultman Main ED, main Outpatient Lab or within the main hospital.
 - **Specimen Handling:** Draw a separate EDTA for Malaria.

-
- **Test Name:** MANGANESE BLOOD

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 724195
- **Aultman Test Code:** MANGAN
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MANNOSE BINDING LECTIN (MBL)
 - **LC Test Code:** 004900
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MEASLES IGM
 - **LC Test Code:** 160178
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MEASLES (RUBEOLA) PCR SWAB
 - **LC Test Code:** 138973
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MELANOCYTE STIMULATING HORMONE
 - **LC Test Code:** 010421
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MENINGITIS ENCEPHALITIS CSF PCR SCREEN
 - **Test ID:** MEPCR
 - **Testing Facility:** Aultman Hospital
 - **Test Includes:** Detection of Cytomegalovirus, Herpes simplex viruses 1 and 2, Enterovirus, Human herpesvirus 6, Human Parechovirus, Varicella Zoster virus, Escherichia coli K1, Haemophilus influenzae, Listeria monocytogenes, Neisseria meningitidis, streptococcus agalactiae, Streptococcus pneumoniae, and Cryptococcus neoformans/gattii.
 - **Turnaround Time:** 2 Hours, Offered 24 hours a day
 - **Specimen Handling:** 0.5ml CSF Fluid
 - **Additional Information:** CSF Collection tube #3 or #4 Refrigerate

-
- **Test Name:** MERCURY BLOOD
 - **LC Test Code:** 085324
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MERCURY SERUM PLASMA
 - **LC Test Code:** 810259
 - **Aultman Test Code:** MERC2
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** MERCURY URINE
 - **LC Test Code:** 007773
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** METHADONE CONF MS UR
 - **LC Test Code:** 727000
 - **Aultman Test Code:** METH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** METANEPHRINES, FRAC, PL, FREE
 - **LC Test Code:** 121806
 - **Aultman Test Code:** ACE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** METANEPHRINES, FRAC, QN, 24-HR
 - **LC Test Code:** 004234
 - **Aultman Test Code:** METU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** METANEPHRINES PHEOCHROMOCYT
 - **LC Test Code:** 004044
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** METHADONE/EDDP RATIO PLASMA
 - **LC Test Code:** 007443
 - **Aultman Test Code:** MMTAB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** METHEMOGLOBIN
 - **Test ID:** METHB
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available within 2 hours.
 - **Specimen Type:** Whole blood
 - **Volume:** 2 mL
 - **Container:** 5 mL green top tube
 - **Specimen Handling:** MUST BE DRAWN AT MAIN HOSPITAL OPD. MUST BE PROCESSED WITHIN 2 HRS
-

- **Test Name:** METHOTREXATE (MTX) SERUM
- **LC Test Code:** 007658
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

Aultman Hospital Laboratory Test Directory

-
- **Test Name:** METHYLMALONIC ACID, SERUM
 - **LC Test Code:** 706961
 - **Aultman Test Code:** MET
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** METHYLPHENIDATE MTB IA QL UR
 - **LC Test Code:** 718731
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** METHYLPHENIDATE + MTB, URINE
 - **LC Test Code:** 791170
 - **Aultman Test Code:** MPHNU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** METHYLPHENIDATE SERUM
 - **LC Test Code:** 715300
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** M GENITALIUM NAA URINE
 - **LC Test Code:** 180025
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MICROALBUMIN, 24 HOUR URINE
 - **Test ID:** MALBU
 - **Test Includes:** Collection time, volume, creatinine, microalbumin 24-hour excretion and excretion rate
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. If specimen is received by 0900, results are available on the same day.
 - **Precollection Instructions:** No preservative necessary. See 24-hour Urine Collection Instructions.
 - **Specimen Type:** 24-hour urine
 - **Volume:** Submit entire urine collection to lab
 - **Container:** Obtain collection container from lab
 - **Specimen Handling:** Keep specimen on ice during collection

-
- **Test Name:** MICROALBUMIN, RANDOM URINE
 - **Test ID:** MALBR
 - **Test Includes:** Microalbumin, albumin/creatinine ratio

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily. If specimen is received by 0900, results are available on the same day.
- **Specimen Type:** Urine, random
- **Volume:** 10 mL
- **Container:** Plastic urine tube

-
- **Test Name:** MICROSOMAL ANTIBODY Or Anti-Thyroid Peroxidase
 - **Test ID:** ATPO
 - **Test Includes:** Microsomal Antibody, Thyroid Peroxidase Antibodies, included in Thyroid Antibody test (THYAB) **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum or plasma lithium heparin – Separate from cells within 2 hours of collection.
 - **Volume:** 2 mL
 - **Container:** SST or Plasma Lithium Heparin.
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial. Freeze.

-
- **Test Name:** MICROSPORIDIA STAIN
 - **LC Test Code:** 828795
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** MITOCHONDRIAL ANTIBODY
 - **Test ID:** MITO
 - **Synonyms:** AMA
 - **Test Includes:** Mitochondrial antibody screen
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M – F. Results are available within 3 days.
 - **Specimen Type:** Serum
 - **Volume:** 1 – 2 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** MLH1 PROMOTER METHYLATION ANALYSIS
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** MLH1 METHYLATION
 - **Reference Lab Link:** [Test Menu | Labcorp Oncology](#)

-
- **Test Name:** MMP-9 (MATRIX METALLOPROT 9)
 - **LC Test Code:** 500124
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

Aultman Hospital Laboratory Test Directory

-
- - **Test Name:** MOLAR PREGNANCY EVALUATION
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** MOLAR PREGNANCY DNA PLOIDY & P57 (IHC)
 - **Reference Lab Link:** [Test Menu](#) | [Labcorp Oncology](#)
-
- **Test Name:** MONONUCLEOSIS
 - **Test ID:** MONO
 - **Synonyms:** Heterophile agglutination test, MONOSPOT
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Routine: Done daily M – F. Results available in 1 day. STAT/urgent: Done as received. Results available in 15 minutes.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis\
-
- **Test Name:** MONKEYPOX (ORTHOPOXVIRUS) PCR
 - **LC Test Code:** 140230
 - **Aultman Test Code:** MONKEY
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** MOZZARELLA CHEESE IGE
 - **LC Test Code:** 202444
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** MPL MUTATION ANALYSIS
 - **LC Test Code:** 489150
 - **Aultman Test Code:** MPL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** MRSA SCREEN PCR
 - **Test ID:** MRSAPCR
 - **Test Includes:** Screening of nares for detection of Methicillin Resistant Staphylococcus aureus.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done Daily 24/7
 - **Specimen Collection /** Insert swabs into nare, parallel to the nasal septum, gently roll 5 times along anterior nare. Slowly
 - **Transfer Instructions:** withdraw swabs and repeat them in other nare with same swabs. Carefully place swabs into the transport tube and close cap.
 - **Specimen Type:** Nares

Aultman Hospital Laboratory Test Directory

- **Volume:** 1 culturette, 2 swabs
- **Container:** 1 culturette, 2 swabs
- **Specimen Handling:** Deliver to Microbiology within 2 hours of collection for inpatients. Refrigerate.

-
- **Test Name:** MTHFR
 - **LC Test Code:** 511238
 - **Aultman Test Code:** MTHFRM
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** MUMPS ANTIBODY
- **Test ID:** MUMP
- **Test Includes:** Mumps IgG. Does not include mumps IgM. Order IgM - MUMPM
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Results available within 5 days
- **Specimen Type:** Serum
- **Volume:** 1 mL
- **Container:** 7 mL SST.
- **Specimen Handling:** Refrigerate specimen.

-
- **Test Name:** MUMPS ANTIBODIES IGM
 - **LC Test Code:** 160499
 - **Aultman Test Code:** MUMPM
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** MUMPS PCR SWAB
 - **LC Test Code:** 140283
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** MURINE TYPHUS ANTIBODIES IGG
 - **LC Test Code:** 016188
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** MUSK ABS, SERUM
 - **LC Test Code:** 504600
 - **Aultman Test Code:** MUSK
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** MYASTHENIA GRAVIS PROFILE
- **LC Test Code:** 165620
- **Aultman Test Code:** MISC

Aultman Hospital Laboratory Test Directory

- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MYCOPLASMA ANTIBODY TITER
 - **Test ID:** MYCO
 - **Test Includes:** Mycoplasma IgG and IgM.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** IgM done daily M – F. Results available within 2 days. IgG results are available within 5 days.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST

-
- **Test Name:** MYCOPLASMA PNEUMONIA CULTURE
 - **LC Test Code:** 086876
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MYCOPLASMA PNEUMONIA PCR
 - **LC Test Code:** 138420
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MYELIN BASIC PROTIEEN CSF
 - **LC Test Code:** 123377
 - **Aultman Test Code:** MBP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MYOGLOBIN URINE
 - **LC Test Code:** 003079
 - **Aultman Test Code:** MYOU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** MYOGLOBIN, PLASMA
 - **Test ID:** MYOS
 - **Synonyms:** Myohemoglobin
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma
 - **Volume:** 1 mL Refrigerate
 - **Container:** 5 mL green top tube
 - **Specimen Handling:** Centrifuge and transfer plasma to separate container within 2 hours

-
- **Test Name:** MYOMARKER 3 PLUS PROFILE (RDL)
 - **LC Test Code:** 520085

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** MYOSPL
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** 5 NUCLEOTIDASE
 - **LC Test Code:** 001701
 - **Aultman Test Code:** NUC5P
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** NASH FIBROSURE (R) PLUS
 - **LC Test Code:** 550960
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** NEUTROPHIL CYTOPLASMIC ANTIBODY
 - **Test ID:** ANCA
 - **Synonyms:** ANCA
 - **Test Includes:** Cytoplasmic neutrophil Ab, C-ANCA, P-ANCA, semi-quantitative by EIA (ELISA) method
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Refrigerate serum
-

- **Test Name:** NGS JAK2 EXONS 12-15
 - **LC Test Code:** 489501
 - **Aultman Test Code:** JAKNON
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** NICKEL BLOOD
 - **LC Test Code:** 790396
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** NICOTINE AND METABOLINE UR QN
 - **LC Test Code:** 070045
 - **Aultman Test Code:** 17OHC
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** NICOTINE AND METABOLITE, QUANT
- **LC Test Code:** 071255
- **Aultman Test Code:** NICOS

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [Labcorp](#)

-
- **Test Name:** NMO IGG AUTOANTIBODIES
 - **LC Test Code:** 004210
 - **Aultman Test Code:** NMOIFA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** NMR LIPO PROFILE+ LIPIDS
 - **LC Test Code:** 884247
 - **Aultman Test Code:** NMRLPD
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** NOROVIRUS, RT-PCR
 - **LC Test Code:** 138307
 - **Aultman Test Code:** NORPCR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** NT PRO BNP
 - **Test ID:** PBNP
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days per week, 24 hours per day.
 - **Specimen Type:** Plasma (Lithium Heparin)
 - **Volume:** 1 mL
 - **Container:** 4 mL green top (Lithium Heparin)
 - **Specimen Handling:** Separate Plasma within 2 hrs. Plasma stable for 3 days.
 - **Storage:** Refrigerate.

-
- **Test Name:** O215-IGE ALPHA GAL
 - **LC Test Code:** 650001
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** N-TELOPEPTIDE, URINE
 - **LC Test Code:** 141093
 - **Aultman Test Code:** UNTX2
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** O2 SATURATION
 - **Test ID:** O2SAT
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available within 1 hour.
 - **Precollection Instructions:** Collect on ice

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** Whole blood
- **Volume:** 1 mL
- **Container:** Arterial blood gas syringe
- **Specimen Handling:** Return to lab immediately

- **Test Name:** OCCULT BLOOD, FECAL
- **Test ID:** OCC
- **Synonyms:** Hemoccult
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week; 24 hours/day. Results available the same day.
- **Specimen Type:** Feces
- **Volume:** 2 g
- **Container:** Hemoccult slide
- **Specimen Handling:** Keep slide at room temperature, DO NOT REFRIGERATE.

-
- **Test Name:** OCCULT BLOOD, GASTRIC
 - **Test ID:** OCCG
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week; 24 hours/day. Results available the same day.
 - **Specimen Type:** Gastric contents
 - **Volume:** 2 g
 - **Container:** Gast occult slide
 - **Specimen Handling:** Keep slide at room temperature, DO NOT REFRIGERATE.

-
- **Test Name:** OLANZAPINE(ZYPREXA)
 - **LC Test Code:** 811513
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Name:** OLIGOCLONAL BANDING SERUM + CSF
 - **LC Test Code:** 019216
 - **Aultman Test Code:** OBAND
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ONCOLOGY
BCR/ABL1
 - **LC Test Code:** 511520
 - **Aultman Test Code:** BCRFSH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OPIATES CONFIRMATION URINE
 - **LC Test Code:** 737846
 - **Aultman Test Code:** OPICON

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [Labcorp](#)

-
- **Test Name:** ORGANIC ACID ANALYSIS URINE
 - **LC Test Code:** 716720
 - **Aultman Test Code:** ORGU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ORRIS ROOT IGE
 - **LC Test Code:** 823452
 - **Aultman Test Code:** ORRIS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OSMOLALITY, SERUM
 - **Test ID:** OSMOS
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST

-
- **Test Name:** OSMOLALITY, URINE
 - **Test ID:** OSMOU
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Collection /** No preservatives should be used
 - **Specimen Type:** Urine, random
 - **Volume:** 1 mL
 - **Container:** Plastic urine tube

-
- **Test Name:** OSMOLALITY FECAL
 - **LC Test Code:** 120071
 - **Aultman Test Code:** OSMOF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OVA + PARASITE EXAM
 - **LC Test Code:** 008623
 - **Aultman Test Code:** OVAP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OXALATE, SERUM/PLASMA
 - **LC Test Code:** 833603
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** OXALATE, QUANT, 24-HOUR URINE
- **LC Test Code:** 003970
- **Aultman Test Code:** OXU24
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OXCARBAZEPINE (TRILEPTAL), S
 - **LC Test Code:** 716928
 - **Aultman Test Code:** TRILEPTAL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OXIDIZED LDL
 - **LC Test Code:** 123023
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OXYCODONE/OXYMORPHONE CONFIRM
 - **LC Test Code:** 763897
 - **Aultman Test Code:** OXYCOD
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** OXYCODONE/ OXYMORPHONE SCR UR
 - **LC Test Code:** 701866
 - **Aultman Test Code:** UOXYCC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** P53 MUTATION ANALYSIS/SEQUENCE
 - **LC Test Code:** 489590
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PAI-1 GENE POLYMORPHISM
 - **LC Test Code:** 500309
 - **Aultman Test Code:** PAIGEN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PANCREASTATIN
 - **LC Test Code:**
 - **Aultman Test Code:** CSA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PANCREATIC ELASTASE, FECAL
 - **LC Test Code:** 123234
 - **Aultman Test Code:** PANCEF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** PANCREATIC POLYPEPTIDE
- **LC Test Code:** 146704
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PARANEOPLASTIC AB
 - **LC Test Code:** 505575
 - **Aultman Test Code:** PARNEO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PARANEOPLASTIC SPINAL FLUID
 - **LC Test Code:** 505700
 - **Aultman Test Code:** PARCSF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PARASITE ID ARTHROPOD
 - **LC Test Code:** 188664
 - **Aultman Test Code:** PARaid
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** Parietal Cell Antibody
 - **Test ID:** PARIE
 - **Synonyms:** APCA
 - **Test Includes:** Detection of gastric parietal cell antibodies
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M – F. Results are available within 3 days.
 - **Specimen Type:** Serum
 - **Volume:** 1 – 2 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** PARVOVIRUS B19, HUMAN, IGG/IGM
 - **LC Test Code:** 163303
 - **Aultman Test Code:** PAB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PARVOVIRUS B19 PCR
 - **LC Test Code:** 138644
 - **Aultman Test Code:** PARPLS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PCR Respiratory ID Panel with COVID

Aultman Hospital Laboratory Test Directory

- **Test ID:** RESCOVID
 - **Test Includes:** Detection of Adenovirus, Coronavirus (4 subtypes), Human Metapneumovirus, Human Rhinovirus/Enterovirus, Influenza A (4 subtypes), Influenza B, Parainfluenza 1-4, RSV, Bordetella pertussis/parapertussis, Chlamydia pneumoniae, and Mycoplasma pneumoniae
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Available 24/7 --STAT 2 hours
 - **Specimen Type:** NPH swab
 - **Container:** Viral Transport Medium
 - **Storage:** Refrigerate
-

- **Test Name:** PD-L1 BY IHC 22C3 KEYTRUDA
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** PD-L1 (LUNG)
 - **Reference Lab Link:** [Test Menu](#) | [Labcorp Oncology](#)
-

- **Test Name:** PERAMPANEL SERUM/PLASMA
 - **LC Test Code:** 821473
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** PERCH OCEAN IGE
 - **LC Test Code:** 809926
 - **Aultman Test Code:** PERCH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

- **Test Name:** PH (venous)
 - **Test ID:** PHV
 - **Synonyms:** pH
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Collection:** Collect on Ice
 - **Specimen Type:** Whole blood
 - **Volume:** 2 mL
 - **Container:** 5 mL green top tube
 - **Specimen Handling:** Collect on ice. Bring it to lab immediately.
-

- **Test Name:** PH, BODY FLUID
- **Test ID:** PHBF
- **Synonyms:** pH, body fluid

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24hrs/day. Results available on the same day.
- **Specimen Collection:** Do not introduce air into specimen. Collect anaerobically.
- **Specimen Type:** Body fluid
- **Volume:** 1 ml
- **Container:** Sterile container or sterile syringe
- **Specimen Handling:** Refrigerate.

-
- **Test Name:** PH, URINE DIPSTICK
 - **Test ID:** PHUA
 - **Test Includes:** Dipstick for urine pH
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Collection /** Keep on ice or refrigerate
 - **Transfer Instructions:**
 - **Specimen Type:** Random urine
 - **Volume:** 2 mL
 - **Container:** Plastic urine

-
- **Test Name:** PHENOBARB
 - **Test ID:** PHBAR
 - **Test Includes:** Time of last dose, phenobarbital
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** one 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma Lithium Heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial.
 - **Storage:** Refrigerate.

-
- **Test Name:** PHENOSENSE(R)
 - **LC Test Code:** 551800
 - **Aultman Test Code:** HIVPHE
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** PHENYLALANINE QN P
 - **LC Test Code:** 700205
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** PHENYTOIN
 - **Test ID:** PHENY
 - **Test Includes:** Time of last dose, phenytoin

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
- **Specimen Type:** Plasma Lithium Heparin (PST)
- **Volume:** 1 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
- **Specimen Handling:** Centrifuge and transfer serum to plastic vial.
- **Storage:** Refrigerate.

-
- **Test Name:** PHENYTOIN FREE SERUM
 - **LC Test Code:** 070763
 - **Aultman Test Code:** PHENF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PHOSPHATIDYLETHANOL(PETH)
 - **LC Test Code:** 791584
 - **Aultman Test Code:** PETH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PHOSPHORUS, 24 HOUR URINE
 - **Test ID:** PHU24
 - **Synonyms:** Inorganic phosphorus
 - **Test Includes:** Collection time, volume, creatinine, phosphorus
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** If specimen is received by 1000, results available the same day.
 - **Precollection Instructions:** No preservative. See 24-hour Urine Collection Instructions.
 - **Specimen Handling:** Keep specimen on ice during collection.

-
- **Test Name:** PHOSPHORUS, RANDOM URINE
 - **Test ID:** PHUR
 - **Synonyms:** Inorganic phosphorus Urine
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Random urine
 - **Volume:** 2 mL
 - **Container:** Plastic urine

-
- **Test Name:** PHOSPHORUS, SERUM
 - **Test ID:** PHOS
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

Aultman Hospital Laboratory Test Directory

- **Specimen Handling:** Avoid hemolysis. Separate serum from cells within 2 hours.
- **Storage:** Refrigerate.

-
- **Test Name:** PINWORM EXAM
 - **Test ID:** PIN
 - **Test Includes:** Examination of scotch tape prep for pinworms (*Enterobius vermicularis*).
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily 0700 - 2400. Results available in 1 day.
 - **Precollection Instructions:** Patient should not be active. The specimen should be collected prior to washing or using the restroom.
 - **Specimen Type:** Scotch tape preparation of stool.
 - **Volume:** 1 scotch tape prep.
 - **Container:** Slide holder.
 - **Specimen Handling:** Collection:
 - Collect specimen in the early morning.
 - Place a 3-inch strip of CLEAR (do NOT use frosted) scotch tape, sticky side out, over one end of a tongue depressor. Hold in place with thumb and finger.
 - Spread buttocks and press the tongue depressor against anal opening, being careful to cover the area between moist and dry areas.
 - Spread the tape smoothly sticky side down on a microscope slide. Label the slide with the patient's name and date of collection. Submit the slide to Microbiology in a cardboard slide holder.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Room temperature.

-
- **Test Name:** PLATELET ANTIBODY PROFILE
 - **LC Test Code:** 117150
 - **Aultman Test Code:** PLTDET
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PLATELET COUNT
 - **Test ID:** PLT
 - **Synonyms:** Quantitative platelet count
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent: 30-60 minutes, Routine: 6 hours
 - **Specimen Type:** Venous blood (EDTA) or free flowing capillary blood
 - **Volume:** 2 mL EDTA blood
 - **Container:** 2.5 mL lav top tube
 - **Specimen Handling:** A platelet count must be performed within 3 hours of collection.
 - **Storage:** Room temperature, or if > 12 hours refrigerate.

Aultman Hospital Laboratory Test Directory

- **Additional Information:** If unopettes are requested for a platelet count, the Hematology staff will set them up from the EDTA specimen collected.

-
- **Test Name:** PLATELET FUNCTION (Platelet works)
 - **Test ID:** PLTF
 - **Synonyms:** Platelet works
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Specimen should be run within 10 minutes of collection.
 - **Specimen Collection /** Clarify orders that read "Platelet Function test" to distinguish Platelet works for PFA 100. Call Coag
 - **Transfer Instructions:**(ext. 33987) for clarification. To be drawn on the floor, OPD, & ER by trained VP techs or trained personnel in OH only.
 - **Container:** Special collection of tubes in COAG. VP to obtain special collection tubes & butterfly collection needle from COAG.
 - **Specimen Handling:** Specimens from the floor, OP, & ER to be HAND CARRIED ONLY to the Main or Stat labs. Transport specimens immediately without jarring or mixing.

-
- **Test Name:** PLATELET FUNCTION ASSAY
 - **Test ID:** PFA
 - **Test Includes:** Done daily. Results within 1 hour.
 - **Testing Facility:** Aultman Laboratory
 - **Specimen Type:** 2 Blue tops (3.2 % sodium citrate) and 1 Lavender top (EDTA)
 - **Volume:** 2 mL
 - **Container:** 1 EDTA and 3 FULL blue top tubes (MUST be full)
 - **Specimen Handling:** Collect with 19-gauge needle or larger and return to lab immediately for testing. Testing MUST be completed within 4 hours of collection in Aultman Lab; alternatively, specimen may be drawn in Aultman Outpatient Lab.

-
- **Test Name:** PMP22 MLPA DEL/DUP ANALYSIS
 - **LC Test Code:** 620081
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** PNEUMOCOCCAL AB (23 SEROTYPE)
 - **LC Test Code:** 812166
 - **Aultman Test Code:** PNE23
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** PNEUMOCOCCAL IM (14 SEROTYPE)
 - **LC Test Code:** 816767
 - **Aultman Test Code:** PNE14
 - **LabCorp Link:** [Search | Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** PNH PANEL BY FCM
 - **CCF Test Code:** PNHPNL
 - **Aultman Test Code:** PNHPNL
 - **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)
-

- **Test Name:** POC/ TISSUE, MICROARRAY
 - **LC Test Code:** 510110
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** POLIOVIRUS (TYPES 1,3) AB
 - **LC Test Code:** 830764
 - **Aultman Test Code:** POLIO
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** PORPHOBILINOGEN QN 24 HR UR
 - **LC Test Code:** 003103
 - **Aultman Test Code:** UPBGQT
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** PORPHOBILINOGEN, QN, RANDOM UR
 - **LC Test Code:** 003065
 - **Aultman Test Code:** UPBGR
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** PORPHYRINS QN RANDOM U
 - **LC Test Code:** 120980
 - **Aultman Test Code:** PORU
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** PORPHYRINS TOTAL SERUM
 - **LC Test Code:** 823246
 - **Aultman Test Code:** SPORPH
 - **LabCorp Link:** [Search | Labcorp](#)
-

- **Test Name:** POST VAS ANALYSIS
- **Test ID:** PVAS
- **Synonyms:** Post Vasectomy Semen Analysis, Post Vasectomy Sperm Count
- **Test Includes:** Presence or absence of spermatozoa and semi-quantitation if present
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Results available the same day. Done M – F, 7 a.m. – 2 p.m.
- **Specimen Type:** Semen

Aultman Hospital Laboratory Test Directory

- **Volume:** 2 mL
- **Container:** Sterile urine container
- **Specimen Handling:** Specimen should be delivered to lab within one hour of collection. Please call ALS for courier service.

-
- **Test Name:** POTASSIUM, 24 HOUR URINE
 - **Test ID:** KU24
 - **Test Includes:** Collection time, volume, creatinine, potassium
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. If specimen is received by 1000, results are available on the same day.
 - **Precollection Instructions:** Collect on ice with no preservative. See 24-hour Urine Collection Instructions.
 - **Specimen Type:** 24-hour urine
 - **Volume:** Submit entire urine collection to lab
 - **Container:** Obtain urine collection container from lab

-
- **Test Name:** POTASSIUM, BODY FLUID
 - **Test ID:** KBF
 - **Test Includes:** Body fluid type, potassium
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Body fluid
 - **Volume:** 2 – 4 mL

-
- **Test Name:** POTASSIUM, RANDOM URINE
 - **Test ID:** KUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Random urine
 - **Volume:** 2 mL
 - **Container:** Plastic urine tube

-
- **Test Name:** POTASSIUM, SERUM
 - **Test ID:** K
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis. Separate serum from cells within 2 hours of collection.
 - **Storage:** Refrigerate

Aultman Hospital Laboratory Test Directory

- **Test Name:** PREALBUMIN
- **Test ID:** PRALB
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7days/week, 24hours/day. Results available the same day.
- **Specimen Type:** Plasma lithium heparin (PST)
- **Volume:** 2 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** PREGNANCY TEST, SERUM QUALITATIVE
 - **Test ID:** PREGS
 - **Synonyms:** Qualitative hCG
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Serum only
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate.

-
- **Test Name:** PREGNANCY TEST, URINE
 - **Test ID:** PREGU
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Random urine
 - **Volume:** 2 mL
 - **Container:** Plastic urine

-
- **Test Name:** PREGNENOLONE MS
 - **LC Test Code:** 140707
 - **Aultman Test Code:** PREGN
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** PRENATAL BLOOD BANK
 - **Synonyms:** prenatal ABO, prenatal ABS
 - **Specimen type:** Plasma
 - **Volume:** 6ml
 - **Container:** EDTA (pink top)
 - **TAT:** Refer to comment in ABS
 - **Test includes:** ABO/RH & ABS; Blood Bank will reflex additional testing when indicated

-
- **Test Name:** PRENATAL PANEL 1
 - **Test ID:** PNP1+PNPB
 - **Test includes:** Blood Bank will reflex additional testing when indicated
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** See Individual Tests
- **Specimen Type:** Multiple
- **Container:** (1) 5 mL Lav top tube, AND (1) 7 mL SST, AND (1) 6 mL pink top EDTA
- **Specimen Handling:** See Individual Tests
- **Additional information:** Refer to comment in ABO for proper labeling

-
- **Test Name:** PRENATAL PANEL 4
 - **Test ID:** PNP4 + PNPB
 - **Test includes:** Blood Bank will reflex additional testing when indicated
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** See Individual Tests
 - **Specimen Type:** Multiple
 - **Container:** (1) 5 mL Lav top tube, AND (1) 7 mL SST, AND (1) 6 mL pink top EDTA
 - **Specimen Handling:** See Individual Tests
 - **Additional information:** Refer to comment in ABO for proper labeling

-
- **Test Name:** PRENATAL PANEL 7
 - **Test ID:** PNP7 + PNBB
 - **Test Includes:** Blood Bank will reflex additional testing when indicated
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** See individual tests
 - **Specimen Type:** Multiple
 - **Container:** (1) 5 ml Lav Top tube, AND (1) 7ml SST tube, AND (1) 6 ml pink top EDTA tube
 - **Additional information:** Refer to comment in ABO for proper labeling

-
- **Test Name:** PRENATAL PANEL 8
 - **Test ID:** YPNP8 & PNPB
 - **Test Includes:** Blood Bank will reflex additional testing when indicated
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** See Individual Tests
 - **Specimen Type:** Multiple
 - **Container:** (1) 5 mL Lav top tube, AND (1) 7 mL SST, AND (1) 6 mL pink top EDTA
 - **Specimen Handling:** See Individual Tests
 - **Additional information:** Refer to comment in ABO for proper labeling

-
- **Test Name:** PRIMIDONE (MYSOLINE(R)) SERUM
 - **LC Test Code:** 007856
 - **Aultman Test Code:** PRIM
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** PROCALCITONIN
 - **LC Test Code:** 164750

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** PROCAL
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PRODUCTS OF CONCEPTION
 - **Testing Facility:** Aultman Laboratory
 - **Precollection Instructions:** Use Tissue Examination Request (Form 23A). If the surgeon is different than the requested physician, write the surgeon's name on the request. Include clinical diagnosis.
 - **Specimen Collection:** Take specimen to Histology Department. After Histology is closed, take specimens to Microbiology.
 - **Transfer Instructions:**
 - **Container:** If specimen is for (Chromosome Analysis) the specimen should be sent fresh or with saline. Do not place specimen in formalin. Formalin will destroy the specimen sent for chromosome analysis.
 - Only if specimen is for Routine Histology, the specimen should be sent in 10% Neutral Buffered Formalin (NBF) biopsy bottles.
 - **Specimen Handling:** Store specimen at room temperature.
 - **Storage:** Refrigerate.

-
- **Test Name:** PROINSULIN
 - **LC Test Code:** 140533
 - **Aultman Test Code:** IPROIN
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** PROLACTIN
 - **Test ID:** PROL
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate.

-
- **Test Name:** PROLACTIN MACROADENOMA
 - **LC Test Code:** 004500
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PROPEPTIDE TYPE I COLLAGEN
 - **LC Test Code:** 140850
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PROPOXYPHENE SERUM
 - **LC Test Code:** 084756

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PROTAGLANDIN D2 SERUM
 - **LC Test Code:** 505530
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PROSTATIC ACID PHOS SERUM
 - **LC Test Code:** 004747
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PROTEIN C ACTIVITY
 - **Test ID:** PROTC
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Test is batched and run once per week (Tuesdays)
 - **Specimen Type:** Plasma (citrated)
 - **Volume:** 2 mL
 - **Container:** (2) full blue top tube – must be FULL
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, the patient may be drawn in the Aultman Outpatient Lab

-
- **Test Name:** PROTEIN ELECTROPHORESIS, SERUM
 - **Test ID:** SPE
 - **Synonyms:** Serum protein fractionation
 - **Test Includes:** Total protein, albumin, alpha 1, alpha 2, beta and gamma fractions, interpretation by a pathologist
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M, W, F. Results available the same day.
 - **Precollection Instructions:** Fasting specimen is preferred
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis and lipemia

-
- **Test Name:** PROTEIN ELECTROPHORESIS, URINE
 - **Test ID:** UPE
 - **Test Includes:** Pattern interpretation by a pathologist
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M, W, F. Results available the same day.

Aultman Hospital Laboratory Test Directory

- **Precollection Instructions:** See 24-hour Urine Collection Instructions. Note: A first morning void specimen can be used instead of a 24-hour collection. Random urines other than the first morning void are discouraged but can be done if ordered by the physician.
- **Specimen Type:** 24-hour urine or random urine
- **Volume:** Submit entire urine collection to lab. Random requires at least 50 mL (30 mL minimum).
- **Container:** Obtain urine collection container from lab

-
- **Test Name:** PROTEIN S FREE ANTIGEN
 - **Test ID:** PRSFA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Batched, run once per week (Tuesday)
 - **Specimen Type:** Plasma (citrated)
 - **Volume:** 2 mL
 - **Container:** (2) full blue top tube – must be FULL
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, the patient may be drawn in the Aultman Outpatient Lab.

-
- **Test Name:** PROTEIN, 24 HOUR URINE
 - **Test ID:** PRU24
 - **Test Includes:** Collection time, volume, creatinine, protein
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** If specimen is received by 1000, results are available the same day
 - **Precollection Instructions:** Collect on ice with no preservatives. See 24-hour Urine Collection Instructions.
 - **Specimen Type:** 24-hour urine
 - **Volume:** Submit entire urine collection to lab
 - **Container:** Obtain urine collection container from lab

-
- **Test Name:** PROTEIN, BODY FLUID
 - **Test ID:** PROTEIN, BODY FLUID
 - **Test Includes:** Body fluid type, protein
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Body fluid
 - **Volume:** 2 – 4 mL

-
- **Test Name:** PROTIEN ELECTRO + INTERP CSF
 - **LC Test Code:** 261503
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PROTEIN, RANDOM URINE

Aultman Hospital Laboratory Test Directory

- **Test ID:** PRUR
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
- **Specimen Type:** Random urine
- **Volume:** 2 mL
- **Container:** Plastic urine tube

-
- **Test Name:** PROTEIN, TOTAL, SERUM
 - **Test ID:** PROT
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis.

-
- **Test Name:** PROTHROMBIN TIME
 - **Test ID:** PRO
 - **Synonyms:** Protime, PT
 - **Test Includes:** Protime, INR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** STAT/urgent: 45 minutes, Routine: 3 hours
 - **Specimen Type:** Plasma (citratd)
 - **Volume:** 2 mL
 - **Container:** One full blue top tube (Must be full)
 - **Specimen Handling:** Store specimen in original tube. Specimen is stable within 24 hours of collection in original capped tube. Alternatively, specimen may be drawn in Aultman Outpatient Lab.

-
- **Test Name:** PROTIME
 - **Test ID:** 50:50 MIX
 - **Synonyms:** Protime, PT
 - **Test Includes:** Protime, INR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** STAT/urgent: 45 minutes, Routine: 3 hours
 - **Specimen Type:** Plasma (citratd)
 - **Volume:** 2 mL
 - **Container:** 5 mL blue top tube (MUST be full)
 - **Specimen Handling:** Store specimen in original tube. Specimen is stable within 24 hours of collection in original capped tube. Alternatively, specimen may be drawn in Aultman Outpatient Lab.

-
- **Test Name:** PSA
 - **Test ID:** PSA

Aultman Hospital Laboratory Test Directory

- **Test Includes:** Done daily. Results available the same day.
- **Testing Facility:** Aultman Laboratory
- **Specimen Type:** Serum only
- **Volume:** 2 mL
- **Container:** 7 mL SST
- **Specimen Handling:** Avoid hemolysis
- **Storage:** Refrigerate.

-
- **Test Name:** PSA TOTAL % FREE
 - **LC Test Code:** 480947
 - **Aultman Test Code:** PSAF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PSA, ULTRASENSITIVE W/O SERIAL
 - **LC Test Code:** 140731
 - **Aultman Test Code:** PSAU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** P-TAU181
 - **LC Test Code:** 483745
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** P-TAU217
 - **LC Test Code:** 484390
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** PTH, INTACT
 - **Test ID:** PTH
 - **Test Includes:** PTH, intact
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 3.5 mL or 5 mL SST. 4 mL Plain Red Top is also acceptable.

-
- **Test Name:** PTHRP (PTH-RELATED PEPTIDE)
 - **LC Test Code:** 503380
 - **Aultman Test Code:** PTHRP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** Q FEVER IGG/IGM ANTIBODY SCRIN

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 806606
- **Aultman Test Code:** QFFEVER
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** QFT-TB PLUS (Client Incubated)
 - **LC Test Code:** 182893
 - **Aultman Test Code:** QUANTTB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** RABIES NEUT ABS TITRAT
 - **LC Test Code:** 164988
 - **Aultman Test Code:** RABIES
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** RATIO PROT/CREAT URINE RANDOM
 - **Test ID:** RPCUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 2 hours.
 - **Specimen Type:** Urine random
 - **Volume:** 10 mL
 - **Container:** Urine

-
- **Test Name:** RBC, URINE
 - **Test ID:** RBCUA
 - **Synonyms:** Urinary RBC, urinary red blood cells.
 - **Test Includes:** Microscopic examination of urine specimen for red blood cells
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Precollection Instructions:** Keep on ice or refrigerate
 - **Specimen Type:** Random urine
 - **Volume:** 2 mL
 - **Container:** Plastic urine

-
- **Test Name:** SPOTTED FEVER GROUP ANTIBODIES
 - **LC Test Code:** 016502
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** STONE ANALYSIS
 - **LC Test Code:** 910180
 - **Aultman Test Code:** CSA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** SULFONYLUREA SCREEN QT

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 814133
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** RECOMBX(R) CV2 AUTOANTIBODY
 - **LC Test Code:** 823616
 - **Aultman Test Code:** CV2
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** RENAL FUNCTION PANEL
 - **Test ID:** RFP
 - **Test Includes:** Glu, Na, K, Cl, CO2, lyte balance, BUN, creat, b/c ratio, calcium, albumin, phosphorus
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** plasma lithium heparin (PST)
 - **Volume:** 2 – 4 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis.
 - **Storage:** Refrigerate.

-
- **Test Name:** RENIN ACTIVITY, PLASMA
 - **LC Test Code:** 002006
 - **Aultman Test Code:** RENIN
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** RETICULOCYTES
 - **Test ID:** RETIC
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 60 minutes
 - **Specimen Type:** Whole blood (EDTA)
 - **Volume:** 2 mL
 - **Container:** 2.5 mL lav top tube
 - **Storage:** Room temperature, or if > 12 hours refrigerate.

-
- **Test Name:** RETICULOCYTE HEMOGLOBIN PANEL
 - **LC Test Code:** 824119
 - **Aultman Test Code:** RTIRHB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** REVERSE T3, SERUM
 - **LC Test Code:** 070104
 - **Aultman Test Code:** REVT3

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search | Labcorp](#)

-
- **Test Name:** RF ISOTYPES IGG, IGA, IGM EIA
 - **LC Test Code:** 520178
 - **Aultman Test Code:** RHEUMA
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** RHEUMATOID FACTOR, SERUM
 - **Test ID:** RF
 - **Synonyms:** RA, Rheumatoid factor IgM
 - **Test Includes:** Enzyme Immunoassay (EIA) test for rheumatoid factor IgM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** RHOGAM WORKUP
 - **Specimen type:** Plasma
 - **Volume:** 6ml
 - **Container:** EDTA (pink top)
 - **TAT:** Refer to comment in ABS
 - **Test includes:** ABO/RH, ABS, FMH. Blood Bank will reflex additional testing when indicated

-
- **Test Name:** RIBOSOMAL P ANTIBODY
 - **Test ID:** RIBAB
 - **Synonyms:** Ribosomal Antibody
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Results available within 5 days
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Specimen Handling:** Refrigerate specimen

-
- **Test Name:** RISPERIDONE (RISPERDAL(R)) S
 - **LC Test Code:** 716563
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** RNA, REAL TIME PCR (NON-GRAPH)
 - **LC Test Code:** 550430

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** HIVLD
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** RNP ANTIBODIES
 - **LC Test Code:** 016353
 - **Aultman Test Code:** RNPAB
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** ROTAVIRUS SCREEN
 - **Test ID:** ROTA
 - **Test Includes:** Rapid membrane immunoassay.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Stool.
 - **Volume:** 1 gram.
 - **Container:** Clean container with a tight-fitting lid.
 - **Specimen Handling:** Collection: Avoid contamination with water, urine, paper or mineral oil. Specimens cannot be removed from the toilet bowl. Diapers are not accepted.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate.

-
- **Test Name:** RPR
 - **Test ID:** RPR
 - **Test Includes:** Screening procedure for the detection of syphilis. RPR titter and FTA are done on all reactive RPR tests.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M – F. Results available in 1 day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST

-
- **Test Name:** RUBELLA ANTIBODIES IGM
 - **LC Test Code:** 096537
 - **Aultman Test Code:** RUBM
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** RUBELLA IMMUNE STATUS
 - **Test ID:** RUBIS
 - **Synonyms:** German measles immune status
 - **Test Includes:** Immune status assay for the detection of IgG antibodies for German measles. This test IS NOT for determining acute (current) or congenital infection. For that diagnosis, use the RUBM test code.
 - **Testing Facility:** Aultman Laboratory

Aultman Hospital Laboratory Test Directory

- **Turnaround Time:** Done M – F. Results available the next day.
 - **Specimen Type:** Serum- 7ml SST
 - **Volume:** 1 mL
-
- **Test Name:** RUBEOLA IGG ANTIBODY
 - **Test ID:** RUBEO
 - **Synonyms:** Measles
 - **Test Includes:** Detects rubeola IgG only
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Results available within 5 days.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
-
- **Test Name:** SACCHAROMYCES CEREVISIAE PANEL
 - **LC Test Code:** 164657
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)
-
- **Test Name:** SAFFLOWER IGE
 - **LC Test Code:** 903686
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)
-
- **Test Name:** SALICYLATE
 - **Test ID:** SAL
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1mL
 - **Container:** SST/Plain Red Top
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial.
 - **Storage:** Refrigerate.
-
- **Test Name:** SALIVARY CORTISOL, MS
 - **LC Test Code:** 500179
 - **Aultman Test Code:** SALCT
 - **LabCorp Link:** [Search | Labcorp](#)
-
- **Test Name:** SARS-COV-2 ANTIBODY IGG
 - **LC Test Code:** 164055
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

Aultman Hospital Laboratory Test Directory

- **Test Name:** SARS-COV-2 AB NUCLEOCAPSID
- **LC Test Code:** 164068
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** SARS-COV-2 SEMI-QUANT SPIKE AB
- **LC Test Code:** 164090
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** SCL-70 ANTIBODY
- **Test ID:** SCL70
- **Synonyms:** Autoantibodies to SCL-70 antigen, Scleroderma Ab
- **Testing Facility:** Reference Laboratory
- **Specimen Type:** Serum
- **Volume:** 1 mL
- **Container:** 7 mL SST
- **Specimen Handling:** Refrigerate specimen

- **Test Name:** SELENIUM SERUM PLASMA
- **LC Test Code:** 716910
- **Aultman Test Code:** PSELEN
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** SEROTONIN SERUM
- **LC Test Code:** 120204
- **Aultman Test Code:** SEROT
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** SERTRALINE(ZOLOFT) S/P
- **LC Test Code:** 808494
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** SEX HORMONE BINDING GLOB, SERUM
- **LC Test Code:** 082016
- **Aultman Test Code:** SHBG2
- **LabCorp Link:** [Search | Labcorp](#)

- **Test Name:** SHIGA TOXINS 1 & 2
- **Test ID:** SHIGA
- **Test Includes:** Shiga toxins 1 and 2. Included with CST/CSTY

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M– F, 0700 – 2400. Results are available in 3 days.
 - **Specimen Type:** Stool.
 - **Volume:** 1 gram or at least visible material on 1 culturette (2 swabs).
 - **Container:** Clean container with a tight-fitting lid or 1 culturette (2 swabs).
 - **Specimen Handling:**
 - Stool
 - The entire contents of the first specimen of the day is preferred.
 - Avoid contamination with water, urine, paper, barium or mineral oil. Do not remove specimens from the toilet bowl.
 - Specimens received in a diaper are not acceptable and will be rejected by Microbiology.
 - **Transport:** Deliver to Microbiology within 2 hours of collection for inpatients.
 - **Storage:** Refrigerate.
-
- **Test Name:** SICKLE CELL
 - **Test ID:** SC
 - **Test Includes:** This test is done for screening purposes. It is a solubility test that differentiates between sickling and non-sickling hemoglobins but does not identify the hemoglobin type. **Cannot be performed on children < 6 months of age. **
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 1-3 days
 - **Specimen Type:** Whole blood (EDTA)
 - **Volume:** 1 mL
 - **Container:** 5 mL lav top tube
 - **Storage:** Refrigerate.
-
- **Test Name:** SIROLIMUS (RAPAMUNE) BLOOD
 - **LC Test Code:** 716712
 - **Aultman Test Code:** RAPA
 - **LabCorp Link:** [Search | Labcorp](#)
-
- **Test Name:** SMITH ANTIBODIES
 - **LC Test Code:** 016360
 - **Aultman Test Code:** SMABS
 - **LabCorp Link:** [Search | Labcorp](#)
-
- **Test Name:** SMOOTH MUSCLE ANTIBODY
 - **Test ID:** SMUSC
 - **Synonyms:** ASMA
 - **Test Includes:** Smooth muscle antibody screen with titer if positive
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done M – F. Results are available in 3 days.

Aultman Hospital Laboratory Test Directory

- **Specimen Type:** Serum
- **Volume:** 1 – 2 mL
- **Container:** 7 mL SST
- **Specimen Handling:** Avoid hemolysis

-
- **Test Name:** SODIUM, 24 HOUR URINE
 - **Test ID:** NAU24
 - **Test Includes:** Collection time, volume, creatinine, sodium
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Specimen received by 1000, results available the same day
 - **Precollection Instructions:** No preservative. See 24-hour Urine Collection Instructions.
 - **Specimen Type:** 24-hour urine
 - **Volume:** Submit entire urine collection to lab
 - **Container:** Obtain urine collection container from lab.
 - **Specimen Handling:** Keep specimen on ice during collection.

-
- **Test Name:** SODIUM, BODY FLUID
 - **Test ID:** NABF
 - **Test Includes:** Body fluid type, sodium
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** one 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Body fluid
 - **Volume:** 2 – 4 mL

-
- **Test Name:** SODIUM, RANDOM URINE
 - **Test ID:** NAUR
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Random urine
 - **Volume:** 4 mL
 - **Container:** Plastic urine tube

-
- **Test Name:** SODIUM, SERUM
 - **Test ID:** NA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma Lithium Heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate

Aultman Hospital Laboratory Test Directory

- **Test Name:** SOLUBLE LIVER AG (IGG AB)
- **LC Test Code:** 007441
- **Aultman Test Code:** SOLIV
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** SOLUBLE TRANSFERRIN RECEPTOR
- **LC Test Code:** 143305
- **Aultman Test Code:** SOLTRANS
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** SOMATOSTATIN
- **LC Test Code:** 812420
- **Aultman Test Code:** MISC
- **LabCorp Link:** [Search](#) | [Labcorp](#)

- **Test Name:** SPECIFIC GRAVITY, BODY FLUID
- **Test ID:** SGBF
- **Test Includes:** Body fluid type, volume, specific gravity
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily. Results available by 1600
- **Specimen Type:** Body fluid
- **Volume:** 2.2 mL minimum

-
- **Test Name:** SPECIFIC GRAVITY, URINE
 - **Test ID:** SGUA
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Collection:** Keep on ice or refrigerate
 - **Specimen Type:** Random urine
 - **Volume:** 2 mL
 - **Container:** Plastic urine tube

-
- **Test Name:** SPINAL FLUID FOR CYTOLOGY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm, 2 working days. Certain non-Gyn cases will require additional stains and will take longer.
 - **Specimen Collection:** Use Form 308A, (Cytology Specimens). Be sure to write the date and time of collection and the clinical diagnosis.
 - **Specimen Type:** Cerebrospinal fluid
 - **Volume:** Minimum 1 mL fluid.
 - **Container:** Sterile CSF plastic tube with cap.

Aultman Hospital Laboratory Test Directory

- **Specimen Handling:** Deliver to the Laboratory immediately. If Microbiology testing is also ordered, specimen is to be delivered to Microbiology first. After hours, deliver specimen to the Microbiology department. Specimen Handling: Refrigerate.
-
- **Test Name:** SPUTUM FOR CYTOLOGY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm, Saturday received by 11:00am. 2 working days. Certain non-Gyn cases will require additional stains and will take longer.
 - **Specimen Collection:** Use Form 308A, (Cytology Specimens). Mark the source of the specimen on the form and include any
 - **Transfer Instructions:** pertinent clinical information.
 - **Specimen Type:** Early morning specimen is preferred.
 - **Container:** Collect in a clean container. If the specimen is also for culture, container must be sterile.
 - **Specimen Handling:** Deliver to the Cytology department. If Microbiology testing is also ordered, specimen is to be delivered to Microbiology first. After hours, deliver specimen to the Microbiology department. Specimen Handling: Refrigerate
-
- **Test Name:** SJOGREN'S AB, ANTI-SS-A/-SS-B
 - **LC Test Code:** 012708
 - **Aultman Test Code:** SSAB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** STRATIFY JCV(TM) AB W/INDEX
 - **LC Test Code:** 819372
 - **Aultman Test Code:** JCVIDX
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** STRIATION ABS SERUM
 - **LC Test Code:** 160184
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-
- **Test Name:** STREP A PCR SCREEN
 - **Test ID:** STREPA
 - **Testing Facility:** Aultman Hospital
 - **Test Includes:** Rapid PCR test for the qualitative detection of Group A streptococcus from throat swab specimens without the need for culture confirmation of negative results.
 - **Specimen Type:** E-Swab collected from throat
 - **Turnaround Time:** 1 hour. Offered 24 hours a day
-
- **Test Name:** STREP PNEUMONIAE URINE ANTIGEN
 - **Test ID:** SPAG

Aultman Hospital Laboratory Test Directory

- **Test Includes:** Rapid immunochromatographic assay for the detection of Streptococcus pneumoniae antigen in urine.
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done Mon – Fri, 24 hours a day, Sat and Sun 0700 – 2400. Performed Sat and Sun 0000-0700 Stat requests only. Results available the same day.
- **Specimen Type:** Urine (catheter or clean void).
- **Volume:** 2 mL
- **Container:** Sterile Container
- **Specimen Handling:** Collection:
 - Catheter: Swab catheter port with povidone iodine. Puncture the port with needle and aspirate urine into a syringe. Do not collect urine from a drainage bag.
 - Clean Void: Cleanse urinary meatus with towelettes. Have the patient void a small amount for discard. Collect a midstream urine specimen.

-
- **Test Name:** STRONGYLOIDES IGG ANTIBODY
 - **LC Test Code:** 164000
 - **Aultman Test Code:** STRSER
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** T001-IGE MAPLE/BOX ELDER
 - **LC Test Code:** 602489
 - **Aultman Test Code:** BELDR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** T003-IGE COMMON SILVER BIRCH
 - **LC Test Code:** 602936
 - **Aultman Test Code:** BIRCHT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** T006-IGE CEDAR MOUNTAIN
 - **LC Test Code:** 602491
 - **Aultman Test Code:** MTCED
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** T011-IGE MAPLE LEAF SYCAMORE
 - **LC Test Code:** 602948
 - **Aultman Test Code:** SYCMOR
 - **LabCorp Link:** [Search](#) | [LabCorp](#)

-
- **Test Name:** T007-IGE OAK WHITE
 - **LC Test Code:** 602480
 - **Aultman Test Code:** OAK

Aultman Hospital Laboratory Test Directory

- LabCorp Link: [Search](#) | [LabCorp](#)
-

- Test Name: T008-IGE ELM AMERICAN
- LC Test Code: 602476
- Aultman Test Code: ELM
- LabCorp Link: [Search](#) | [Labcorp](#)

- Test Name: T010-IGE WALNUT
 - LC Test Code: 602494
 - Aultman Test Code: WNUTTR
 - LabCorp Link: [Search](#) | [Labcorp](#)
-

- Test Name: T014-IGE COTONWOOD
 - LC Test Code: 602518
 - Aultman Test Code: COTTW
 - LabCorp Link: [Search](#) | [Labcorp](#)
-

- Test Name: T015- IGE ASH WHITE
 - LC Test Code: 602927
 - Aultman Test Code: WHTASH
 - LabCorp Link: [Search](#) | [Labcorp](#)
-

- Test Name: T016-IGE PINE, WHITE
 - LC Test Code: 602928
 - Aultman Test Code: WTPINE
 - LabCorp Link: [Search](#) | [LabCorp](#)
-

- Test Name: T022-IGE PECAN HICKORY
 - LC Test Code: 602533
 - Aultman Test Code: HIKPEC
 - LabCorp Link: [Search](#) | [Labcorp](#)
-

- Test Name: T041-IGE HICKORY WHITE
 - LC Test Code: 602941
 - Aultman Test Code: MISC
 - LabCorp Link: [Search](#) | [Labcorp](#)
-

- Test Name: T057-IGE CEDAR RED
 - LC Test Code: 605616
 - Aultman Test Code: RECEDR
 - LabCorp Link: [Search](#) | [Labcorp](#)
-

- Test Name: T070-IGE WHITE MULBERRY

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 602569
- **Aultman Test Code:** MULBRY
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** T-CELL GENE REARRANGEMENT BY PCR
 - **CCF Test Code:** TGAMMA
 - **Aultman Test Code:** TGAMMA
 - **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** T-CELL RECEPTOR GENE REARRANGEMENTS PROFILE γ AND δ
 - **Reference Lab:** LAB CORP
 - **Aultman Test Name-Pathology:** T-CELL GENE REARRANGEMENT (TRG/TRB)
 - **Reference Lab Link:** <https://www.labcorp.com/test-menu/>

-
- **Test Name:** TP53 SOMATIC MUTATION PROGNOSTIC
 - **CCF Test Code:** TP53MU
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Test Directory - Online Test Catalog | Cleveland Clinic Laboratories](#)

-
- **Test Name:** TRAP
 - **LC Test Code:** 451840
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** T UPTAKE
 - **Test ID:** TUP
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate.

-
- **Test Name:** T3, FREE
 - **Test ID:** FT3
 - **Synonyms:** Unbound triiodothyronine
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** 24 hours/day, 7 days/week
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate.
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** T4, FREE
- **Test ID:** FT4
- **Synonyms:** Free thyroxine
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7days/week, 24hours/day. Results available on the same day.
- **Specimen Type:** Serum or Plasma Lithium Heparin – Separate from cells within 2 hours of collection.
- **Volume:** 1 mL
- **Container:** SST or Plasma Lithium Heparin
- **Storage:** Refrigerate.

-
- **Test Name:** T4, TOTAL
 - **Test ID:** T4
 - **Synonyms:** T4
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST
 - **Storage:** Refrigerate.

-
- **Test Name:** TACROLIMUS BY IMMUNOASSAY
 - **LC Test Code:** 711034
 - **Aultman Test Code:** TACRO
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** THYROXINE BINDING GLOBULIN
 - **LC Test Code:** 001735
 - **Aultman Test Code:** TBG
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TICK- BORNE DISEASE AB PROFILE
 - **LC Test Code:** 164705
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TESTOSTERONE, TOTAL
 - **Test ID:** TESTO
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day. Specimen Type Serum or Plasma Lithium Heparin – Separate from cells within 2 hours of collection.
 - **Specimen Type:** Plasma Lithium Heparin (PST)
 - **Volume:** 1 mL.
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

Aultman Hospital Laboratory Test Directory

- **Storage:** Refrigerate.

-
- **Test Name:** TESTOSTERONE, TOTAL AND FREE
 - **LC Test Code:** 140103
 - **Aultman Test Code:** TFTEST
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TETANUS ANTITOXOID IGG AB
 - **LC Test Code:** 163691
 - **Aultman Test Code:** TETAN
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** THEOPHYLLINE
 - **Test ID:** THEO
 - **Test Includes:** Time of last dose, theophylline
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial. Avoid hemolysis
 - **Storage:** Refrigerate.

-
- **Test Name:** THIOPURINE METABOLITES
 - **LC Test Code:** 503800
 - **Aultman Test Code:** THIOPM
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** THIOPURINE METHYLTRANSFERASE
 - **LC Test Code:** 510750
 - **Aultman Test Code:** TPMTNON
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** THROMBIN TIME
 - **Test ID:** TT
 - **Test Includes:** Heparin dose, thrombin time
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Urgent/STAT: 45 minutes, Routine
 - **Specimen Type:** Plasma (citratd)
 - **Volume:** 5 mL
 - **Container:** 5 mL blue top tube (MUST be full)

Aultman Hospital Laboratory Test Directory

- **Specimen Handling:** Specimen must be tested within 4 hours of collection in Aultman Lab; alternatively, may be drawn in Aultman Outpatient Lab.

-
- **Test Name:** TGAB+THYROGLOBULIN, IMA OR LCMS
 - **LC Test Code:** 042045
 - **Aultman Test Code:** THYRORF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** THYROID ANTIBODIES
 - **Test ID:** THYAB
 - **Test Includes:** Thyroglobulin antibodies, Thyroid Peroxidase Antibodies (Microsomal). NOTE: Thyroid Peroxidase
 - Antibodies (Microsomal) can be ordered individually – see aTPO
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24 hours/day. Results available on the same day.
 - **Specimen Type:** Serum
 - **Volume:** 2 mL
 - **Container:** SST

-
- **Test Name:** THYROID CASCADE PROFILE
 - **LC Test Code:** 330015
 - **Aultman Test Code:** THYCAS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** THYROID STIM IMMUNOGLOBULIN
 - **LC Test Code:** 140749
 - **Aultman Test Code:** TSI
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** THYROID STIMULATING HORMONE
 - **Test ID:** TSH
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 MI
 - **Container:** SST or plasma lithium heparin.
 - **Specimen Handling:** Avoid gross hemolysis
 - **Storage:** Refrigerate.

-
- **Test Name:** THYROTROPIN RECEPTOR AB SERUM
 - **LC Test Code:** 010314

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** TRAB
- **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** T-TRANSGLUTAMINASE (TTG) IGG
 - **LC Test Code:** 164988
 - **Aultman Test Code:** TTGIGG
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TITANIUM WB
 - **LC Test Code:** 792620
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TMAO (TRIMETHYLAMINE N-OXIDE)
 - **LC Test Code:** 123413
 - **Aultman Test Code:** TMAO
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TOBRAMYCIN
 - **Test Includes:** Time of last dose, Tobramycin
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Test performed 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Storage:** Refrigerate.

-
- **Test Name:** TOPIRAMATE (TOPOMAX) SERUM
 - **LC Test Code:** 716285
 - **Aultman Test Code:** TOP
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TORCH CONGENITAL DISEASE
 - **Test ID:** TORCH
 - **Synonyms:** This test is to be ordered only on neonates or cord blood specimens. For others, order individual tests.
 - **Test Includes:** Toxoplasma IgG and IgM, CMV IgG and IgM, Rubella IgM and quantitative IgM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly M - F. Results are available within 3 days.
 - **Precollection Instructions:** Note if specimen is cord blood
 - **Specimen Type:** Serum
 - **Volume:** 1 mL

Aultman Hospital Laboratory Test Directory

- **Container:** 7 mL SST

-
- **Test Name:** TOTAL GLUTATHIONE
 - **LC Test Code:** 007700
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TOXASSURE FLEX 15, UR
 - **LC Test Code:** 912334
 - **Aultman Test Code:** TOXFLE15
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TOX ASSURE FLEX 23 UR
 - **LC Test Code:** 912335
 - **Aultman Test Code:** TOXFLE23
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TOX ASSURE FLEX 23 URINE
 - **LC Test Code:** 912344
 - **Aultman Test Code:** TOXFLE23
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TOXOPLASMOSIS ANTIBODY TITER
 - **Test ID:** TOXO
 - **Synonyms:** Toxoplasma titer
 - **Test Includes:** Toxoplasma IgG and IgM
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly M – F. Results are available within 4 days.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST

-
- **Test Name:** TOXOPLASMA GONDII AB, IGG
 - **LC Test Code:** 006478
 - **Aultman Test Code:** TOXG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TOXOPLASMA GONDII AB, IGM
 - **LC Test Code:** 096651
 - **Aultman Test Code:** TOXM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TOXOPLASMA GONDII PCR
 - **LC Test Code:** 138602

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** TXPCR
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TPMT AND NUDT15 GENOTYPING
 - **LC Test Code:** 512300
 - **Aultman Test Code:** TPMGEN
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TRAMADOL& METABOLITE QUANT
 - **LC Test Code:** 071545
 - **Aultman Test Code:** TRAQNT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TRAMADOL + METABOLITES URINE
 - **LC Test Code:** 804100
 - **Aultman Test Code:** TRANQNTU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TRANSFERRIN
 - **Test ID:** TRF
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** TREPONEMA PALLIDUM ANTIBODIES
 - **LC Test Code:** 0823370
 - **Aultman Test Code:** TPAG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TRICH VAG BY NAA
 - **LC Test Code:** 188052
 - **Aultman Test Code:** TRVAMP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** TRIGLYCERIDES
 - **Test ID:** TRIG
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis

Aultman Hospital Laboratory Test Directory

- **Storage:** Refrigerate.

-
- **Test Name:** Troponin I Highly Sensitive
 - **Testing Facility:** Aultman Hospital
 - **Turnaround Time:** Done 7days/week, 24hours/day. Results available within 2 hours of receipt.
 - **Specimen Type:** Plasma Lithium Heparin
 - **Volume:** 1 ml
 - **Specimen Handling:** Separate plasma from cells within 2 hours of collection and store in refrigerator. Freeze if not performed within 24 hours.

-
- **Test Name:** TROPONIN T
 - **Test ID:** TNT
 - **Testing Facility:** Reference Laboratory
 - **Specimen Handling:** 1 ML SERUM - REF

-
- **Test Name:** TRYPTASE
 - **LC Test Code:** 004280
 - **Aultman Test Code:** TRYPT
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TUMOR NECROSIS FACTOR ALPHA
 - **LC Test Code:** 140673
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** TYPE AND SCREEN
 - **Test ID:** TS
 - **Specimen type:** Plasma
 - **Volume:** 6ml
 - **Container:** EDTA (pink top)
 - **TAT:** Refer to comments in ABS
 - **Test includes** ABO/RH & ABS. Blood Bank will reflex additional testing when indicated

-
- **Test Name:** UREA NITROGEN, 24 HOUR URINE
 - **Test ID:** UNU24
 - **Synonyms:** Urine BUN, urea, UUN
 - **Test Includes:** Collection time, volume, creatinine, urea nitrogen
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. If specimen received by 1000, results are available the same day.
 - **Precollection Instructions:** No preservative. See 24-hour Urine Collection Instructions.
 - **Specimen Type:** 24-hour urine
 - **Volume:** Submit entire urine collection to lab

Aultman Hospital Laboratory Test Directory

- **Container:** Obtain urine collection container from lab
- **Specimen Handling:** Keep specimen on ice during collection.

-
- **Test Name:** UREA NITROGEN, RANDOM URINE
 - **Test ID:** UNUR
 - **Synonyms:** Urine BUN, urea, UUN
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Random urine
 - **Volume:** 2 MI
 - **Container:** Plastic urine tube

- **Test Name:** UREAPLASMA/MYCOPLASMA HOMINIS
- **LC Test Code:** 086884
- **Aultman Test Code:** MISCNB
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** URIC ACID, 24 HOUR URINE
 - **Test ID:** URU24
 - **Test Includes:** Collection time, volume, creation of uric acid
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. If specimen received before 1000, results available the same day.
 - **Precollection Instructions:** No preservative. See 24-hour Urine Collection Instructions.
 - **Specimen Type:** 24-hour urine
 - **Volume:** Submit entire urine collection to lab
 - **Container:** Obtain urine collection container from lab
 - **Specimen Handling:** Keep specimen on ice during collection
 - **Storage:** Refrigerate.

-
- **Test Name:** URIC ACID, BODY FLUID
 - **Test ID:** URIBF
 - **Test Includes:** Body fluid type, uric acid
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Body fluid
 - **Volume:** 2 – 4 mL
 - **Storage:** Refrigerate.

-
- **Test Name:** URIC ACID, RANDOM URINE
 - **Test ID:** URUR

Aultman Hospital Laboratory Test Directory

- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
- **Specimen Type:** Random urine
- **Volume:** 2 mL
- **Container:** Plastic urine tube
- **Storage:** Refrigerate.

-
- **Test Name:** URIC ACID, SERUM
 - **Test ID:** URIC
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma Lithium Heparin (PST).
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis
 - **Storage:** Refrigerate.
 - **Additional Information:** **If patient is on Elite; the patient must be drawn in the main hospital Outpatient Lab only; Draw a pre-chilled green top tube. Transport specimen on ice**

-
- **Test Name:** URINALYSIS
 - **Test ID:** UA
 - **Synonyms:** UA
 - **Test Includes:** Source, color, appearance, urine dipstick for specific gravity, glucose, bilirubin, ketones, blood, pH, protein, urobilinogen, nitrate, leukocytes. Microscopic analysis will be reflected by abnormal results.
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Precollection Instructions:** A first morning void specimen is preferred
 - **Specimen Collection:** Keep on ice or refrigerate
 - **Transfer Instructions:**
 - **Specimen Type:** Random urine
 - **Volume:** 2 mL (minimum)
 - **Container:** Plastic urine tube

-
- **Test Name:** URINE FOR CYTOLOGY
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Monday-Friday received by 3:00pm, Saturday received by 11:00am. 2 working days. Certain non-Gyn cases will require additional stains and will take longer.
 - **Specimen Collection:** Use Form 308A, (Cytology Specimens). Mark the source of the specimen on the form and include any
 - **Transfer Instructions:** pertinent clinical information. Mark VOID or CATH.
 - **Specimen Type:** Collect the first part or all (not midstream) of the second or later void of the day.
 - **Volume:** Minimum of 50 mL for adult patients, 10 mL minimum for pediatric patients.

Aultman Hospital Laboratory Test Directory

- **Container:** Cytology urine container with fixative. If Microbiology or Urinalysis is also ordered, send separate specimens for these tests.
 - **Specimen Handling:** Deliver to the Cytology department. After hours, deliver specimen to the Microbiology department. Specimen Handling: Refrigerate
-

- **Test Name: VALPROIC ACID**
 - **Test ID:** VALPR
 - **Test Includes:** Time of last dose, valproic acid
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma Lithium Heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial if collected in red top.
 - **Storage:** Refrigerate.
-

- **Test Name: VANCOMYCIN**
 - **Test Includes:** Time of last dose, vancomycin
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma Lithium Heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Centrifuge and transfer serum to plastic vial collected in red top.
 - **Storage:** Refrigerate.
-

- **Test Name: VANCOMYCIN (Peak)**
 - **Test ID:** VANCP
 - **Testing Facility:** Aultman Laboratory
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
-

- **Test Name: VANCOMYCIN (Random)**
 - **Test ID:** VANCN
 - **Test Includes:** Time of last dose, vancomycin
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done 7 days/week, 24 hours/day. Results available the same day.
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** VANCOMYCIN (Trough)
- **Test ID:** VANCT
- **Testing Facility:** Aultman Laboratory
- **Specimen Type:** Plasma lithium heparin (PST)
- **Volume:** 1 mL
- **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.

-
- **Test Name:** VANILLYLMANDELIC ACID 24-HR U
 - **LC Test Code:** 004143
 - **Aultman Test Code:** VMAU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VARICELLA IMMUNE STATUS
 - **Test ID:** VARIS
 - **Synonyms:** Chicken pox immune status
 - **Test Includes:** Detection of IgG antibodies for Varicella zoster
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done twice weekly.
 - **Specimen Type:** Serum
 - **Volume:** 1 mL
 - **Container:** 7 mL SST

-
- **Test Name:** VARICELLA-ZOSTER AB, IGM
 - **LC Test Code:** 096776
 - **Aultman Test Code:** VZVM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VDRL, CSF
 - **LC Test Code:** 006445
 - **Aultman Test Code:** VDRL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VEDOLIZUMAB DRUG+ ANTIBODY
 - **LC Test Code:** 504567
 - **Aultman Test Code:** VEDOLIZ
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VEGF SERUM
 - **LC Test Code:** 117021
 - **Aultman Test Code:** VEGF
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VENOM EUROPEAN HORN
 - **LC Test Code:** 070029

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** ZINCRB
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VGCC ANTIBODY
 - **LC Test Code:** 140640
 - **Aultman Test Code:** VOLTCA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VIP PLASMA
 - **LC Test Code:** 010397
 - **Aultman Test Code:** VIP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VIRAL CULT RAPID RESPIRATORY
 - **LC Test Code:** 186015
 - **Aultman Test Code:** MISCNB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VISCOSITY, SERUM
 - **LC Test Code:** 004861
 - **Aultman Test Code:** VISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VISTASEQ APC COMPREHENSIVE
 - **LC Test Code:** 483484
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VISTASEQ LYNCH SYNDROME PANEL
 - **LC Test Code:** 483543
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN A SERUM
 - **LC Test Code:** 017509
 - **Aultman Test Code:** VITA
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN B1 (THIAMINE), BLOOD
 - **LC Test Code:** 121186
 - **Aultman Test Code:** B1WB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN B3(NIANCIN+METABOLITE)
 - **LC Test Code:** 070115

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** VITB3
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN B5
 - **LC Test Code:** 070086
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN B12
 - **Test ID:** B12
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Done daily. Results available on the same day
 - **Specimen Type:** Plasma lithium heparin (PST)
 - **Volume:** 1 mL
 - **Container:** PST. Also, acceptable SST – Separate from cells within 2 hours of collection.
 - **Specimen Handling:** Avoid hemolysis
 - **Storage:** Refrigerate.

-
- **Test Name:** VITAMIN B12 DEFICIENCY CASCADE
 - **LC Test Code:** 141503
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN B2 WHOLE BLOOD
 - **LC Test Code:** 123220
 - **Aultman Test Code:** VITB2
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN B6, Plasma
 - **LC Test Code:** 004655
 - **Aultman Test Code:** VITB6
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN B7
 - **LC Test Code:** 070097
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN C
 - **LC Test Code:** 001805
 - **Aultman Test Code:** VITC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VITAMIN D, 25-HYDROXY

Aultman Hospital Laboratory Test Directory

- **Test ID:** VIDH
- **Test Includes:** Total combined 25-Hydroxy D2 and D3
- **Testing Facility:** Aultman Laboratory
- **Turnaround Time:** Done daily.
- **Specimen Type:** Serum
- **Volume:** 1 mL minimum
- **Container:** 7ml SST
- **Specimen Handling:** Separate serum from cells within 2 hours of collection.
- **Storage:** Refrigerate.

-
- **Test Name:** VITAMIN E
 - **LC Test Code:** 070140
 - **Aultman Test Code:** VITE
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** VITAMIN K1
 - **LC Test Code:** 121200
 - **Aultman Test Code:** VITK
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** VOLATILE SCREEN, SERUM
 - **Test ID:** VOLS
 - **Testing Facility:** Reference Laboratory
 - **Specimen Handling:** 2 ML SERUM - REF
 - **Additional Information:** CALL COURIER TO TRANSPORT

-
- **Test Name:** VON WILLIEBRAND FACTOR ANTIGEN
 - **Test ID:** F8AG
 - **Testing Facility:** Aultman Laboratory
 - **Turnaround Time:** Batched and done once per week.
 - **Specimen Type:** Plasma (citrated)
 - **Volume:** 2mL
 - **Container:** (2) full blue top tubes (MUST be full); Do Not Refrigerate Blue Tops
 - **Specimen Handling:** Specimen must be processed in Aultman Lab within 4 hours of collection; alternatively, it may be drawn in Aultman Outpatient Department.

-
- **Test Name:** VON WILLEBRAND FACTOR MULTI
 - **LC Test Code:** 500148
 - **Aultman Test Code:** VWMUL
 - **LabCorp Link:** [Search | Labcorp](#)

-
- **Test Name:** VORICONAZOLE SERUM/PLASMA
 - **LC Test Code:** 700003

Aultman Hospital Laboratory Test Directory

- **Aultman Test Code:** VORCON
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VWF ACTIVITY
 - **LC Test Code:** 164509
 - **Aultman Test Code:** RISTO
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** VWF GPIBM ACTIVITY
 - **LC Test Code:** 520553
 - **Aultman Test Code:** GPIBM
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W001- IGE RAGWEED SHORT
 - **LC Test Code:** 602463
 - **Aultman Test Code:** SRAGWD
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W003-IGE RAGWEED GIANT
 - **LC Test Code:** 602964
 - **Aultman Test Code:** GRAGWD
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W008-IGE DANDELION
 - **LC Test Code:** 602968
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W009- IGE PLANTAIN, ENGLISH
 - **LC Test Code:** 602536
 - **Aultman Test Code:** ENGPLT
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W010- IGE LAMB'S QUARTERS
 - **LC Test Code:** 602537
 - **Aultman Test Code:** LAMBQU
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W011-IGE THISTLE RUSSIAN
 - **LC Test Code:** 602515
 - **Aultman Test Code:** RUSTHI
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W013-IGE COCKLEBUR

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 602970
- **Aultman Test Code:** COKBUR
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W014-IGE PIGWEED COMMON
 - **LC Test Code:** 602484
 - **Aultman Test Code:** PGWEED
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W016-IGE ROUGH MARSHELDER
 - **LC Test Code:** 602570
 - **Aultman Test Code:** RMELDR
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W017-IGE KOCHIA
 - **LC Test Code:** 602972
 - **Aultman Test Code:** FIRBSH
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W018-IGE SHEEP SORREL
 - **LC Test Code:** 602542
 - **Aultman Test Code:** SORREL
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** W020-IGE NETTLE
 - **LC Test Code:** 602973
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** WARFARIN SERUM/PLASMA
 - **LC Test Code:** 832122
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** WEST NILE VIRUS ANTIBODY SERUM
 - **LC Test Code:** 138842
 - **Aultman Test Code:** NILE
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** WEST NILE VIRUS ANTIBODY CSF
 - **LC Test Code:** 138966
 - **Aultman Test Code:** CWESTG
 - **LabCorp Link:** [Search](#) | [Labcorp](#)
-

Aultman Hospital Laboratory Test Directory

- **Test Name:** WHITE BLOOD CELL
- **Test ID:** WBC
- **Testing Facility:** Aultman Hospital
- **Turnaround Time:** STAT/URGENT 30-60min, ROUTINE 6 hours
- **Specimen Handling:** 2ml Whole Blood EDTA from Lavender tube
- **Additional Information:** Room Temperature

-
- **Test Name:** XANTHAN GUM IGE
 - **LC Test Code:** 230935
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** XT (DNA) & XR (RNA) SOLID TUMOR PD-L1 IHC
 - **Reference Lab:** TEMPUS
 - **Aultman Test Name-Pathology:** PD-L1/NGS (LUNG)
 - **Reference Lab Link:** [Order A Test - Tempus](#)

-
- **Test Name:** XYLAZINE AND MTB MS UR
 - **LC Test Code:** 703054
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZINC, PLASMA OR SERUM
 - **LC Test Code:** 001800
 - **Aultman Test Code:** ZINC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZINC RBC
 - **LC Test Code:** 070029
 - **Aultman Test Code:** ZINCRB
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZIKA VIRUS ANTIBODY IGM
 - **LC Test Code:** 163084
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZOLPIDEM (AMBIEN), SERUM
 - **LC Test Code:** 8081315
 - **Aultman Test Code:** ZOLP
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZOLPIDEM UR

Aultman Hospital Laboratory Test Directory

- **LC Test Code:** 081100
- **Aultman Test Code:** MISCNB
- **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZOLPIDEM WB
 - **LC Test Code:** 809646
 - **Aultman Test Code:** MISC
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZONISAMIDE (ZONEGRAN) SERUM
 - **LC Test Code:** 007915
 - **Aultman Test Code:** ZONIS
 - **LabCorp Link:** [Search](#) | [Labcorp](#)

-
- **Test Name:** ZNT8 ANTIBODIES
 - **LC Test Code:** 503995
 - **Aultman Test Code:** ZNT8A
 - **LabCorp Link:** [Search](#) | [Labcorp](#)